Developed by:	Approved by:
MSc Alicja Nogala Clinical Specialist @ EGZOTech	Dr Katarzyna Bienias Senior Clinical Specialist @ EGZOTech

REFERENCES OF EGZOTECH REHABILITATION DEVICES

Developed on: 2024-11-29

Approved on: 2024-11-29

1. Executive summary

The purpose of this document is to present clinical opinions on EGZOTech devices in Europe and the Asia Pacific region.

2. Table of contents

1. Executive summary	1
2. Table of contents	1
3. Luna EMG	8
3.1. Luna EMG in Europe	8
3.1.1. Belgium	8
3.1.1.1. Rehabilitation Center of Liège University CNRF	8
3.1.2. Germany	9
3.1.2.1. THERAMotion	9
3.1.2.2. Praxis für Ergotherapie	11
3.1.2.3. ErgoVision	13
3.1.2.4. Therapy Center at Spreecenter Hellersdorf	15
3.1.2.5. Bei der Arena Therapy Center	17
3.1.2.6. Aviva Berlin	19
3.1.2.7. Ergotherapy Julia Friedsam	22
3.1.2.8. Therapeuticum Gehrden	26
3.1.2.9. Move forward Ergotherapie	28
3.1.3. Italy	30
3.1.3.1. Centro Polifunzionale Don Calabria	30
3.1.4. Latvia	32
3.1.4.1. Aslimnica Rehabilitation Clinic, Riga	32
3.1.5. Czech Republic	33

3.1.5.1. University Hospital Brno	33
3.1.5.2. University Hospital in Pilsen	35
3.1.6. Poland	37
3.1.6.1. Kuyavian-Pomeranian Center of Pulmonology in Bydgoszcz	37
3.1.6.2. J. Dietl Specialist Hospital in Cracow	39
3.1.6.3. Karol Marcinkowski University Hospital in Zielona Góra	41
3.1.6.4. University Clinical Centre named after prof. K. Gibiński Medical University of Silesia in Katowice	43
3.1.6.5. SPZOZ Radzyń Podlaski	45
3.1.6.6. Military Medical Institute Warsaw	47
3.1.6.7. BONAMED sp. z o.o. Jarcewo Hospital	49
3.1.6.8. SPZZOZ Gryfice	51
3.1.6.9. Specialist Municipal Hospital Toruń	53
3.1.6.10. Głogów District Hospital	55
3.1.6.11. The Rehabilitation Center of the Musculoskeletal System "Krzeszowice"	57
3.1.6.12. Janusz Korczak Specialist Hospital Slupsk, Neurological Rehabilitation Department	59
3.1.6.13. St. John Paul II Municipal Hospital in Elbląg	61
3.1.6.14. Hospital St. Rocha Ozimek	63
3.1.6.15. Pomeranian Hospital, Gdynia	65
3.1.6.16. Physiotherapist Team in the Neurological Department in Ciechanów	67
3.1.6.17. Kraszewo-Czubaki, Neurological Rehabilitation Department	69
3.1.6.18. Upper Silesian Rehabilitation Centre "REPTY"	71
3.1.6.19. Upper-Silesian Rehabilitation Center, Kamienna Góra Department of Neurological Rehabilitation	73
3.1.6.20. Country Hospital in Chrzanów Neurological Rehabilitation Department	75
3.1.6.21. Independent Public Hospital Health Care Facility Voivodeship Specialist Hospital No. 3 in Rybnik	77
3.1.6.22. Bernadeta Wiśniowska - Head of the Rehabilitation Unit with the Neurological Rehabilitation Sub-unit	79
3.1.6.23. Department of Rehabilitation/Neurological Rehabilitation Cardinal Primate Stefan Wyszyński Voivodeship Hospital in Sieradz	81
3.1.6.24. 1st Military Clinical Hospital with Polyclinic SPZOZ in Lublin	83
3.1.6.25. AMED Upper-Silesian Medical and Rehabilitation Center	85
3.1.6.26. Krzysztof Krop - Director of the AMED Upper Silesian Center of Medicine and Rehabilitation	89
3.1.6.27. Desiderata Nursing Home	91
3.1.6.28. Rehabilitation Center GR, Bielsko Biata	93
3.1.6.29. RehStab non-public Healthcare Facility Neurological Rehabilitation Department, Limanowa	95

3.1.6.30. Health Resort Kamień Pomorski , Rehabilitation Department	97
3.1.6.31. Provincial Specialist Hospital in Olsztyn	99
3.1.6.32. Health care and rehabilitation center "Zdrowie", Częstochowa	101
3.1.6.33. John Paul II's Medical Institute In Szczecin Rehabilitation Hospital of St. Charles Borromeo Rehabilitation Centre	103
3.1.6.34. Regional Hospital in Kotobrzeg, Rehabilitation Department	105
3.1.6.35. REHAMED-CENTER Tajęcina	107
3.1.6.36. Independent Public Healthcare Facility of the Ministry of Internal Affairs and Administration with Warmian-Masurian Oncology Center in Olsztyn	110
3.1.6.37. Sanatorium Róża in Ustroń	112
3.1.6.38. Masovian Bródno Hospital	114
3.1.6.39. Holy Cross Cancer Centre in Kielce	116
3.1.6.40. "Kwitnąca" Rehabilitation and Care Center	118
3.1.6.41. Adam Nowak - Director of "Kwitnąca" Stationary Specialized Rehabilitation And Long-Term Care Center	120
3.1.6.42. Jelenia Góra Valley Voivodeship Hospital Center	122
3.1.6.43. Prof. Franciszek Łukaszczyk Oncology Center	125
3.1.6.44. Medical Center HCP	127
3.1.6.45. Center of Oncology of the Lublin Region St. Jana z Dukli	129
3.1.6.46. M. Kopernik Voivodeship Multispecialist Center for Oncology and Traumatology in Łódź	131
3.1.6.47. M. Specialist Hospital in Kościerzyna	133
3.1.6.48. Provincial specialist hospital in Biała Podlaska	135
3.1.6.49. St. Queen Jadwiga's Clinical Regional Hospital No. 2 in Rzeszów	137
3.1.6.50. "Leśna Polana" Care and Rehabilitation Center	139
3.1.6.51. "Medico" Rehabilitation Center	141
3.1.6.52. Voivodeship Hospital in Poznań	143
3.1.6.53. Białystok Oncology Center	145
3.1.6.54. University Clinical Center in Gdańsk	147
3.1.6.55. The Murcki Hospital	149
3.1.6.56. Voivodeship Specialist Hospital in Legnica	151
3.1.6.57. SPZOZ Lubartów	153
3.1.6.58. Dr. K. Jonscher Hospital	155
3.1.6.59. Nicolaus Copernicus Independent Voivodeship Hospital in Piotrkow Trybunalski	157
3.1.6.60. Powiat Health Center in Opole Lubelskie	159
3.1.6.61. "Vita" Medical Center	161
3.1.6.62. Powiat Hospital in Iława	163
3.1.6.63. University Hospital No. 2 in Bydgoszcz	165
3.1.6.64. Voivodeship Specialist Hospital in Częstochowa	167

3.1.6.65. Jelenia Góra Valley Voivodeship Hospital Center	169
3.1.6.66. Local Government Hospital in Kutno	171
3.1.6.67. Powiat Hospital in Mikołów	173
3.1.6.68. Voivodeship Hospital in Łomża	176
3.1.6.69. Voivodeship Combined Hospital in Konin	178
3.1.6.70. Centre of Physiotherapy in Zgorzelec	180
3.1.6.71. Centre of Physiotherapy in Świętochłowice.	182
3.1.6.72. The Cardinal Stefan Wyszyński Provincial Specialist Hospital	184
3.1.6.73. Non-Public Health Care Centre Sanus by Jacek Madej	187
3.1.6.74. Provincial Specialist Hospital named L. Rydygier	189
3.1.6.75. Institute of Psychiatry and Neurology in Warsaw	191
3.1.6.76. Central Clinical Hospital of the Ministry of Interior and Administration in Warsaw	193
3.1.6.77. John Paul II Subcarpathian Provincial Hospital in Krosno	195
3.1.6.78. Iwonicz Spa Resort	197
3.1.6.79. L. Zamenhof University Children's Clinical Hospital in Białystok	199
3.1.6.80. NeuroRehab Neurorehabilitation Centre	201
3.1.6.81. Mazovian Hospital in Garwolin	203
3.1.6.82. SPSK No.4 in Lublin	205
3.1.6.83. Polish Association for Persons with Intellectual Disability	207
3.1.6.84. SAWIMED Centers for Medical Care and Rehabilitation	209
3.1.6.85. SP ZOZ Łapy	211
3.1.6.86. Silesian Hospital in Cieszyn	213
3.1.6.87. University of Rzeszow	215
3.1.6.88. Manual-Med	217
3.1.6.89. Independent Public Clinical Hospital named after prof. Adam Gruca CMKP	219
3.1.6.90. Mazovian specialized hospital in Ostrołęka	221
3.1.6.91. Regional hospital in Toruń	223
3.1.6.92. WAM-CSW Clinical University Hospital in Łódź	225
3.1.6.93. Bonifraterskie Physiotherapy Center	227
3.1.6.94. Pojda Rehabilitation and Physiotherapy	229
3.1.6.95. Lower Silesian Specialist Hospital named after T. Marciniak	231
3.1.6.96. National Residential and Rehabilitation Center for people suffering from MS Dąbek	233
3.1.6.97. Voivodeship Hospital Complex named after Ludwik Perzyna in Kalisz	235
3.1.6.98. Rehabilitation Sanatorium-Hospital Complex for the Musculoskeletal System 'Gwarek'	237
3.1.6.99. District Hospital in Wyrzysk	239
3.1.6.100. Sosnowiec Municipal Hospital	241
3.2. APAC facilities	243

3.2.1. China	243
3.2.1.1. Nanjing Pukou Central Hospital	243
3.2.1.2. Shanghai Jinhui Rehabilitation Hospital	245
3.2.1.3. People's Hospital of Shiping County, Yunnan Province	247
3.3. Malaysia	249
3.3.1. Perkeso Medical Rehabilitation Centre	249
3.3.1.1. Gleneagles Hospital Penang	250
3.3.1.2. Asian Institute of Medicine, Science and Technology (AIMST University)	251
3.3.1.3. Hospital Universiti Malaysia Sabah	252
3.4. Middle East & Africa Clinics	253
3.4.1. Saudi Arabia	253
3.4.1.1. Health Care Department In Mecca Health Department At-Taif King Faisal Medical Complex	253
4. Mezos SIT	254
4.1. European Clinical Facilities	254
4.1.1. Germany	254
4.1.1.1. Aviva Berlin	254
5. Stella BIO	256
5.1. European Clinical Facilities	256
5.1.1. Germany	256
5.1.1.1. Physiotherapie Berten Nachf. Tobias Hamann	256
5.1.1.2. Medical Park Chiemsee	258
5.1.1.3. Aviva Berlin	260
5.1.1.4. Theraphysia Hellersdorf	263
5.1.1.5. Human Motion	265
5.1.1.6. Therapeuticum Gehrden	267
5.1.1.7. Move forward Ergotherapie	269
5.1.2. Czech Republic	271
5.1.2.1. University Hospital Brno	271
5.1.3. Poland	273
5.1.3.1. Functional Rehabilitation Center ORTHOS	273
5.1.3.2. COZL Rehabilitation Centre	275
5.1.3.3. Bialystok Oncology Center	277
5.1.3.4. "Kwitnaça" Rehabilitation and Care Center	279
5.1.3.5. Desiderata Nursing Home	281
5.1.3.6. UFMED Massage Studio	283
5.1.3.7. Regional Center of Sports Medicine SPORTVITA	285
5.1.3.8. Mazovian "Bródnowski" Hospital	287

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3.1. Luna EMG in Europe

3.1.1. Belgium

3.1.1.1. Rehabilitation Center of Liège University CNRF

“Of the many advantages of Luna EMG, we can mention: the active work can be started even when the muscle activity is very low, an increased cost-effectiveness (therapists can work with more patients at once).”

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Opinion

The Rehabilitation Center of Liège University has been experimenting the Luna robot for more than one year. To date, patients with neurological conditions (multiple sclerosis, strokes, and Guillain Baré syndrome), but also orthopedic patients at early stages, fully benefit from this innovative approach.

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- The patients positive attitude towards the device.
- A minimized setup time compared with other robotic systems.
- The active work can be started even when the muscle activity is very low.
- The ability for the patient to train autonomously. This specific feature is essential given that scientific research has demonstrated that the rehabilitation time is largely in short supply. By increasing the number of repetitions, Luna allows to bridge this gap while therapists do less physical labour.
- An increased cost-effectiveness (therapists can work with more patients at once).
- Data are recorded in the software and can be easily transferred in any formats.

Our close cooperation with Egzotech is also another point on which we are extremely satisfied. Because a steady evolution is what matters the most for us, we can assert that our hands-on experience is taken into great consideration.

In the province of Liège, a multicenter study on Luna EMG is due to be carried out over the next few weeks.

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3.1.2. Germany

3.1.2.1. THERAMotion

“Luna includes all the key core components of motor learning such as biofeedback, external focus, intensive active and passive repetition, shaping, positive reinforcement and capacity limit work.”

Maik Hartwig, MSc NR, OT

TheraMotion
Ergotherapy, physical therapy, dysphagia therapy

TESTIMONIAL LUNA

The robot-supported Luna therapy unit has been actively used at the TheraMotion therapy institute in Schweinfurt for nine months as part of physical therapy and ergotherapy interventions for neurologically and orthopaedically impaired clients. Under scientific, professional and practical supervision, the unit was critically tested with respect to all requirements of evidence-based motor rehabilitation.

Luna is perfectly suited to treat the most diverse of problems for the above-mentioned patient groups. The robot is characterised primarily by high flexibility and practicality. Patients suffering from the most diverse levels of problems may be treated in the most varied of initial positions, from lying on the hospital bed all the way to standing upright. The unique thing is that Luna offers multiple treatment of upper and lower extremities. A great advantage comes primarily in the inclusion of EMG-triggered functional electrostimulation. Existing limitations following orthopaedic or neurological damage may be improved purposefully both actively and passively through diverse software capabilities.

Luna includes all the key core components of motor learning such as biofeedback, external focus, intensive active and passive repetition, shaping, positive reinforcement and capacity limit work. Deficient tone such as hypotonus, hypertonus and spasticity can be easily included in treatment on the Luna and appropriately handled.

Thanks to integrated games and varied biofeedback, patients exhibit high motivation and compliance during treatment. Transfer of the achieved interventions into daily activity on the Luna and the results of the executed assessments have led in all patients to significant improvements of motor deficits.

We consider the Luna to be an important component in neurological and orthopaedic rehabilitation, based on our external evidence up to date.

Maik Hartwig, MSc NR, OT
CEO, TheraMotion
Medical device advisor
Certified study head
Textbook author
Lecturer in neurorehabilitation

Theramotion, Rossbrunnstrasse 12, 97421 Schweinfurt, Germany
www.thera-motion.de
Phone no. 0049 972 11 88 2302

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“From our point of view, Luna offers the following advantages for neurological and orthopedic rehabilitation: high flexibility, high acceptance with patients of all age groups, EMG makes it possible for patients to see the activity of their muscles on the screen - providing additional motivation.”

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Luna – Application assessment

The ergotherapeutic practice of Maik Hentschel has used the robot-controlled **Luna** biofeedback system by Egzotech for a year now with patients with neurological and orthopaedic difficulties and illnesses.

From our point of view, Luna offers the following advantages for neurological and orthopaedic rehabilitation:

- High flexibility – **Luna** may be used in the most diverse of initial positions (lying down, sitting, standing up).
- Usage for upper and lower extremities
- Usage in patients with diverse difficulty levels of limitations (passive and active training)
- Diverse options of use thanks to varied additional units (hand, arm, foot, leg, shoulder, finger, etc.)
- Luna can be applied very quickly – limited time required for individual, patient-tailored software settings
- High acceptance with patients of all age groups
- High motivation and compliance of patients thanks to integrated games
- EMG makes it possible for patients to see the activity of their muscles on the screen – providing additional motivation
- Patient progress is stored automatically and may be generated (e. g. as a report) for future use.

From our point of view, the robot-controlled **Luna** biofeedback system offers a broad range of options of use and is a valuable expansion for any practice.

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The person issuing the reference:

Maik Hentschel - Practice owner occupational therapist

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3.1.2.3. ErgoVision

“As therapists, we also perceive Luna as a relief from everyday therapy. No more tiring use of force when moving extremities, no more heavy arms and, above all, motivated patients who always react enthusiastically to the Luna.”

Marete Schmermbeck

ErgoVision
Lüdenscheid

Luna in ErgoVision Lüdenscheid

The LUNA therapy device has become an integral part of our ErgoVision Lüdenscheid practice for more than a year.

It is used several times a day to cover different treatment areas. Be it neurological, orthopedic or surgical disease's clinical picture, regardless of age, the Luna is versatile and is very much appreciated by young and old. All patients work on this device with a high level of motivation.

The high flexibility of the device is already evident from the fact that it does not matter in which position the patient works with it. It's possible to work while lying down, standing, sitting and even in combination with many other conventional therapy methods. A wide variety of applicators enable increased usability on the patient. Arms, legs, feet or hands, every muscle can be trained, every joint mobilized and every patient activated. It is customizable for every type of patient.

The versatility in the use of the Luna is readily providing a high level of luxurious work for any therapist. Both passive mobilization as well as an active subsequent exercise enable designing each therapy unit individually. The joint-gentle use for both hypotonic and hypertonic clients is effective for relaxing joint mobilization and the increase in activity resulting from it. Through the integrated games, patients learn to control and train their muscles with an external focus. An active, repetitive repetition of the movements is made possible while playing and each exercise can also be individually adapted to the level of difficulty.

The feedback from all patients is consistently positive. Even the little ones often ask when they can next save space or just pop a few bubbles. As therapists, we also perceive Luna as a relief from everyday therapy. No more tiring use of force when moving extremities, no more heavy arms and, above all, motivated patients who always react enthusiastically to the Luna.

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Lüd, 09.10.20

3.1.2.4. Therapy Center at Spreecenter Hellersdorf

“ In our opinion, this device contains all the relevant programs and capabilities for stimulating motor learning and neural plasticity, their improvement and stabilization.”

Gino Sayegh

Therapy center at Spreecenter Hellersdorf

Assessment of the test phase status: Luna

As a part of the neurological treatments we perform, we had the opportunity to test and use the "Luna" robot-assisted device in the specialized area of ergotherapy and physiotherapy. We have been using the device in our Berlin therapy office since August 2022, and have always sought and practiced interdisciplinary exchange opportunities for mutual exchange of experience. During the process, we were guided by the following criteria: workload, ability to understand and ease of use and programming of the device, comprehensibility of task performance and requirements, practical usability, results regarding motion/force/body perception.

We have used the Luna device for treating apoplexy, amyotrophic lateral sclerosis (ALS) and Parkinson's disease.

From our point of view, the Luna device is perfect for treating neurological and orthopedic conditions. In our opinion, this device contains all the relevant programs and capabilities for stimulating motor learning and neural plasticity, their improvement and stabilization. With the device, we were able to significantly improve decreased and increased tonic muscle tension, where we achieved visible and noticeable results. In addition, 80% of patients experienced a visible and marked increase in motivation, which can be justified by the fact that muscle activity is visualized on the device (visual feedback)! The only points of criticism relate to the time investment (generally 10 minutes) required to set the device, which significantly limits the standard 30-minute outpatient therapy procedure!

Thanks to the effectiveness of the Luna device, we can unequivocally recommend it for use in neurological outpatient clinics.

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Gino Sayegh

Prexisstempel/Unterschrift

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“We were particularly impressed by the observation we made during working with patients with a frozen shoulder disorder when we used the Continuous Passive Motion function (CPM). Patients were able to relax the shoulder muscles- they talked about their “first pain-free night's sleep for a long time” after passive mobilisation was applied to them using Luna EMG device. We made similar observations with other patients reporting disorders.”

Carsten Bier

LunaEMG

At our specialist Therapy Centre Bei der Arena 5 in Ingolstadt, we have been using the LunaEMG robotic device for occupational therapy in patients affected by neurological and orthopaedic disorders since 2019.

Thanks to the wide range of applications for therapeutic treatments of the lower and upper limbs using passive, active-assisted and EMG-assisted movement, the LunaEMG is a permanent part of our daily therapeutic work. We can individually adapt the level of effort and load required for treatment of each patient, thanks to the various extensions available and the possibility to perform exercises in lying, sitting and standing positions- the device is extremely universal in use. Both patients with severe neurological disorders and patients with orthopaedic disorders can be helped with hand rehabilitation. Easy and intuitive operation have given the LunaEMG device many excellent opinions among our therapists, while our patients are highly motivated to make exercises using it.

We were particularly impressed by the observation we made during working with patients with a frozen shoulder disorder when we used the Continuous Passive Motion function (CPM) . Patients were able to relax the shoulder muscles- they talked about their 'first pain-free night's sleep for a long time' after passive mobilisation was applied to them using the LunaEMG device. We made similar observations with other patients reporting disorders. We hypothesised that the confidence placed in the guidance by the machine leads to a reduction in tension and thus to a reduction in pain.

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3.1.2.6. Aviva Berlin

“Thanks to the continuous improvement process supported by the Luna device, our male and female patients experience a reduction in pain, an extension of the movement range and a regulation of muscle tension that we were not able to achieve before.”

Franziska Schlagwein

Aviva Berlin
Pfarrer-Goosmanstr. 11

12489 Berlin

The opinion on the Luna EMG device made by AVIVA Berlin in Berlin-Adlershof. AVIVA Berlin in Berlin-Adlershof is an interdisciplinary facility focused on health, movement and contact in a friendly and welcoming atmosphere. We offer dedicated therapeutic procedures in the fields of occupational therapy, physiotherapy and speech therapy as well as sports courses and medical training therapy of a rehabilitative and preventive nature.

We use the Luna EMG robotic device in our work with neurological patients and clients. The range of diseases for which we offer rehabilitation is extensive, including post-stroke patients, patients with multiple sclerosis, those suffering from nerve damage, after craniocerebral trauma and spinal injuries.

A huge advantage, which we were unable to assess sufficiently in advance, is the device's mobilising function, which is applicable to both male and female therapists. For the latter, improving patients' limbs often means a huge physical effort, as they partly deal with spasticity and often perform a huge number of repetitions of activities that exhaust their strength.

The Luna EMG takes over these activities in a targeted manner, as the device can be used in a variety of ways thanks to its various extensions, in addition to which there is the advantage of high repetition rate over fixed time intervals, so that the therapist, due to limited frame times, is hardly able to perform as much.

At this point, we must say that the device is by no means a substitute for the therapist - therapeutic knowledge is desirable in order to programme a spasticity-targeted course of action (braking when the spasticity appears and disappears until it finally reaches a value of "3" on the Ashworth scale). Thanks to the continuous improvement process supported by the Luna device, our male and female patients experience a reduction in pain, an extension of the movement range and a regulation of muscle tension that we were not able to achieve before. The ability to perform EMG-based exercises with the same device within a newly developed range of movement completes the picture of the device's overall use in therapy. The fine motor kit, which was delivered with the device, in our opinion provides a solid basis for occupational therapy, but needs to be made more specific and focused on the needs of patients and clients.

We look forward to learning more about the Luna EMG device and using it to work with and motivate patients and clients on their rehabilitation journey.

Yours faithfully

Franziska Schlagwein

Head of the Occupational Therapy Department

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LunaEMG internal report | Ergotherapie Julia Friedsam

EGZOTech's innovative robotic rehabilitation technology LunaEMG has been used in Julia Friedsam's ergotherapy practice every day since March 2023 in the context of specialised occupational therapies and training for orthopaedic, post-surgical hand, neurorehabilitation and paediatric patients. Looking from the perspective of both internal and external factual records, it is safe to say that this unique therapeutic robot meets all the requirements and criteria for a modern, innovative rehabilitation system with proven effectiveness.

The LunaEMG is ideally suited for the therapy of all patients on the neurological, orthopaedic, post-operative hand spectrum and also partly paediatric and gerontoneurological. Patients in the acute, intermediate and chronic stages of the disease with varying degrees of severity and symptoms can train with the robot in a wide range of starting positions, performing single and multi-joint, single and multi-joint movements.

LunaEMG is able to incorporate all relevant elements of motor learning, such as, for example, activity, motivation, biofeedback, external concentration, active and passive repetition, shaping, tracking results and effectiveness, as well as exercises at the limit of the patient's physical capacity. Pathological muscle tension states such as hypotonia, hypertonia and/or spasticity are not an obstacle to the use of LunaEMG, but can be taken into account intuitively, simply and purposefully.

The therapy robot requires engagement and dynamic handling by the therapist. In my opinion, however, this is not a negative feature, but a positive challenge that allows the professional skills of therapists to be developed through the complexity of LunaEMG. Practical set-up, including pre-selection of the appropriate bits, takes about five minutes.

LunaEMG covers the following practice areas:

- state-of-the-art motor rehabilitation for all disorders
- robot-assisted therapy for upper and lower limbs
- EMG
- constraint-induced necessity therapy (CIMT) and modified forms of mCIMT (for upper limbs) and

	Owner: Julia Friedsam	DE130 402 99 30	Ballbank Ost Franken eG	Erlangen Tax Office
	IN 48000880	praxis@ergotherapie-friedsam.de	DE187500 9002 0000 0364 71	Tax number: 218/218/0476
	Hauptstr. 1, 91091 Großenseebach	www.ergotherapie-friedsam.de	BIC: GEN32133030	Vat number: VAT DE350474180

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The person issuing the reference:

Julia Friedsam - Practice owner, OT, honorary
lecturer

constraint-induced necessity therapy (upper and lower limbs)

- hand therapy for orthopaedic, post-operative and neurological disorders
- repetitive hand function exercises
- training in motor skills and coordination
- observation of movement
- neuropsychological therapies (including symptoms of attention deficit disorder and related problems such as spatial-construction deficits, hemianopsia, repulsion syndromes)
- task and target exercises
- balance, coordination and strength exercises
- upper limb exercises for hemiparesis and paralysis
- basic and developmental arm exercises
- bilateral symmetrical exercises
- mobilisation and regulation of muscle tone in cases of spasticity and/or hypertonia
- evaluating and documenting performance and progress based on ICF assessments
- planning of the therapy and its goals, documentation according to ICF criteria
- professional self-monitoring exercise programmes (intermittent supervision)

LunaEMG is a unique device, extremely useful in neurorehabilitation, which helps us every day with contemporary, innovative and, above all, evidence- and results-based therapies.

"We are creating and shaping the future of rehabilitation based on actual results."

Julia Friedsam

Practice owner | OT | Associate Docent

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„We create and shape the future of rehabilitation with evidence.“

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3.1.2.8. Therapeuticum Gehrden

“Patients feel extremely motivated if they can see on the screen exactly how much they can activate their muscles. They then focus on the screen rather than the limb itself. Thanks to the games installed on the LunaEMG device, patients do not even notice how often they repeat particular movements and how intensely they exercise.”

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Therapeuticum Gehrden
Dammstraße 18
30989 Gehrden, Germany

The person issuing the reference:

Alisa Barthel - Physiotherapeutic anagement,
Owner

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Alisa Barthel und Team

3.1.2.9. Move forward Ergotherapie

“Luna has proven to be an excellent therapeutic device for patients of all severity levels. Patients receive immediate visual and audio feedback in response to their motor control.”

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“We believe that Luna EMG is a valid operative and evaluative system in the intervention with neurological subjects, as our users are.”

Barbara Montagnana

Don Calabria Multifunctional Center

We are a Rehabilitation Center includes the out-of-hospital rehabilitation facility and two Day Centers called Occupational Welfare Service operating under accreditation and in agreement with the NHS.

The out-of-hospital rehabilitation facility is a structure that offers multidisciplinary interventions aimed at the rehabilitation of people with disabilities.

The Sectors have the task of carrying out the fundamental mission of the Presidium: providing diagnosis, treatment and rehabilitation services.

The Intensive Rehabilitation Service for adolescents with disabilities (from 14 to 20 years) operates in the Development Age Sector (for people up to 20 years old).

A Service for disability acquired in the post-acute phase has been set up in the Adult Age Sector (for people over 20 years old).

Two Units operate at the service of the two Sectors: for the Evaluation and for the Analysis of movement.

The latter, since January 2019, has implemented its intervention sector with the acquisition of a series of robotic machinery for rehabilitation.

Among the equipment we have in use Luna EMG which is used for all sectors.

Specifically, we used the system to:

- evaluation protocol for treatment with botulinum toxin (work developed in a specialization thesis in psychiatry) pre- and post-inoculation

- evaluation and rehabilitation protocol for the treatment of the upper limb in patients - subacute and chronic hemiplegics

- evaluation and rehabilitation protocol for patients with Multiple Sclerosis

- evaluation and rehabilitation protocol for patients with outcomes of Infantile Cerebral Palsy, in the different developmental typologies.

The results obtained, compared in some cases also with the Movement Analysis report, have always been good and coincided with the clinical evaluations carried out.

We believe that Luna EMG is a valid operative and evaluative system in the intervention with neurological subjects, as our users are.

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3.1.4. Latvia

3.1.4.1. Aslimnica Rehabilitation Clinic, Riga

“Also I have to acknowledge that it was very satisfying to have an immediate diagnostics for each patient, also all of the necessary information about patients training was saved, so that if one PT was not present that day, other colleagues could simply repeat yesterday’s training.”

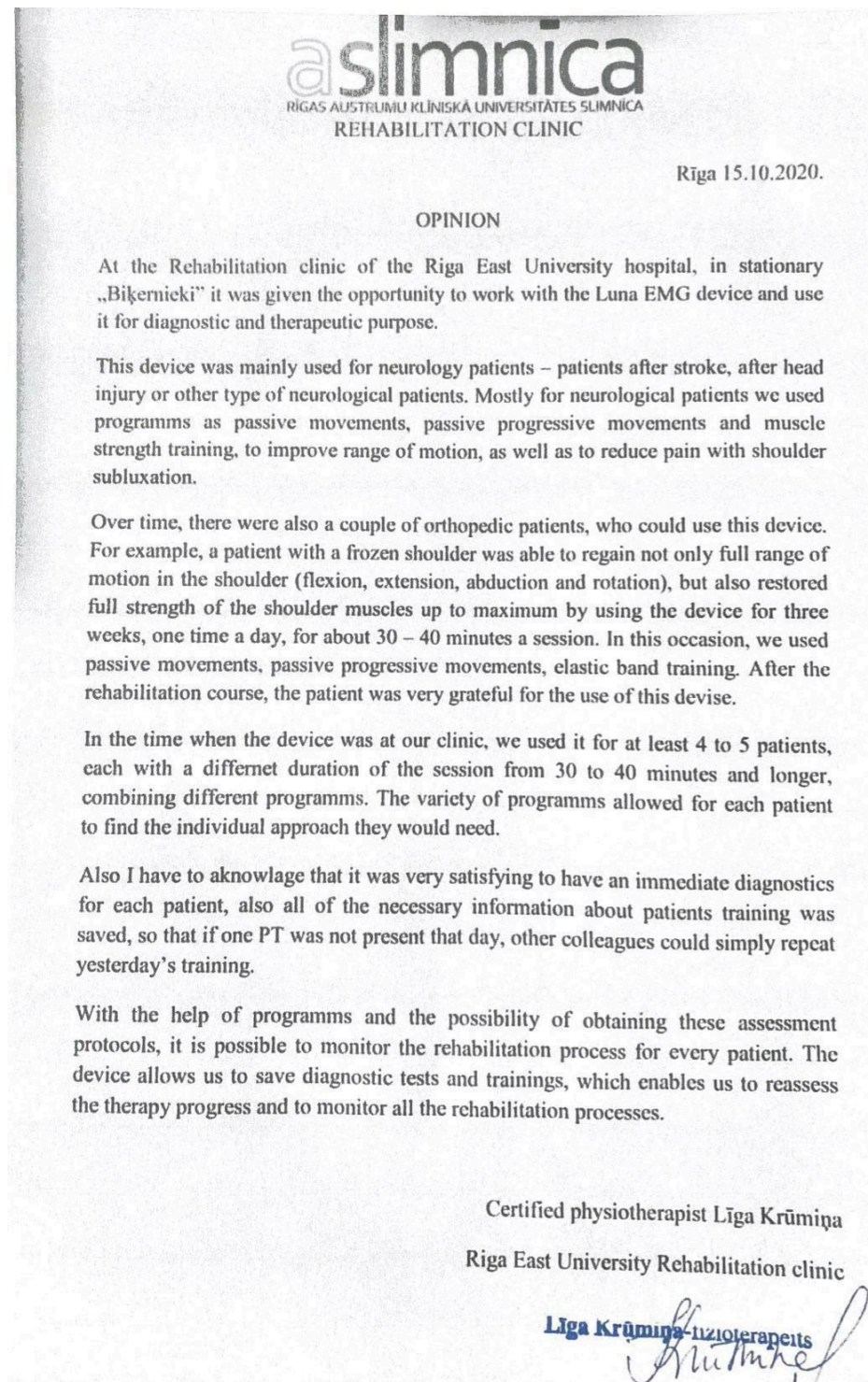
Līga Krūmiņa

Headquarters:

Rehabilitation Clinic
Riga East University Rehabilitation
Clinic
Hipokrāta iela 2
Rīga, LV-1038, Latvia

The person issuing the reference:

Līga Krūmiņa - Certified
Physiotherapist

3.1.5. Czech Republic

3.1.5.1. University Hospital Brno

“The device was highly used with a wide range of accessories and programs. The device operation is not complicated. Compared to other devices, the device is relatively small and compact.”

Marcela Nováková

Simona Šrubařová

DRO Nový Lískovec
Zoubkova 18
623 00 Brno, Czech Republic

Evaluation of the LUNA EMG device

We borrowed EGZOTech's LUNA EMG device for a month.

After thorough training, we used the device mainly with patients in order to add variety to their daily rehabilitation programs.

We used the device to strengthen weakened muscles in orthopedic, surgical and trauma patients. Thanks to different combinations, training can be set up in different modifications (weight lifting, dynamic resistance exercises, elastic resistance, etc.), therefore training even with a single movement for a long time is not monotonous.

We found the EMG reading to be a clear benefit. Setting the agonist-antagonist contribution within a movement to the role of working or relaxed muscles, adjusting their level of activation/inhibition, significantly supports muscle activation and subsequent movement, realizing the difference between muscle contraction and relaxation, training correct movement stereotypes in both healthy individuals (as tested in case of physiotherapists) and patients (neurological, internal medicine, orthopedic patients).

The device was highly used with a wide range of accessories and programs. The device operation is not complicated. Compared to other devices, the device is relatively small and compact.

The only drawback was that we did not have the original exercise couch, so that the patient could get a really good training session, without leaning down and while being in the correct position.

Thank you very much for renting the equipment.

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Mgr. Simona Šrubářová, Ph.D. 

3.1.5.2. University Hospital in Pilsen

“The device can be used for a wide range of diagnoses, the control is intuitive, and the advantage is immediate information about the effect of the treatment.”

Andrea Kunschová

Subject: Evaluation of the LUNA EMG device.

The rented Luna EMG device was tested at an inpatient rehabilitation unit for five weeks. Patients with orthopedic and neurological diagnoses were selected. We used a program of continuous passive movement, dynamic resistance exercises, elastic resistance, proprioception with visible and hidden positions, and EMG with Trigger and Hold and Trigger and Release positions. At the end of each patient's exercises, we also included games that our patients enjoyed.

We positively evaluate the use of devices for passive and active movement of all large joints within one device. After activating weakened muscles, patients with a neurological disease were able to perform full-length movements in the Trigger and Release program, which they really liked.

In some patients, we had problems with positioning the patient to the device, which led to pathological bending. Unfortunately, we were unable to assess whether this inconvenience could be ruled out by using a special positioning chair for the Luna EMG device.

Thank you for renting electromedical equipment and allowing us to look at robotic rehabilitation. The device can be used for a wide range of diagnoses, the control is intuitive, and the advantage is immediate information about the effect of the treatment.

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Translated by
SUPERTLUMACZ®
ul. Wysoka 15/17, 44-200 Rybnik

Headquarters:

University Hospital in Pilsen
Edvard Benes 1128/13
301 00 Pilzno-Bory

The person issuing the reference::

Andrea Kunschová - director of medical rehabilitation

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Edvarda Beneše 1128/13, 301 00 Plzeň - Bory
alej Svobody 923/80, 323 00 Plzeň - Lochotín
IČO 00669806, tel.: 377 401 111, 377 103 111

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ID datové schránky: pbtvj63
DIČ: CZ00669806

3.1.6. Poland

3.1.6.1. Kuyavian-Pomeranian Center of Pulmonology in Bydgoszcz

Bydgoszcz, March 3rd, 2020.

“LUNA EMG very well complements the work of the physiotherapist at every stage of rehabilitation, both in the acute phase and in the chronic phase after a vascular incident or orthopedic trauma.”
MSc Sebastian Szyper

Sebastian Szyper
Head of the Rehabilitation Department
Kuyavian-Pomeranian Center of Pulmonology
in Bydgoszcz

Opinion

Physiotherapists working in our Rehabilitation Center have successfully supplemented patients' physiotherapy with the possibilities offered by LUNA EMG. The wide use of the above mentioned equipment is ideal for therapy among patients with focal brain damage, patients after spinal cord injuries, after knee arthroplasty.

Thanks to the use of electromyography and the correct positioning of the Luna's extensions, it allows for effective learning of patient-controlled muscle contractions, which previously had a deep paresis. It also works well for improving the range of peripheral joint mobility.

LUNA EMG very well complements the work of the physiotherapist at every stage of rehabilitation, both in the acute phase and in the chronic phase after a vascular incident or orthopedic trauma. It is also worth to mention that Luna provides real assessment of the effectiveness of physiotherapy in accordance with EBM.

Headquarters:

Kuyavian-Pomeranian Center of
Pulmonology
in Bydgoszcz
Meysnera 9 Street
85-472 Bydgoszcz

Translated by Clinical Specialist at EGZOTech

EGZOTech Sp. z o.o.
ul. Romualda Traugutta 6h
44-100 Gliwice
NIP: 631-265-06-04
tel: (32) 750-49-45

The person issuing the reference:

MSc Sebastian Szyper - Head of the
Rehabilitation Department

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3.1.6.2. J. Dietl Specialist Hospital in Cracow

“The department of daily rehabilitation of the J. Dietl in Cracow, kindly informs that the purchase of the Luna robot was a really good idea. Luna robot increases the desire to exercise and mobilizes patients to work intensively on themselves, which accelerates the therapy progress. By using EMG exercises, patients get additional motivation and stimuli to intensify exercises, as they see that the limb, despite its paresis, can move through the muscles' tension.”

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Headquarters:

J. Dietl Specialist Hospital
Focha 33 Street
30-119 Kraków

The person issuing the reference:

MSc Jarosław Juźwiak- Head of physiotherapy
MSc Anna Warzęcha - Head of Rehabilitation
Department

J. Dietl Specialist Hospital
Cracow

Cracow, February 28th, 2020

Dear EGZOTech Company

The department of daily rehabilitation of the J. Dietl Specialist Hospital in Cracow, kindly informs that the purchase of the Luna robot was a really good idea.

The process of therapeutic improvement and rehabilitation has changed dramatically. Patients gained new possibilities of exercising and mobilization.

Luna robot increases the desire to exercise and mobilizes patients to work intensively on themselves, which accelerates the therapy progress. By using EMG exercises, patients get additional motivation and stimuli to intensify exercises, as they see that the limb, despite its paresis, can move through the muscles' tension. This is very important in the process of physical rehabilitation and faster recovery.

In summary, we are very happy to possess such a great equipment.

Translated by Clinical Specialist at EGZOTech

EGZOTech Sp. z o.o.
ul. Romualda Traugutta 6h
44-100 Gliwice
NIP: 631-265-06-04
tel: (32) 750-49-45

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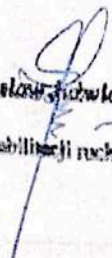
3.1.6.3. Karol Marcinkowski University Hospital in Zielona Góra

Zielona Góra, 18.03.2020

Karol Marcinkowski
University Hospital in Zielona Góra
Rehabilitation Department with the
Neurological Rehabilitation Subdivision

Opinion

“Luna EMG device has become the alternative way to conduct therapy among patients whose functional and structural state forces them to work at the bedside.”

MSc Zbigniew Guzek

We have had the LUNA EMG device in the Rehabilitation Department with the Neurological Rehabilitation Subdivision since October 2019. Until now we have used LUNA EMG in physiotherapy of patients with a history of stroke, traumatic brain injuries, spinal cord and other diseases of the central nervous system. The device has become a supplement to the current neurorehabilitation conducted in the ward. LUNA EMG allows to conduct exercises for upper limbs as well as lower limbs in a lying, sitting or standing position. It has become the alternative way to conduct therapy among patients whose functional and structural state forces to work at the bedside. Thanks to the use of EMG, the therapy is adjusted individually to the patient's current capabilities and needs. The device allows to use a variety of therapeutic programs with the ability to update them, what distinguishes it from other devices. So far 5-10 patients use LUNA EMG for the therapy daily. Summarizing, the device LUNA EMG is a very useful form of therapy and complement to the comprehensive rehabilitation the patient undergoes in our ward. It gives the opportunity to work with a neurological patient in a different age with retaining the redo functions. It can also be useful for orthopedic patients who are undergoing physiotherapy in an orthopedic and trauma ward.

Headquarters:

Karol Marcinkowski University
Hospital in Zielona Góra
Rehabilitation Department with the
Neurological Rehabilitation
Subdivision
Zyty 26 Street
65-046 Zielona Góra

Head of the Rehabilitation Department
with the Neurological Rehabilitation Subdivision

Translated by Clinical Specialist at EGZOTech

The person issuing the reference:

MSc Zbigniew Guzek- Head of the
Rehabilitation Department with the
Neurological Rehabilitation Subdivision

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3.1.6.4. University Clinical Centre named after prof. K. Gibiński Medical University of Silesia in Katowice

"Luna is a great diagnostic and therapeutic tool, which we use in our everyday work with patients. One of the biggest advantages is the possibility to use the device within variety of illnesses and in their different stages, with patients treated in our facility. Therapy with Luna EMG is effective and exciting!"

MSc MSc Jolanta Nowicka

February 28th, 2020

University Clinical Centre named after prof. K. Gibiński
Medical University of Silesia in Katowice

Department of Neurological Rehabilitation
Laboratory of Movement Rehabilitation and Physical Therapy

Opinion

Luna EMG, also known as a rehabilitation robot, is an innovative device with rich equipment, which is a great therapeutic option. It was purchased at the neurological rehabilitation department on December 9th, 2019, and shortly became an indispensable device for working with patients after strokes, multiple sclerosis, Parkinson's disease and after neurosurgical operations.

Luna is a great diagnostic and therapeutic tool, which we use in our everyday work with patients. One of the biggest advantages is the possibility to use the device within variety of illnesses and in their different stages, with patients treated in our facility. With patients with hemiplegic after stroke, we mostly use CPM program, while in case of hemiparesis we use EMG electrodes to separate muscle signal and process it into our patient's active work.

The next advantage is the documentation. We are able, in a clear and simple way, to save the training data and compare it with the previous one. Biofeedback in the form of games, allows for greater patient engagement, which translates into higher therapy effectiveness.

Therapy with Luna EMG is effective and exciting!

Translated by Clinical Specialist at EGZOTech

Headquarters:

University Clinical Centre named
after prof. K. Gibiński Medical
University of Silesia in Katowice
Ceglana 35 street
40-514 Katowice

EGZOTech Sp. z o.o.
ul. Romualda Traugotta 6h
44-100 Gliwice
NIP: 631-265-06-04
tel: (32) 750-49-45

The person issuing the reference:

MSc Jolanta Nowicka
- Head of the Motor Rehabilitation and
Physical Therapy Laboratory

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3.1.6.5. SPZOZ Radzyń Podlaski

Radzyń Podlaski, January 22nd, 2020

“Work with a patient on a Luna EMG robot can be recorded at any stage of rehabilitation, which allows for objective assessment and analysis of its progress ”

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Rehabilitation Department with the
Neurological Rehabilitation Department
Radzyń Podlaski

Opinion

We have had Luna EMG device in our neurological rehabilitation department since December, 9 2019. It has been used so far for physiotherapy in patients: after strokes, with multiple sclerosis, after spinal operations and in orthopedic patients, mostly after knee replacement.

Luna EMG is a device which enables us to carry out exercises for upper and lower limbs, and also activation of the trunk. They can take place in different starting positions – from lying in bed to standing. Electromyography is used during robot-assisted exercises. This enables the patient's active work to an extent optimally adapted to his capabilities, even with severe paresis. The work can be based on biofeedback, repeatedly actively moving, focusing on the target and dispensing difficulties.

The device gives the possibility of using a variety of therapeutic programs that can be automatically updated. Work with a patient on a Luna robot can be recorded at any stage of rehabilitation, which allows to objective assessment and analysis of its progress.

To conclude, in our opinion Luna EMG is a very useful device for the rehabilitation of patients with various motor function disorders, regardless of age, cause of disability and general condition.

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Headquarters:

SPZOZ Radzyń Podlaski
Rehabilitation Department with the
Neurological Rehabilitation
Department
Wisznicka 111 street
21-300 Radzyń Podlaski

The person issuing the reference:

Dr n. med. Edyta Rumińska- Head of
the Rehabilitation Department with
the Neurological Rehabilitation
Department in Radzyń Podlaski
Mgr Edyta Wołowik- Physiotherapist
Coordinator of Rehabilitation Department
with the Neurological Rehabilitation
Department

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3.1.6.6. Military Medical Institute Warsaw

“Thanks to the possibility of using a wide variety of therapeutic programs based on neurophysiological knowledge, it is an important element of motor learning not only for neurological but also orthopedic and other patients regardless of the patient's age. Luna EMG is one of the most innovative devices that we have had the opportunity to test over the last 10 years”.

dr Anna Olczak

Rehabilitation Clinic with the Neurological Rehabilitation Unit
Military Medical Institute

Clinical Opinion

LUNA EMG – the device is being used at Rehabilitation Unit of the Neurological Rehabilitation and Rehabilitation Department Daily Military Medical Institute in Warsaw since December 2019. Functions as part of the physiotherapy, occupational therapy, and for selected muscles activity diagnostic according to the protocol SENIAM. Luna EMG is used to work with neurological patients (after strokes and craniocerebral trauma, spinal cord injuries, Parkinson's disease, Multiple sclerosis, peripheral nerve injuries and others). Patients after stroke and with lumbar spine pain syndromes have been subjected to muscle activity diagnostic.

Luna EMG allows to conduct exercises for both upper and lower limbs, as well as the trunk activation. It is characterized by high flexibility and the possibility of practical use when working with neurological patients with varying degrees of dysfunction (both with reduced and with increased muscle tone). The robot allows also to work in various positions - from lying position to standing position, depending on the needs of patient and therapeutic purpose.

Thanks to the possibility of using a wide variety of therapeutic programs based on neurophysiological knowledge, it is an important element of motor learning not only for neurological but also orthopedic and other patients regardless of the patient's age.

Surface electromyography, which can be used during robot-assisted exercises, plays a very important role. It enables active work of the patient, even with severe paresis.

In addition, the work can be based on biofeedback, multiple passive or actively repeated movement, focused on the goal, gradation of difficulty. Important is that appropriately selected parameters, make the work optimal on the border of patient performance. LUNA EMG is one of the most innovative devices that we have had the opportunity to test over the last 10 years. Well-equipped and at the same time relatively cheap for wide range of capabilities. This device can be an important element of physiotherapy which affects the functional improvement of neurological, orthopedic and other patients at different age.

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Headquarters:

Military Medical Institute
Rehabilitation Clinic with the
Neurological Rehabilitation Unit
Szaserów 128 street
04-141 Warsaw

EGZOTech Sp. z o.o.
ul. Romualda Traugutta 6h
44-100 Gliwice
NIP: 631-255-06-04
tel: (32) 750-49-45

The person issuing the reference:

dr n. med. Józef Mróz - Head of Rehabilitation
Clinic
dr Anna Olczak-Manager of the
Physiotherapy Team

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dr n. med. Józef MRÓZ

3.1.6.7. BONAMED sp. z o.o. Jarcewo Hospital

“The therapies conducted with the use of the robot allowed, among other things, to achieve better results in terms of improving muscle strength and improving ranges of joint mobility than before. Patients are enthusiastic about the exercises with Luna EMG while they are able to observe the progress ”

MSc Karolina Kopeć

Jarcewo, February 27th, 2020

Jarcewo Hospital

REFERENCE

The EgzoTECH company has presented and delivered to our rehabilitation facility a robot Luna EMG for automatic physiotherapy and diagnostic. We use the equipment to work mainly with patients with neurological deficits : after strokes and cerebral infarctions. The therapies conducted with the use of the robot allowed, among other things, to achieve better results in terms of improving muscle strength and improving ranges of joint mobility than before. Patients are enthusiastic about the exercises with LUNA EMG while they are able to observe their progress.

We recommend cooperation with EgzoTECH to other rehabilitation centers .

Translated by Clinical Specialist at EGZO Tech

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The person issuing the reference:

MSc Karolina Kopeć- Head of Physiotherapy
Department
Barbara Bonna- Chairman of the Board

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3.1.6.8. SPZZOZ Gryfice

“Luna EMG also works well for the rehabilitation of patients after burns. In our opinion, the LUNA EMG robot combines important elements such as biofeedback , neuromuscular re-education, selection of trainings adapted to the patient's capabilities and monitoring of his progress.”

MSc Anna Malinowska

Gryfice, March 19th, 2020

Rehabilitation Department with the
Neurological Rehabilitation Unit
Gryfice

Opinion

We have had the Luna EMG device at the Rehabilitation Ward with the Neurological Rehabilitation Subdivision since 16.12.2019.

We use the robot for the rehabilitation of patients with hemiparesis after stroke, Duchenne dystrophy , after neurosurgical and orthopedic procedures. The hospital has a Burn Treatment Unit, from which patients are transferred to the Rehabilitation Department. LUNA EMG also works well for the rehabilitation of patients after burns.

The advantage of this device is the ability to control and monitor therapy along with training reports as well as the possibility to use personalized and automatic training programs.

In our opinion, the LUNA EMG robot combines important elements such as biofeedback , neuromuscular re-education, selection of trainings adapted to the patient's capabilities and monitoring of his progress.

As a rehabilitation team, we are pleased to be able to use this device and would like to thank Mrs. Dominika Kozak for the wonderful training.

Headquarters:

SPZZOZ Gryfice
Rehabilitation Department with the
Neurological Rehabilitation Unit
Niechorska 27 street
72-300 Gryfice

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The person issuing the reference:

Jan Rachwalski- Coordinator of Rehabilitation
Department
MSc Anna Malinowska- Manager of
Physiotherapy Team

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3.1.6.9. Specialist Municipal Hospital Toruń

Toruń, March 31th, 2020.

“In our Rehabilitation Department we have patients with neurological, neurosurgical and orthopedic diseases. Luna EMG complements our work with above mentioned patients. With the aid of the robot, we achieve better Prorioceptive Neuromuscular Facilitation and improved proprioception.”

MSc Marzenna Bogus-Serocka

Mikołaj Kopernik
Specialist Municipal Hospital
Department of Motor Rehabilitation

OPINION

We received the Luna EMG device in December 2019. In our Rehabilitation Department, we have patients with neurological, neurosurgical and orthopedic diseases. Luna complements our work with the aforementioned patients. With the aid of the robot, we achieve better Prorioceptive Neuromuscular Facilitation and improved proprioception. All of this using the "Trigger and release" reactive electromyography training. Thanks to interchangeable tips, we have a variety of possibilities for using the device. We use built-in rehabilitation games to increase patient involvement and motivation and improve coordination and proprioception. Luna gives you the opportunity to repeat it numerous times and exercise on your own. Training reports enable monitoring of patients' work. We are happy to have the device on site. We are aware of the great possibilities of this equipment and its use for the benefit of the patient.

Headquarters:

Mikolaj Kopernik Specialist Municipal Hospital
Department of Motor Rehabilitation
Batorego17/19 Street
87-100 Toruń

The person issuing the reference:

Msc Marzenna Bogus Serocka- Manager of
Physiotherapy Team

Translated by www.lingualab.pl
LINGUA LAB s.c.
W. Szyszkiewicz, M. Dembińska
ul. Piastowska 8A/33, 30-211 Kraków
REGON 1425422359

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3.1.6.10. Głogów District Hospital

Głogów, March 25th, 2020

“The big advantage is that each training with Luna EMG is registered in the system memory, each patient has his/her training set and we can monitor it after they are done, as well as analyze the progress at every rehabilitation stage”

Msc Genowefa Grela-Morgiel

Rehabilitation Center
Department of Neurological Rehabilitation

Opinion

We have had Luna EMG device in our neurological rehabilitation department since December 10th, 2019. It has been used in physiotherapy of our patients who endured ischemic and hemorrhage stroke in both periods, decreased and spastic tension.

Luna EMG is a device which enables us to carry out exercises for upper and lower limbs. Conducting this therapy takes place in our department, where a patient has to be in recumbent position when exercising the lower limb, and sitting position when exercising the upper one. Because of frequent pain in shoulder joints, we mostly use extensions to exercises elbow and wrist joints while for lower limbs the extension intended for ankle joints. The programs, which we most frequently use, and which our patients are willing to do are: CPM for passive exercises or with patients with muscle strength 1-2 in Lovett scale, progressive CPM for patients with spasticity, elastic resistance and orthopedic games for patients with muscle strength between 3-4. At least half of our patients exercise using Luna EMG device. Unfortunately, because of such factors like the lack of time, we do not use the electrodes while working with programs based on EMG. The big advantage is that each training with Luna EMG is registered in the system memory, each patient has his/her trainings set and we can monitor it after they are done, as well as analyze the progress at every rehabilitation stage.

To conclude, Luna EMG is an addition to our patient rehabilitation in our department. Each patient can benefit from it once a physiotherapist determines a neurological dysfunction and selects a proper program on the device.

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3.1.6.11. The Rehabilitation Center of the Musculoskeletal System “Krzyszowice”

“To conclude, Luna EMG successfully complements and differentiates therapy, helps in achieving goals, boosts motivation and gives reliable feedback for both, patients and therapists”

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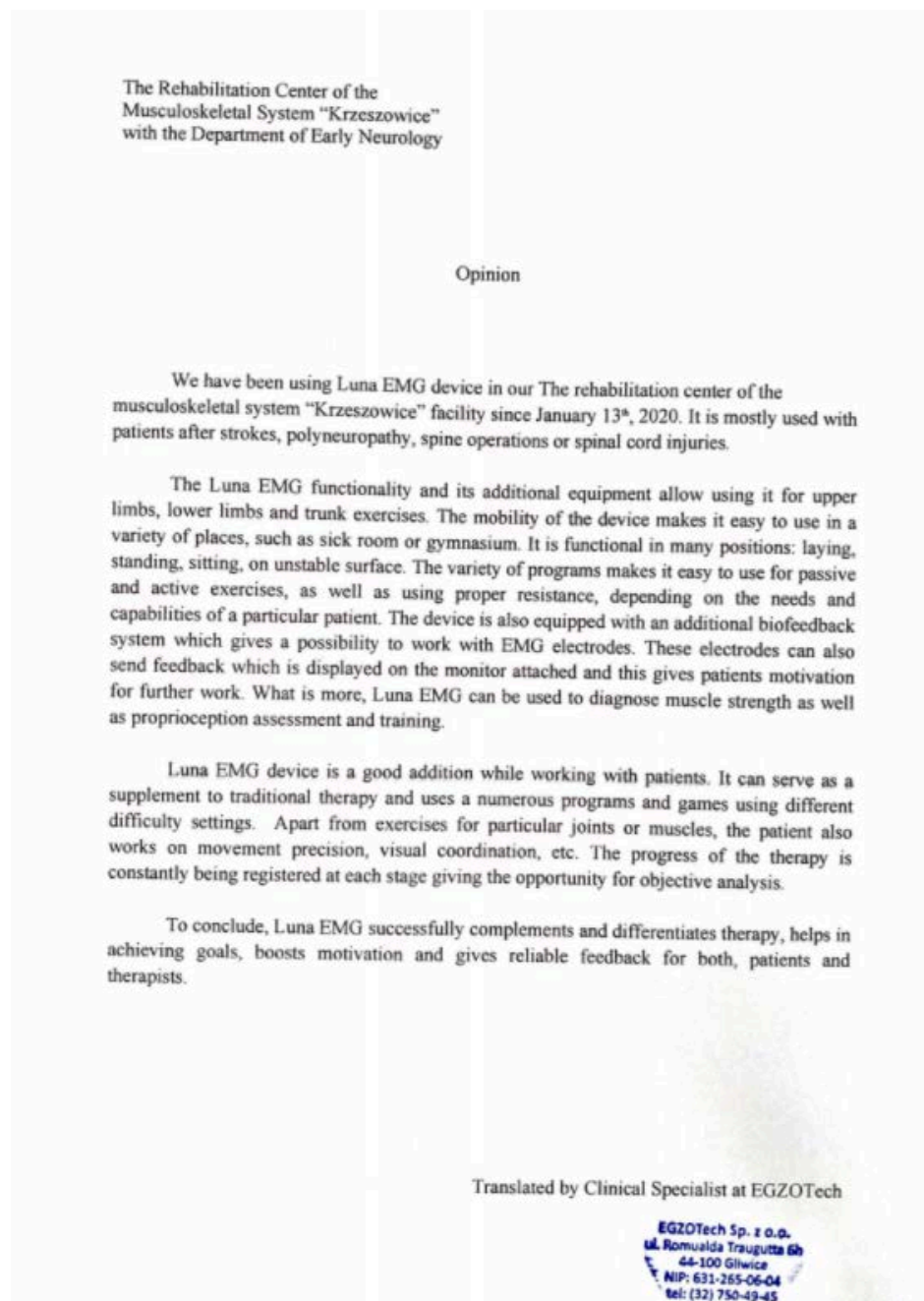
Headquarters:

The Rehabilitation Centre of the
Musculoskeletal System
with the Department of Early
Neurology
Daszyńskiego Street 1
32-065 Krzeszowice

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3.1.6.12. Janusz Korczak Specialist Hospital Slupsk, Neurological Rehabilitation Department

“Luna EMG provides you with the opportunity to work based on biofeedback, on numerous repeated movements, both passively and actively. While working with the patient, we can adjust the level exercise difficulty. Luna EMG is a very useful device for the rehabilitation of patients with various functional disorders, regardless of age and general condition”

MSc Piotr Libelt

Headquarters:

J. Korczak Regional Specialist Hospital
Rehabilitation and Neurological Rehabilitation Department
Hubalese Town Hall 1
76-200 Słupsk

The person issuing the reference:

doctor Jacek Wójcikiewicz - Director of Medicine, coordinator of the Department of Rehabilitation and Neurological Rehabilitation
mgr Piotr Libelt- physiotherapist
mgr Przemysław Łukaszuk- physiotherapist

Słupsk, April 2nd, 2020.

Janusz Korczak Specialist Hospital
Neurological Rehabilitation Department,
Rehabilitation Department

Opinion

We have had the LUNA EMG device in our department since November 2019. So far, we have used the device in physiotherapy in patients after orthopedic procedures and in neurological patients after stroke.

Luna is a robot which helps in exercises with muscles in different body areas. Thanks to the fact that the robot contains an electromyograph, it enables the patient to work in an active way adapted to his or her capabilities, even with significant paresis. Luna EMG provides you with the opportunity to work based on biofeedback, on numerous repeated movements, both passively and actively. While working with the patient, we can adjust the level exercise difficulty.

Luna EMG is a very useful device for the rehabilitation of patients with various functional disorders, regardless of age and general condition.

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3.1.6.13. St. John Paul II Municipal Hospital in Elbląg

“Thanks to its small size and easy transfer to the patient, LUNA EMG is a practical device that can be used almost anywhere. After training the personnel, the device's easy, intuitive software allows effective physiotherapist work. When one patient exercises with the robot, you can perform a task with another person. Luna EMG device is an invaluable help in our daily rehabilitation work”

MSc Anna Mazurek-Kordan

Headquarters:

John Paul II City Hospital,
Neurological Rehabilitation Unit
Komeńskiego Street 35 ,
82-300 Elbląg

The person issuing the reference:

Doctor Ewa Bieg, Head of the Rehabilitation
Ward with Neurological Rehabilitation
Sub-Ward
Anna Mazurek-Kordan MA - Senior
Physiotherapy Assistant

St. John Paul II Municipal Hospital in Elbląg
Rehabilitation Department with the Neurological Rehabilitation Unit

Opinion

We have had the LUNA EMG device in the Rehabilitation Department with the Neurological Rehabilitation Unit since 12.2019. We use it in physiotherapy in neurological patients after strokes, after neurosurgical operations, and also in patients after orthopedic surgeries.

Thanks to its small size and easy transfer to the patient, LUNA EMG is a practical device that can be used almost anywhere. After training the personnel, the device's easy, intuitive software allows effective physiotherapist work. When one patient exercises with the robot, you can perform a task with another person. LUNA EMG enables the use of various therapeutic programs, using biofeedback, in the form of rehabilitation games. These games are adapted to different levels of cognitive abilities of the patient. Patients have great fun when competing with the robot, and this results in positive motivation for rehabilitation, which is often hard and tedious. Even elderly people, with great aphasia, are very eager to do exercises with a robot. We see faster progress in increasing motor coordination, speed of motor response to stimuli, association abilities. LUNA EMG allows the initial assessment of the patient's condition, periodic progress assessment and final evaluation. In addition, all this data is stored and reproducible.

In conclusion, the LUNA EMG device is an invaluable help in our daily rehabilitation work. Moreover, it is a modern working tool willingly used by patients. Working with a robot motivates patients to work hard and allows us to see its effects.

Translated by www.lingualab.pl
LINGUA LAB s.c.
W. Szyszkiewicz, M. Dembińska
ul. Piłsnecka 5A/31, 30-211 Kraków
NIP 677-23531-15 REGON 121425850

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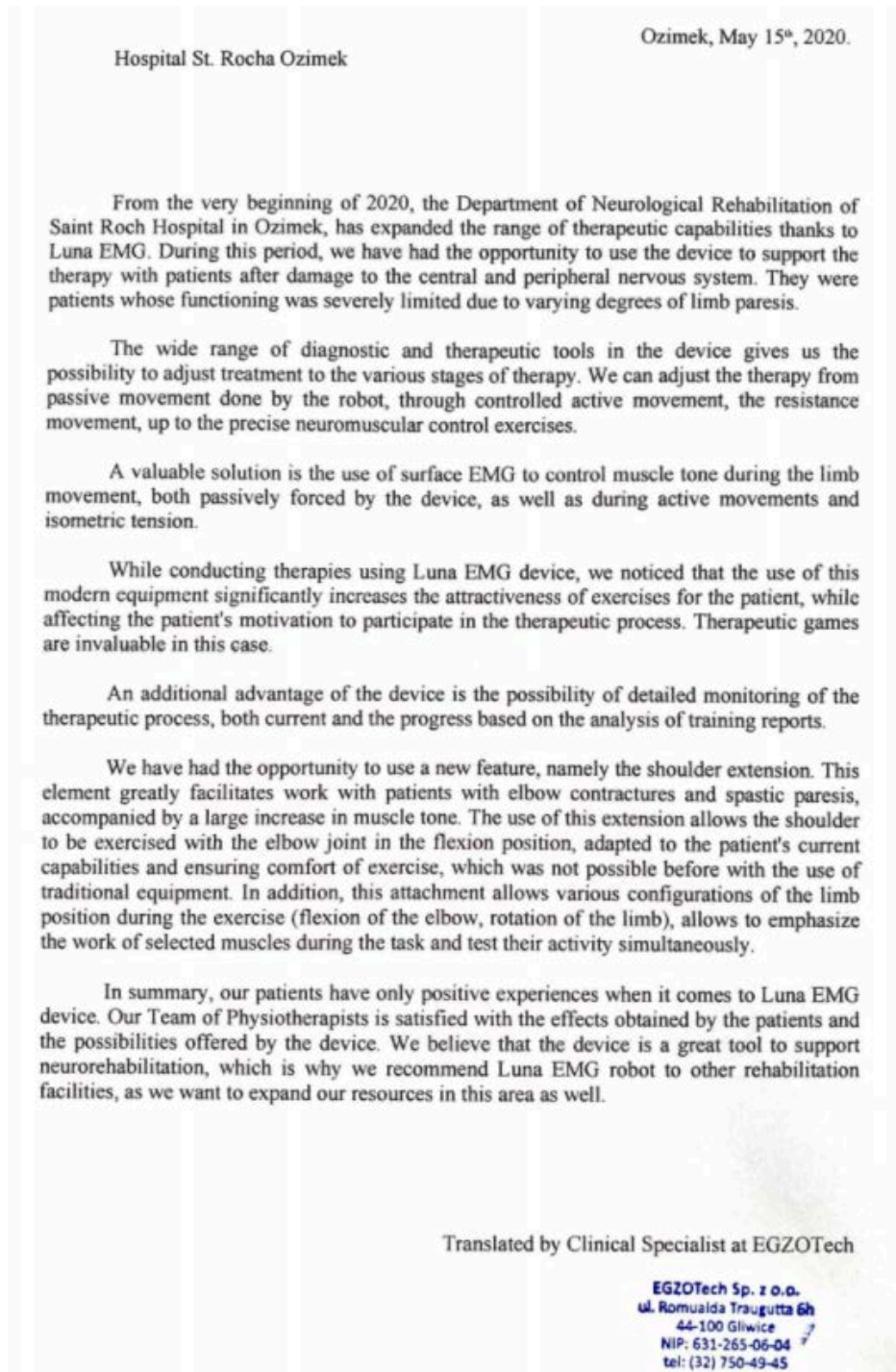
3.1.6.14. Hospital St. Rocha Ozimek

“Our patients have only positive experiences when it comes to Luna EMG device. Our Team of Physiotherapists is satisfied with effects by the patients and the possibilities offered by the device”

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3.1.6.15. Pomeranian Hospitals, Gdynia

“During the exercises, the electromyography used gives the patient the possibility of active work at a degree optimally adapted to his or her functional state. The device enables work based on biofeedback and many other therapeutic programs that automatically update themselves.”

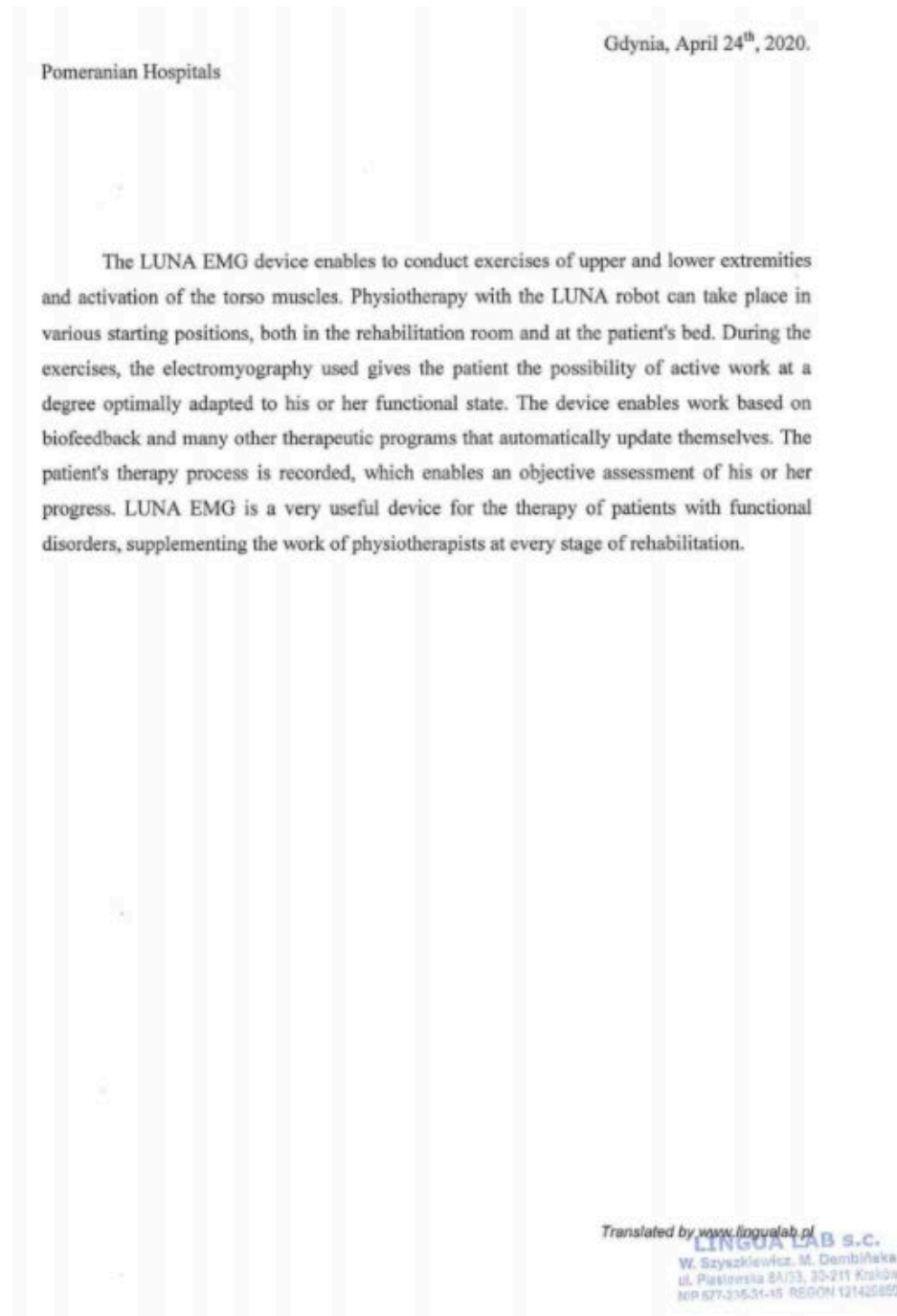
MSc Marcin Romanowski

Headquarters:

Pomeranian Hospital
January Uprising Hive 1
81-519 Gdynia

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3.1.6.16. Physiotherapist Team in the Neurological Department in Ciechanów

Ciechanów, May 10th, 2020.

Physiotherapist Team
in the Neurological Department
in Ciechanów

“Thanks to
electromyography, Luna
allows for effective learning
of patient-controlled muscle
spasms in a person who
previously suffered from
deep paresis”

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OPINION

Luna EMG perfectly supports the work of physiotherapists in the Neurology Department. The extensive use of the above equipment is suitable for the treatment of patients after stroke and other brain damage. Thanks to electromyography, Luna allows for effective learning of patient-controlled muscle spasms in a person who previously suffered from deep paresis. It also works well for achieving greater mobility of peripheral joints. Luna cannot be fooled, it is the patient who has to start moving, but she can read his or her intentions and provide help. It reacts to the energy activity that comes from the muscles. It is also worth noting that Luna provides a realistic assessment of the effectiveness of physiotherapy consistent with EBM.

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3.1.6.17. Kraszewo-Czubaki, Neurological Rehabilitation Department

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“The wide application of the Luna EMG device and the possibility of diagnostics allows for a comprehensive complementation of patient treatment and monitoring of therapy progress”

mgr Katarzyna Sienkiewicz

The LUNA EMG device from EgzoTech significantly complements the work on improving the function and biomechanics of movement in patients of our department. The patients staying in our facility are mainly those who suffered from ischemic and hemorrhagic strokes, cerebrocranial trauma, with confirmed polyneuropathy, spinal injuries resulting in paraplegia, spinal cord infarctions resulting in paraplegia and other neurological conditions.

The wide application of the Luna EMG device and the possibility of diagnostics allows for a comprehensive complementation of patient treatment and monitoring of therapy progress.

The most commonly used programs in our department are

- EMG Treatment hold
- EMG Treatment release
- proprioception
- CPM
- elastic resistance
- dynamic continuous reversal.

I believe that the Luna EMG is an excellent device for working with neurological patients, which has perfectly complemented and enriched the therapy. I strongly recommend this device and I believe that it should be placed in every department with patients in severe conditions.

Translated by www.lingualab.pl
LINGUA LAB s.c.
W. Szyszkiwicz, M. Dembińska
ul. Piastowska 8A/33, 30-211 Krosno
NP 877-236-31-15 REGON 141420450

Headquarters:

Medical Centre
Health Group
09-140 Raciąż
Karczewo-Czubaki 23a

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3.1.6.18. Upper Silesian Rehabilitation Centre "REPTY"

"Patients treated with Luna demonstrate engagement and interest in exercises performed, as well as eager to analyse with the therapist their results which are saved in reports after each training"

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The person issuing the reference:

Norbert Komatr - Director
Msc Sylwia Romanowska -
Director of Physiotherapy
Msc Katarzyna Krawczuk -
Msc of Coordinating Degree in
Neurological Rehabilitation Unit

Upper Silesian Rehabilitation Center "Repty"

Tarnowskie Góry, May 21st, 2020

From December 2019, the Department of Neurological Rehabilitation has been using Luna EMG to treat patients after neurological incidents. We use Luna EMG, among others, with patients after spinal cord injuries, strokes, polyneuropathy and with peripheral nerves paresis.

Patients treated with Luna demonstrate engagement and interest in exercises performed, as well as eager to analyze with the therapist their results which are saved in reports after each training. An additional advantage of Luna EMG are games which can be used to control any of the extension, which is especially popular among the younger patients.

Luna EMG has various programs to conduct the right therapy or to supplement it. Additional application of EMG allows the physiotherapist to objectively assess the lost function and monitor the progress in order to restore it.

In conclusion, Luna EMG is useful in physiotherapy of neurological patients in both, therapy and diagnostic.

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3.1.6.19. Upper-Silesian Rehabilitation Center, Kamienna Góra Department of Neurological Rehabilitation

Kamienna Góra, August 14th, 2020.

“Working with Luna enables more repetitions during training, increases motivation and involvement of patients and gives us feedback on progress in rehabilitation”

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OPINION

At our department, we conduct individual therapy with patients using e.g. the latest Luna EMG robot. It is a highly specialized pieces of equipment, supporting kinesitherapy of neurological and orthopedic diseases. The Luna robot therapy complements the traditional rehabilitation of patients with peripheral and CNS paralysis.

Working with Luna enables more repetitions during training, increases motivation and involvement of patients and gives us feedback on progress in rehabilitation. Working with the robot helps patients increase muscle strength, increase the range of joint mobility and improve coordination and proprioception. Thanks to built-in therapeutic programs: reactive electromyography, resistance training, passive exercises and EMG biofeedback, Luna allows therapists to select the right parameters according to the patient's needs.

Luna EMG is used to assess the patient's health by measuring the range of mobility, measuring muscle strength and electromyographic measurements. A set of interchangeable tips for individual joints and limbs allows you to adjust the training to the individual needs and abilities of the patient. Built-in orthopedic and neurological games make this form of therapy more attractive.

During the exercises, Luna collects information about the training, which is presented in the form of a report evaluating the therapy progress. According to the report, from 01.2020 to 10.08.2020, there were rehabilitation procedures performed with a total of 200 hours.

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3.1.6.20. Country Hospital in Chrzanów Neurological Rehabilitation Department

Chrzanów, September 2nd, 2020

County Hospital in Chrzanów
Neurological Rehabilitation Department

OPINION

LUNA EMG is a modern and high-performance device used in rehabilitation of neurological and orthopedic patients. Using the LUNA device during rehabilitation of patients after strokes, peripheral nerve damage and with implanted endoprostheses, we achieved very good and objective results of rehabilitation. We are pleased that we can use such an innovative method of patient rehabilitation, thanks to which we can thoroughly examine, diagnose and adjust the appropriate therapy. Having such a piece of equipment increases the quality of provided rehabilitation services. The possibility to use various rehabilitation programs (games) provides a new interesting dimension to rehabilitation. All patients who worked with the LUNA device willingly participated in the rehabilitation, and were satisfied with the therapy and rehabilitation results achieved.

The LUNA EMG device is a very useful tool in the process of rehabilitation of patients with various motor function disorders, regardless of their age, cause of disability and general condition.

Translated by www.lingualab.pl
LINGUA LAB S.C.
W. Szyszkiewicz, M. Dąbłńska
ul. Piastowska 5A/33, 30-211 Kraków
NIP 877-215-31-15 REGON 121429859

“We are pleased that we can use such an innovative method of patient rehabilitation, thanks to which we can thoroughly examine, diagnose and adjust the appropriate therapy”

dr Lucyna Wędrys, msc Agata Mędel

Headquarters:

Country Hospital in Chrzanów
Neurological Rehabilitation
Department
Topolowa Street 16
32-500 Horseradish

The person issuing the reference:

Dr Lucyna Wędrys - Chief Rehabilitation
Officer
Msc Agata Mędel - Physiotherapist

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3.1.6.21. Independent Public Hospital Health Care Facility Voivodeship Specialist Hospital No. 3 in Rybnik

“The device greatly relieved the work of physiotherapists through the possibility of diversifying the course of exercises, without the need for investments concerning additional equipment”

dr Bernadeta Wiśniowska

Independent Public Health Care Facility

Voivodeship Specialist Hospital No. 3 in Rybnik

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Luna EMG has introduced many new possibilities of working with neurological and orthopedic patients who are staying in the Rehabilitation Department with the Neurological Rehabilitation Unit in the SP ZOZ WSS No. 3 in Rybnik. The device greatly relieved the work of physiotherapists through the possibility of diversifying the course of exercises, without the need for investments concerning additional equipment. Thanks to the possibility of using different attachments, it is possible to exercise individual body parts. Luna allows you to adjust the difficulty of the exercises and the loads you are applying, individually to your needs. With the EMG function, we are able to isolate even the slightest muscle tension that initiates movement. Games using biofeedback are the most popular ones among patients. The use of the device also gave the opportunity to compare the patient's results and achievements at every rehabilitation stage.

The constant access to the Internet is a great advantage provided by Luna EMG, allowing for seamless software updates.

Headquarters:

Independent Public Hospital Health
Care Facility Voivodeship Specialist
Hospital No. 3 in Rybnik
Energy Street 46
44-200 Fisherman

Translated by www.lingualab.pl

LINGUA LAB s.c.
W. Szyszkiwicz, M. Dembińska
ul. Piastowska 8A/33, 30-211 Kraków
NIP 679.934.91.45 REGON 141422689

The person issuing the reference:

Doctor Bernadeta Wiśniowska - Head of the
Rehabilitation Ward with Neurological
Rehabilitation Sub-Department

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BGŻ BNP Paribas SA 38 1600 1055 1833 4024 4000 0001

3.1.6.22. Bernadeta Wiśniowska - Head of the Rehabilitation Unit with the Neurological Rehabilitation Sub-unit

Rybnik, 9 June 2022

“Thanks to robots, the intensity and repeatability of exercises in the patient is increased, and thus we influence the neuroplasticity of the brain, which in turn improves the clinical condition of a patient. This, in turn, allows for the shorter time of patient’s hospitalisation, so that they can return home sooner, which increases their chance of being independent in daily activities.”

At the beginning of 2022, the Rehabilitation Unit with the Neurological Rehabilitation Sub-unit in the Independent Health Care Unit, Provincial Specialist Hospital no. 3 in Rybnik began implementing the system comprising 4 Luna EMG devices – which are rehabilitation robots for neurological and orthopaedic patients as well as patients with a history of cancer treatment. In the unit, we decided to introduce a work system based on one physical therapist and more robots. This type of setting enables rehabilitation of up to 4 patients at the same time, while the appropriate quality of work is maintained and patient’s safety guaranteed. Thanks to the use of robots, we are able to provide therapy to patients with various muscle strength deficits and paresis (flaccid, spastic), using EMG and Biofeedback, which was not possible with other equipment; at the same time, under the supervision of one therapist.

We believe that the use of modern technology in health care facilities significantly increases the quality of the rehabilitation process and improves work with patients, while maintaining high quality of the therapy. Thanks to robots, the intensity and repeatability of exercises in the patient is increased, and thus we influence the neuroplasticity of the brain, which in turn improves the clinical condition of a patient. This, in turn, allows for the shorter time of patient’s hospitalisation, so that they can return home sooner, which increases their chance of being independent in daily activities. Shorter hospitalisation time also enables faster admission of new patients to the unit, and thus shortening the waiting time for the necessary rehabilitation.

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3.1.6.23. Department of Rehabilitation/Neurological Rehabilitation Cardinal Primate Stefan Wyszyński Voivodeship Hospital in Sieradz

“The equipment has electromyography capabilities (i.e. reading electrical signals from the muscles), thanks to which it is very good in neurological patients, all it takes is the patient performing a minimal muscle contraction, and LUNA will help complete the movement.”

MSc Anna Skotowska-Macha

Headquarters:

Department of
Rehabilitation/Neurological
Rehabilitation Cardinal Primate
Stefan Wyszyński Voivodeship
Hospital in Sieradz
Home Army Street 7
98-200 Sieradz

The person issuing the reference:

Doctor Barbara Cypryńska - Head of
the Rehabilitation/ Neurological
Rehabilitation Unit
Msc Anna Skotowska-Macha - Coordinator of
the Physiotherapists and Masseur
Technicians Team

Sieradz, September 21th, 2020

Department of Rehabilitation/Neurological Rehabilitation
Cardinal Primate Stefan Wyszyński Voivodeship Hospital
in Sieradz

Opinion

The LUNA device, which we have at the Department of Rehabilitation/Neurological Rehabilitation, has allowed us to expand the scope of the procedures we perform. We use them for training patients after cerebral strokes and orthopedic procedures.

The equipment has electromyography capabilities (i.e. reading electrical signals from the muscles), thanks to which it is very good in neurological patients, all it takes is the patient performing a minimal muscle contraction, and LUNA will help complete the movement. The course of exercises can be optimally adapted to the patient's health and abilities. Recording capabilities allow to track progress in therapy.

To sum up, the LUNA EMG is a very useful device for the rehabilitation of patients with various impaired motor functions. A wide range of programs allows you to prepare and adjust the range of exercises to most conditions.

Translated by www.lingualab.pl
LINGUA LAB S.C.
W. Szyszkiewicz, M. Dembińska
ul. Piastowska 8A/33, 33-211 Kraków
NIP 677-516-31-45 REGON 142420359

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3.1.6.24. 1st Military Clinical Hospital with Polyclinic SPZOZ in Lublin

“Another advantage of this device is that patients insist on in use in their rehabilitation program, and physical therapists trained to operate it have no problem prescribing an appropriate rehabilitation program.”

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1st MILITARY CLINICAL HOSPITAL WITH POLYCLINIC
Independent Public Health Care Unit
REHABILITATION UNIT
Subunit of Neurological Rehabilitation

Lublin, 28 September 2020

The Luna EMG neurorehabilitation robot has been used in the Rehabilitation Unit of the Military Clinical Hospital since October 2019.

The Luna EMG robot is a therapeutic and diagnostic device designed for the treatment of neurological patients (including patients with a history of stroke and traumatic brain injury, spinal cord injury, and suffering from multiple sclerosis) and orthopaedic patients (e.g. with a history of ligament reconstruction surgery, or implantation of endoprosthesis), as well as cancer patients, in whom various problems resulting from the underlying disease and its consequences are combined.

A unique feature of the device is software that uses surface electromyography for triggered motion assisted by the robot. This allows for active work of patients with significant neurological deficits. The robot also allows for passive resistance exercises, based on EMG feedback, including games available on the screen of the device. Diagnostic software, e.g. for proprioception, range of motion, muscle strength and innervation is also available. Additionally, EMG can be used to monitor and evaluate the bioelectrical activity of muscles in various movement patterns. The software is based on neurophysiological knowledge and fits into the concept of motor learning, while its diversity allows for its application at every stage of rehabilitation.

Another advantage is that the robot is small and mobile, which allows it to be conveniently adapted to the capabilities of the patient and their environment. Replaceable and adjustable tips, enable the therapy of the upper limb, the lower limb, and the torso, in various starting positions, from lying to standing.

To summarise, the Luna EMG robot is a very useful tool in the rehabilitation program of neurological and cancer patients, as well as patients with musculoskeletal disorders and injuries. All exercises and tests performed by patients are automatically saved and presented in the form of reports, which facilitates objective assessment and monitoring of their progress. Additionally, Luna EMG can be successfully used in scientific research, both as a therapeutic and diagnostic device. Another advantage of this device is that patients insist on its use in their rehabilitation program, and physical therapists trained to operate it have no problem to prescribe an appropriate rehabilitation program.

Headquarters:

1 st Military Clinical Hospital with
Polyclinic SPZOZ in Lublin
Ul. Al. Racławickie 23
20-049 Lublin

Translated by www.lingualab.pl
LINGUA LAB S.C.
W. Szyszkiewicz, M. Dambłńska
ul. Pastowska 5A/33, 33-211 Kraków
NIP 674-315-51-15 REGON 145112393

The person issuing the reference:

Prof. Piotr Majcher - Head of the
Rehabilitation Department
Msc. Beata Słowińska - Head of
physiotherapists and massage technicians
team

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3.1.6.25. AMED Upper-Silesian Medical and Rehabilitation Center

“The Luna EMG robot is used on a daily basis at AMED, it complements the therapy of our physiotherapists. The variety of available programs allows to adjust the training to the needs of a given patient.”

Krzysztof Krop

Katowice 14.05.2020

Opinion

In the Upper Silesian Center of Medicine and Rehabilitation AMED in Katowice, we have got 4 LUNA EMG devices and use them for diagnostic and therapeutic purpose. In our Center, we use LUNA EMG to work in the neurological department, working mainly with patients after stroke and brain injuries. Among our patients, there are people with MPD, multiple sclerosis and with spinal cord diseases and injuries. In these cases, we mostly use CPM program, reactive electromyography and proprioception training. Luna EMG device has also been used with orthopedic patients, where we use all the available programs on the device, aiming to increase strength, the range of motion and to improve movement coordination.

In Amed Center, we also treat patients after amputation. Thanks to the Luna we assess the innervation of the stump, the range of motion in the joint and muscle resistance. The device is excellent in training with reactive EMG with the use of electrodes. While working with patients after amputation we use Trigger and Hold/Release programs with two EMG channels for antagonistic muscles, as well as rehabilitation games. Thanks to that we are able to prepare the patient well to selective muscles work, needed for proper operating of bionic prosthesis.

Headquarters:

AMED Upper-Silesian Medical and
Rehabilitation Center
Ceglana 67c street
40-514 Katowice
Poland

The person issuing the reference:

Krzysztof Krop - manager of AMED
Upper-Silesian Medical and Rehabilitation
Center

In the USCM&R AMED we use Luna EMG robot every day and it serves as a therapy supplement. The diversity of the programs available allow us to adjust the training for the needs of our patients. The Biofeedback system implemented in the device gives feedback to the patient and the therapist. This is particularly favored by our patients and motivates them to work. We are very satisfied with the diagnostic possibilities of the device. The device allows us to save diagnostic tests and trainings which enables us to reassess the therapy progress and to monitor all the rehabilitation processes.

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3.1.6.26. Krzysztof Krop - Director of the AMED Upper Silesian Center of Medicine and Rehabilitation

“The implementation of Luna EMG rehabilitation robots into our center has increased the number of commercial patients utilizing our rehabilitation services. The effectiveness of Luna EMG robotic therapy allowed to increase the cost of the procedure provided using the robotic technology.”

Katowice, 17.09.2021

Krzysztof Krop

Director of the AMED Upper Silesian Center of Medicine and Rehabilitation

Opinion

We have 4 Luna EMG devices in the AMED Upper Silesian Center of Medicine and Rehabilitation in Katowice, which significantly improved the work of physiotherapists. By using the robotic technology - Luna EMG, therapists are able to effectively work with more than 1 patient at a time. Our physical therapists praise working with robots, and it is standard in the patient provided by us.

The implementation of Luna EMG rehabilitation robots into our center has increased the number of commercial patients utilizing our rehabilitation services. The effectiveness of Luna EMG robotic therapy allowed to increase the cost of the procedure provided using the robotic technology. The investment in 4 units of Luna EMG rehabilitation robots was a good decision, generating not only return of the costs, but also profit.

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Krzysztof Krop
Krzysztof Krop

3.1.6.27. Desiderata Nursing Home

“A special advantage of the Luna EMG robot is software that uses reactive electromyography, which provides the possibility of active work even to patients for whom independent movement in the therapy process.”

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Praszka, September 10th, 2020.

DESIDERATA Nursing Home

OPINION

The Luna EMG neurorehabilitation robot has been in the Desiderata Nursing Home since January 2020.

It is a device that offers diagnostic and therapeutic possibilities. The Luna EMG device is used by the staff primarily to work with neurological patients. Most of the patients of the Nursing Home are stroke survivors, but the robot is also used for the therapy of people with a history of spinal cord injury, traumatic brain injury, or other neurological disorders, such as e.g. multiple sclerosis.

A special advantage of the LUNA EMG robot is software that uses reactive electromyography, which provides the possibility of active work even to patients for whom independent movement is still impossible to perform. This increases their motivation and involvement in the therapy process. Luna EMG also offers software for passive exercises, resistance exercises, software using biofeedback and games. The latter introduce an element of fun, but also allow for the work with patients in order to improve their cognitive functions.

Thanks to the replaceable and adjustable tips, it is also possible to perform therapy of not only upper and lower limbs, but also of the torso.

Built-in diagnostic software allows for a quick and objective assessment of the range of motion, proprioception or muscle activity. All data from tests and exercises are automatically recorded so that the staff can monitor the progress of the patient.

To summarise, the Luna EMG robot is a helpful tool used in the everyday work. The diversity of software allows for a varied work with patients with different functional problems and at every stage of rehabilitation. In addition, the patients willingly participate in the exercises using the Luna EMG robot and are satisfied with their effects.

Translated by www.lingualab.pl
LINGUA LAB s.c.
W. Szyszkiewicz, M. Dembiński
ul. Piastowska 5A/33, 30-211 Kraków
NIP 877-235-31-15 REGON 121429867

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3.1.6.28. Rehabilitation Center GR, Bielsko Biała

"This device allows for optimal and objective analysis and measurements of the therapy. Thanks to this, we can use and test parameters such as: muscle strength, muscle bioelectric activity and tissue resistance during movement. "

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Bielsko-Biała, June 12th, 2020.

REFERENCES

Rehabilitation Center GR Grzegorz Gałuszka Wojciech Romanowski with its registered office in Bielsko - Biała 43-300 ul. Komorowicka 63 recommends the company **EGZOTech Sp. Z o.o.** ul. Dąbrowskiego 24 44-100 Gliwice as a reliable entity, showing great care for the customer and its needs. Furthermore, EGZOTech is open to suggestions for the development and improvement of the **Luna EMG** flagship product.

This device allows for optimal and objective analysis and measurement of the therapy. This allows us to use and parameters such as as: muscle strength, bioelectric activity of muscles and tissue resistance during movement.

Luna EMG is a device that supports rehabilitation with the patient, while biofeedback during therapy motivates the patient to continue working.

The analysis of the results in an objective way informs us about the direction and intensity of changes and the response to the stimulation of a given patient.

To sum up, based on the past cooperation with EGZOTech using the Luna EMG robot in neurorehabilitation, I would like to provide a positive testimonial, at the same time thanking for the successful cooperation on the substantive and practical level.

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3.1.6.29. RehStab non-public Healthcare Facility Neurological Rehabilitation Department, Limanowa

"In summary, the patients who used the Luna EMG device were very satisfied, willingly participated in the therapeutic process and were satisfied with the results they were able to achieve - especially when the patient with severe paresis could perform assisted movement."

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Limanowa, August 14th, 2020.

RehStab Non-public Healthcare Facility
Neurological Rehabilitation Department

OPINION

We received the Luna EMG neurorehabilitation robot in the department on January 9, 2019. The device has been and is mainly used to work with neurological patients, although orthopedic and rheumatological patients, as well as those returning to active sporting activities, also experience therapy with it.

Working with the Luna EMG has a number of applications, which provides the possibility to work on the torso, as well as on the upper and lower extremities. The use of superficial electromyography provides additional possibilities, especially in patients with severe paresis, poor muscle strength and severe limitations. The possibility of combining movement with biofeedback allows not only to diversify the physiotherapy process, but also to motivate the patient to work more efficiently and to be more involved in the therapeutic process. By using games we can introduce an element of fun and competition, which also increases motivation in many cases. This type of training with Luna has proven itself in young patients, as well as in older people.

Each training performed on Luna is recorded, which allows you to monitor the progress of a given patient on an ongoing basis, comparing not only the results but also the performance of a given training.

To sum up, the patients who used Luna EMG were very satisfied, willing to participate in the therapeutic process and were satisfied with the effects they were able to achieve - especially when a patient with severe paresis could perform assisted movement. Thanks to the great possibilities offered by the robot, we can use it to work with patients with various functional problems, at any age and at any stage of rehabilitation.

Translated by www.lingualab.pl

LINGUA LAB s.c.
W. Szyszkiwicz, M. Damblińska
ul. Piastowska 6A/33, 30-211 Kraków
NIP 677-235-31-15 REGON 121422859

Headquarters:

RehStab non-public Healthcare
Facility Neurological Rehabilitation
Department

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3.1.6.30. Health Resort Kamień Pomorski , Rehabilitation Department

“Thanks to electromyography and correct arm position, LUNA allows for effective learning of patient-controlled muscle spasms in a person who previously suffered from deep paresis.”

MSc Renata Janus

Kamień Pomorski, March 23th, 2020.

Aleksandra Bakowicz
Coordinator of the Rehabilitation Department
Kamień Pomorski

Opinion

Physiotherapists working in our center effectively complemented the physiotherapy of patients with the possibilities offered by LUNA EMG. The wide use of the foregoing equipment is perfectly suited for the treatment of patients after spinal cord injuries, the main specialty of our center.

Thanks to electromyography and correct arm positioning, LUNA allows for effective learning of patient-controlled muscle spasms in a person who previously suffered from deep paresis. It also works well for achieving greater mobility range of peripheral joints.

The device provides very good supplement for physiotherapist work at every stage of rehabilitation.

Headquarters:

Health Resort Kamień Pomorski
Polish Army Street 2
72-400 Kamień Pomorski

Translated by www.lingualab.pl
LINGUA LAB s.c.
W. Szyszkałowicz, M. Dąbrowska
ul. Piastowska 8A/33, 30-211 Kraków
NIP 877-294-51-45 REGON 121422859

The person issuing the reference:

Msc Renata Janus - Head of Rehabilitation
Department

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mgr Renata Janus

3.1.6.31. Provincial Specialist Hospital in Olsztyn

“Thanks to the use of electromyography, LUNA allows for effective learning of muscle contractions controlled by the patient. It perfectly complements the therapy at every stage of rehabilitation, both in the acute and chronic phase.”

MSc Marzena Zielińska

Headquarters:

Provincial Specialist Hospital in
Olsztyn
Żołnierska 18 Street
10-561 Olsztyn

The person issuing the reference:

Msc Marzena Zielińska - Coordinator of
physical therapists

Olsztyn, 14 April 2020

Provincial Specialist Hospital
in Olsztyn
Żołnierska 18, 10-561 Olsztyn, 8 April 2020 Olsztyn
Marzena Zielińska, MSc
Coordinator of Physical Therapists
Department of Rehabilitation
PSH in Olsztyn

Opinion

Physical therapists employed at the Department of Rehabilitation of the WSS in Olsztyn successfully supplemented the physiotherapy of their patients with additional possibilities offered by LUNA EMG. The wide application of the above-mentioned device is perfect for therapy in patients with central brain injury, and in patients with history of spinal cord injury. It is perfect for the treatment of orthopaedic patients after knee arthroplasty and after the reconstruction of the clavicular joint. Thanks to the use of electromyography, LUNA allows for effective learning of muscle contractions controlled by the patient. It perfectly complements the therapy at every stage of rehabilitation, both in the acute and chronic phase. It proves to be effective in therapy of patients with both neurological and orthopaedic disorders. LUNA provides a real, measurable and comparable assessment of the effectiveness of the conducted treatment.

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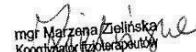
3.1.6.32. Health care and rehabilitation center "Zdrowie", Częstochowa

"EMG games, possibly involving the arm, steering wheel or the EMG itself, are an important advantage of the device, thanks to which the patients with significant muscle weakness during exercise can observe the effective of the tension within the given muscle group"

Msc Izabela Bajerska

Headquarters:

Health care and rehabilitation center
"Zdrowie"

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The person issuing the reference:

Msc Izabela Bajerska - Physiotherapist

Częstochowa, March 19th, 2020.

Health care and rehabilitation center "Zdrowie"

Detailed notes on the use of the Luna therapeutic robot

Working with Luna in cognitive patients provides you with the possibility to use all the programs available in the robot. EMG games, possibly involving the arm, steering wheel or the EMG itself, are an important advantage of the device, thanks to which the patients with significant muscle weakness during exercise can observe the effectiveness of the tension within the given muscle group. The equipment facilitates work on increasing muscle strength, coordination within antagonistic groups, increasing the motor range and allows for objective assessment of the patient's progress. The simplified EMG image allows for an easy and quick analysis of the work of the muscle groups undergoing therapy. Thanks to a clear, easy to use control panel and intuitive software, the patient can actively participate in the rehabilitation process and observe its course on an ongoing basis.

Patients with impaired consciousness had a problem with correct fixation of the Patient's hand, which resulted in patients "escaping" with their elbow off the motor axis, which in turn resulted in non-physiological movement in the remaining joints. Resistance from the Patient only causes a stop in the movement that the robot wants to complete anyway, which in some cases caused the entire machine to move and change the plane of movement in space. There is no "soft" end to the arm movement, which would be similar to the physiological movement of the human hand. The robot completes the set range of motion in the situation when the set procedure time has run out. The power cable gets tangled up and it is hard to untangle. It would be useful to have it rolling automatically, e.g. like in vacuum cleaners. During work with the robot, sometimes it was freezing, and despite the time, the robot's arm was not leading CPM movement. You had to reset the program.

In conclusion, the device is mainly suitable for the therapeutic process in cognitive patients. In case of Patients with reduced consciousness, therapy involving the Luna robot is significantly more difficult, but it can be very useful in the rehabilitation process for patients awakened from a coma.

Translated by www.lingualab.pl
LINGUA LAB s.c.
W. Szyzkiel, M. Dąbilska
ul. Piastowska 5A/33, 30-211 Kraków
NIP 677-035-31-15 REGON 121420950

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Dyjtłi M.
Formalnik K.

Bajerska Izabela
A. Sobieraj

3.1.6.33. John Paul II's Medical Institute In Szczecin Rehabilitation Hospital of St. Charles Borromeo Rehabilitation Centre

"We believe that therapy with LUNA significantly accelerates the therapeutic effects, especially in patients with flaccid paralysis of the upper limbs."

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Szczecin 18 August 2020

John Paul II's Medical Institute in Szczecin
Rehabilitation Hospital of St. Charles Borromeo
Rehabilitation Centre

Dear Ladies and Gentlemen

The tested LUNA EMG rehabilitation robot is the best ever device for upper limb therapy present in the Centre. Thanks to the courtesy of the EGZOTECH company, almost 30 patients with neurological disorders could benefit from the most modern form of therapy. Most of them felt very positive about their treatment with LUNA. Noticeable changes in the EMG recording, as well as in the function of the limb additionally motivated the Patients to continue their hard work.

We believe that the therapy with LUNA significantly accelerates the therapeutic effects, especially in patients with flaccid paralysis of the upper limbs.

My kindest regards

Mateusz Pytlak, MD, PhD

Head of the Rehabilitation Centre

Headquarters:

John Paul II's Medical Institute In
Szczecin Rehabilitation Hospital of At.
Charles Borromeo Rehabilitation
Centre

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3.1.6.34. Regional Hospital in Kołobrzeg, Rehabilitation Department

"The Luna EMG neurorehabilitation robot allows patients to perform robot-assisted physiotherapy exercises, as well as selected configurations with reactive electromyography. It is used both in rehabilitation therapy and in occupational therapy with the application of biofeedback."

Dr Bogdan Paczkowski

Kołobrzeg, 14 September 2020

Opinion

The Luna EMG neurorehabilitation robot allows patients to perform robot-assisted physiotherapy exercises, as well as selected configurations with reactive electromyography. It is used both in rehabilitation therapy and in occupational therapy with the application of biofeedback. It supports and motivates patients with different disease entities to undergo physiotherapy in a way which is interesting for them. It uses new sensory technologies to support and mobilise brain neuroplasticity, and thus restores lost functions of the nervous system. Conductive diagnostics and selected configurations with reactive electromyography enables active therapy of patients. Therapy with the use of the Luna EMG robot is a highly specialised tool enriching the techniques of physical therapists, athletic recovery instructors and physicians. Its valuable characteristic in the work of physical therapist is the possibility of repeating training, testing, and storing documentation. The robot is an innovative device, with rich equipment, and it should be present in every physiotherapy unit.

Headquarters:

Regional Hospital in Kołobrzeg,
Rehabilitation Department
Łopuskiego 31-33 Street
78-100 Kołobrzeg,
Poland

Translated by www.lingualab.pl

LINGUA LAB s.c.
W. Szyszkiłowicz, M. Dembiński
ul. Piastowska 2A/33, 50-211 Wrocław
tel. 71-374-51-45, 71-374-51-46

The person issuing the reference:

dr Bogdan Paczkowski- Head of
Rehabilitation Department

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3.1.6.35. REHAMED-CENTER Tajęcina

Tajęcina, 6.11.2019

“The Luna system uses electromyography to diagnose the bioelectrical activity of the muscles and to exercise with biofeedback. Thanks to the use of EMG, movement is possible even with the low muscle activity, while biofeedback allows for constant monitoring of motor tasks by the patient.”

dr Wojciech Rusek

The opinion on the use of the LUNA EMG device

Place of using the device: Rehamed-Centre Ltd. in Tajęcina

Duration of the use: 30.08.2018 – 06.11.2019

Number of patients subjected to diagnostics: 14

Number of therapies: 140

Types of diseases diagnosed: S06, S14, S24, S34 (according to the international classification ICD-10)

For over a year, our Rehabilitation Centre was testing the Luna EMG device for training the upper limbs, lower limbs and torso. Before starting to work independently with patients, several physiotherapists were trained in the use of the device. During several hours of training, they learned, among others how to add new patients, match the device to the patient, perform diagnostics and plan therapies.

The device is equipped with three tips for exercising the upper limbs and two tips for exercising the lower limbs, each of which is properly constructed in terms of ergonomics. The ease of selection of the tips and of the adjustment of parameters allows the Luna device to be used for exercises performed in various starting positions, both in children and adults. The quality and durability of the materials from which the device was made are praiseworthy.

The touch screen of the tablet used to operate the device is intuitive and simple. The method of control is easy and quick to learn.

The Luna system uses electromyography to diagnose the bioelectrical activity of the muscles and to exercise with biofeedback. Thanks to the use of EMG, movement is possible even with the low muscle activity, while biofeedback allows for constant monitoring of motor tasks by the patient.

Electromyography is also important to designing a rehabilitation program. Due to the fact that it is a repeatable method, it enables objective monitoring of the progress of therapy. The EMG signal is lucidly presented on the screen of the tablet.

The ability to quickly generate detailed reports is a very valuable functionality of the device. This provides clear information on the progress of therapy to the patient and to the physiotherapist.

We used the device mainly in patients after a spinal cord injury, stroke, both for diagnosis and therapy. The device enabled us to reliably measure the bioelectrical activity of the muscles and on this basis introduce modifications to improve the effects of therapy. Thanks to the function of biofeedback, the patient learned to consciously initiate the work of muscles and to maintain for an appropriate time the tension obtained during exercises.

We propose to modify the device in such a way so as to create additional games adapted to the previously performed tests.

Doctor of physical culture sciences
Wojciech Rusek
Physiotherapist

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3.1.6.36. Independent Public Healthcare Facility of the Ministry of Internal Affairs and Administration with Warmian-Masurian Oncology Center in Olsztyn

“This innovative robot provides an extra pair of hands to work, which turned out to be useful, especially during the COVID19 pandemic, when direct work with the patients was limited.”

MSc Paweł Płodzik

As a Department of Medical Rehabilitation, we are very pleased with the purchase of the LUNA rehabilitation robot. We can recommend it with a clear conscience.

This innovative robot provides an extra pair of hands to work, which turned out to be useful, especially during the COVID19 pandemic, when direct work with the patients was limited.

Luna gives us a wide range of exercise possibilities, from passive to resistance exercises for patients suffering from orthopedic and neurological problems. It also provides tremendous diversity when working with the patient over the course of several weeks.

Patients give a lot of praise to the possibility of completing the necessary physiotherapeutic activities in the form of playing games.

Yours faithfully,
ZUL Employees

Headquarters:

Independent Public Healthcare
Facility of the Ministry of Internal
Affairs and Administration with
Warmian-Masurian Oncology Center
in Olsztyn
Wojska Polskiego 37 Street
10-228 Olsztyn

LINGUA LAB s.c.
W. Szyszkiewicz, M. Dembińska
ul. Piastowska 6A/33, 30-211 Kraków
NIP 677-235-31-15 REGON 121429850

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3.1.6.37. Sanatorium Róża in Ustroń

Ustroń, 27.11.2020

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“The most frequently used functions of the device are programs of the electromyography group, the purpose of which is to learn muscle tension after prolonged immobilization. In conclusion, the Luna EMG robot is a very useful and friendly diagnostic and therapeutic device, perfectly supplements our patient therapy on a daily basis.”

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Opinion on the LUNA EMG device

The LUNA EMG robot has been used in our facility for 3 years, mainly for orthopedic patients after surgical treatment of the upper and lower limbs, as well as the spine. We also use the device in neurological patients with damage to the central and peripheral nervous system.

- The most frequently used functions of the device are programs of the electromyography group, the purpose of which is to learn muscle tension after prolonged immobilization. LUNA is both a therapeutic and diagnostic component. The CPM program works very well, replacing CPM machines, especially for the knee, foot and hip joints. These procedures are performed in low and semi-high positions during the initial period of therapy, depending on the patient's abilities. The game mode offered by the device is used in the final phase of the therapeutic session, as it is an interesting form of exercises for all patients, even the elderly ones.

In conclusion, the LUNA EMG robot is a very useful and friendly diagnostic and therapeutic device, it perfectly supplements our patient therapy on a daily basis.

LINGUA LAB s.c.
W. Szyszkiewicz, M. Dembińska
ul. Piastowska 8A/33, 30-211 Kraków
NIP 677-235-31-15 REGON 121428650

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The person issuing the reference:

Karina Krzemińska - Head of rehabilitation
Piotr Gomola - Msc in Physiotherapy

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Piotr Gomola
mgr fizjoterapii
18882

3.1.6.38. Masovian Bródno Hospital

“The LUNA EMG therapy enables precise parameter adjustment to each patient’s needs. The potential for exercise editing gives a chance for more advanced tasks along with the progress observed, which ensures the best use of the therapy time for a therapist and, certainly, beneficial effects and improved patients’ health.”

MSc Anna Koseska

Warsaw, 15 Jan 2021

Department of Rehabilitation and Physical Therapy
Mazowiecki Szpital Bródnowski [Masovian Bródno Hospital]
ul. Ludwika Kondratowicza 8
03-242 Warsaw

Opinion

LUNA EMG has been utilised in the Department of Rehabilitation and Physical Therapy since November 2020. To date, it has been used in patients with a history of neurological episodes (i.e. stroke) and in patients with orthopaedic problems (i.e. a reduced joint range of motion due to so-called frozen shoulder or rotator cuff tendon injuries).

The modern technology in LUNA EMG lets patients work on improvement concerning muscle strength, joint range of motion and neuromotor control. In view of specific functioning of the Department, our therapists mostly use CPM programs, CPM elastic resistance and EMG isometric games. The primary groups of patients referred to the LUNA EMG therapy are post-stroke patients with upper limb paresis or affected muscle tone and patients with shoulder girdle impairment of various aetiologies. The CPM program, progressive CPM and orthopaedic games were involved in the therapy. At the end of its course (a ten- or fifteen-day cycle), improved motion within the shoulder girdle (mainly flexion and abduction at the shoulder joint) in most of the orthopaedic patients were observed, while the post-stroke patients demonstrated better forearm pronation and supination. In addition, the latter group showed improved neuromotor control, particularly due to games that made the patients get strongly involved. Objective improvement indicators were the device-recorded results for each patient.

The LUNA EMG therapy enables precise parameter adjustment to each patient’s needs. The potential for exercise editing gives a chance for more advanced tasks along with the progress observed, which ensures the best use of the therapy time for a therapist and, certainly, beneficial effects and improved patients’ health.

Anna Koseska, MSc

Headquarters:

Department of Rehabilitation and
Physical Therapy
Masovian Bródno Hospital
Kondratowicza 8 Street
03-242 Warsaw

The person issuing the reference:

MSc Anna Koseska - Coordinator of the
Kinesiotherapy Department

LINGUA LAB S.C.
W. Szyszkiewicz, M. Dembińska
ul. Piastowska 8A/33, 30-211 Kraków
NIP 677-235-31-15 REGON 121429850

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mgr Anna Koseska

3.1.6.39. Holy Cross Cancer Centre in Kielce

“The Luna robot lets us increase the joint range of motion as well as improve muscle strength and motor coordination by using appropriate software and rehabilitation games. In addition, the device enables EMG diagnostic assessments to objectively evaluate functions and to track therapeutic progress.”

Dr Anna Opuchlik

*Department of Rehabilitation
ul. S. Artwińskiego 3
Head of Department
Anna Opuchlik, MD, PhD*

*www.onkol.kielce.pl
annaop@onkol.kielce.pl
tel.: 41 36 74 171 (168)*

Kielce, 7 Jan 2021

REFERENCE

We have had Luna EMG in the Department of Rehabilitation at the Holy Cross Cancer Center in Kielce since November 2020. To date, the device has been used for physiotherapy in adult neurological patients with a history of stroke and in orthopaedic patients following injuries and shoulder replacement surgery.

The Luna robot lets us increase the joint range of motion as well as improve muscle strength and motor coordination by using appropriate software and rehabilitation games. In addition, the device enables EMG diagnostic assessments to objectively evaluate functions and to track therapeutic progress.

Summing up, the Luna device is extremely useful for a broad range of patients and ensures a variety of interesting activities as part of the physiotherapy procedures as well as active patients' involvement in the kinesiotherapy process.

Yours faithfully,
Anna Opuchlik, MD, PhD
Functional Rehabilitation Specialist

Headquarters:

Holy Cross Cancer Centre in Kielce
Artwińskiego 3 Street
25-734 Kielce

The person issuing the reference:

Anna Opuchlik, MD, PhD -
Functional Rehabilitation Specialist

LINGUA LAB S.C.
W. Szyszkiewicz, M. Dembińska
ul. Piastowska 8A/33, 30-211 Kraków
NIP 677-235-31-15 REGON 121429850

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3.1.6.40. "Kwitnaça" Rehabilitation and Care Center

"Due to a wide range of possibilities and individual adjustment to the patient's needs, LUNA EMG is a great addition to the rehabilitation process, and it should be present in every rehabilitation department."

MSc Jan Hankus

Zagórzycze Dworskie, 10.02.2021

Opinion

We have the LUNA EMG device from EGZOTech in the Rehabilitation Department since 11.12.2020. So far, we have used Luna mainly for physiotherapy of patients after strokes, spinal cord injuries, peripheral nerve injuries and all kinds of orthopedic procedures.

LUNA EMG allows to restore function to the upper limbs, lower limbs and the trunk. Working with the patient can take place in lying, sitting, standing and also in non-standard positions. By using EMG, we are able to monitor the operation of superficial muscles very precisely. The activity of these muscles is also seen by the patient, making it easier for him or her to perform the motion characterized by the deficits. In the case of stroke patients or patients with paresis, LUNA EMG supports a specific movement with even minimal activity of the corresponding muscles. Each training session is preceded by a quick calibration, which allows for individual adjustment of the robot to the patient's current capabilities.

The device also works well as a diagnostic tool for the patient's muscle activity. The work with the patient is recorded at each stage of the physiotherapy process, allowing objective progress assessment.

Due to a wide range of possibilities and individual adjustment to the patient's needs, LUNA EMG is a great addition to the rehabilitation process, and it should be present in every rehabilitation department.

Head of Rehabilitation Department
MSc Jan Hankus

Headquarters:

"Kwitnaça" Rehabilitation and Care
Center
Kwitnaça 3 Street
32-091 Zagórzycze Dworskie

LINGUA LAB s.c.
W. Szyszkiewicz, M. Dembińska
ul. Sereno Fenna 6/14, 31-143 Kraków
NIP 677-235-31-15 REGON 121429850
Translated by www.lingualab.pl

The person issuing the reference:

MSc Jan Hankus -
Head of Rehabilitation Department

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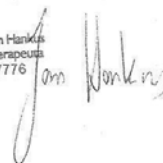
Mgr Jan Hankus

mgr Jan Hankus
Fizjoterapeuta
27776

3.1.6.41. Adam Nowak - Director of "Kwitnąca" Stationary Specialized Rehabilitation And Long-Term Care Center

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Opinion

"During this time, we conducted 120 training sessions with 4 robots running simultaneously for 69.2% of the time. The devices were operated by a maximum of one physiotherapist 99.2% of the time. Such an equipment use allows us to work with patients without interruption, even when staffing is severely limited."

We use 4 Luna EMG machines in the rehabilitation of neurological patients at the "Kwitnąca" Stationary Specialized Rehabilitation And Long-Term Care Center. Over a period of 3 months, we observed what the robot load was, as well as the number of physiotherapists needed to operate them. During this time, we conducted 120 training sessions with 4 robots running simultaneously for 69.2% of the time, 3 robots for 30% of the time, and one robot for 0.8% of the time. The devices were operated by a maximum of one physiotherapist 99.2% of the time. Patient training duration was between 30 and 60 minutes (30 minutes in most cases), with patients in various functional conditions being treated. Most included patients walking with assistance or unaided, and patients sitting unaided. Such an equipment use allows us to work with patients without interruption, even when staffing is severely limited (e.g., due to illness). With the foregoing occupancy of the equipment, the rehabilitation profits obtained are significantly increased without having to hire more physiotherapists.

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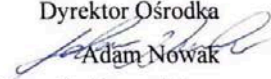
3.1.6.42. Jelenia Góra Valley Voivodeship Hospital Center

“LUNA EMG is a very popular device among the patients, because of its cutting-edge technology, attractiveness and complete absence of pain associated with the exercises. It is valued by physiotherapists for its ability to monitor the rehabilitation progress and the relief from heavy physical work provided by it.”

MD. Piotr Binek

Headquarters:

Jelenia Góra Valley Voivodeship
Hospital Center
Department of Neurological
Rehabilitation
Remote Unit
Kowary

The person issuing the reference:

MD. Piotr Binek - Acting Head of the
Department of Neurological Rehabilitation

Kowary, 11.02.2021

Department of Neurological Rehabilitation
Remote Unit
Kowary

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We have a LUNA EMG device in the Department of Neurological Rehabilitation since December 2019.

After completing the training and familiarizing with the multi-task nature of the device, the capabilities of which were tested by the rehabilitators on themselves, our medical team decided to include regular training when working with our patients.

Most patients admitted to our facility are those reporting in early stages of conditions, referred from the departments of neurology, neurosurgery, and angiology.

We have developed our own training regimen based on LUNA programs, in which we focus on working with limb spasticity, flaccid limbs and improving proprioception.

We take advantage of Luna's capabilities consisting primarily in a very precise selection of parameters for training and progress monitoring, thanks to the possibility of recording them. The painless exercises allow to increase the range of motion and strength at a satisfactory pace and improve motor coordination.

We are an early-neurological department, but our patients suffer from heavy health burden, therefore we have to deal with multi-organ dysfunction in most cases. This is why we use orthopedic programs – games and biofeedback – in our trainings, with great success.

LUNA EMG is a very popular device among the patients, because of its cutting-edge technology, attractiveness and complete absence of pain associated with the exercises. It is valued by physiotherapists for its ability to monitor the rehabilitation progress and the relief from heavy physical work provided by it.

Unfortunately, only one patient can work with LUNA at a time, which limits the number of people who can use the device. LUNA cannot be used when working with patients with dementia or those with impaired verbal contact and consciousness.

LUNA EMG is a very attractive device enjoyed by our physiotherapists and patients.

We have not been able to use all the functions of the device so far, but we gradually learn more about its capabilities.

Lek. med. Piotr Binek
Specialist Neurologist

LINGUA LAB s.c.
W. Szyszkiewicz, M. Dembińska
ul. Sereno Fenia 6/14, 31-143 Kraków
NIP 677-235-31-15 REGON 121425850

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3.1.6.43. Prof. Franciszek Łukaszczyk Oncology Center

“The LUNA EMG robot very well fitted the needs of daily work with patients suffering from neurological, orthopedic and oncological conditions. Luna EMG adapts to the needs of the patient, and at the same time provides a new quality for the physiotherapist. It is a very good tool for work, diagnostics and observation of progress in individually conducted motor therapy.”

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The person issuing the reference:

MSc Jolanta Kołodziej - Head of the Department of Rehabilitation
MSc Dorota Czwartek - Physiotherapist

Bydgoszcz, 22-01-2021

Department of Rehabilitation
Oncology Center

Opinion

The LUNA EMG robot very well fitted the needs of daily work with patients suffering from neurological, orthopedic and oncological conditions treated in the Department of Rehabilitation of the Prof. Franciszek Łukaszczyk Oncology Center in Bydgoszcz, Poland.

In our facility, LUNA EMG brings variety to kinesitherapy for patients after amputations due to oncological indications. Patients with soft tissue tumor lesions removed work more effectively to stimulate disrupted kinematic chains. Based on biofeedback, with the right rehabilitation attachments, we make sure that the established exercise is performed correctly.

People who have undergone brain tumor resections are eager to take on the tasks that the Luna EMG device puts before them. Through multi-channel stimulation, the robot accelerates the return of lost functions in neurological patients.

In patients after mastectomy, the robot is a tool to determine the active and passive range of motion of the shoulder joint.

There are more and more people unable to control their urination after cancer treatment who report to our facility. Electromyography allows us to conduct effective training of deep muscles, e.g. in male patients after prostate surgery, as well as in female patients after long-term treatment of endometrial cancer.

Luna EMG adapts to the needs of the patient, and at the same time provides a new quality for the physiotherapist. It is a very good tool for work, diagnostics and observation of progress in individually conducted motor therapy.

Yours faithfully,
Head of the Department of Rehabilitation
MSc Jolanta Kołodziej
Dorota Czwartek
Master of Physiotherapy

LINGUA LAB s.c.
W. Szyszkiewicz, M. Dembińska
ul. Sereno Fenn'a 6/14, 31-143 Kraków
NIP 677-235-31-15 REGON 121429650

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3.1.6.44. Medical Center HCP

“The LUNA EMG robot by EGZOTech is a very good device, proving useful in physiotherapist work. A number of functions and ease of use relieve the physiotherapist, diversify kinesiotherapy and accelerate the return of lost function. The robot is a state-of-the-art device improving kinesiotherapist work.”

MSc Marek Kaczmarczyk
MSc Dorota Kaniewska

Headquarters:

Medical Center HCP
Rehabilitation Laboratory
Department of Rehabilitation
28 Czerwca 1956 Street
61-485 Poznań

The person issuing the reference:

MSc Marek Kaczmarczyk - Head of the
Rehabilitation Laboratory
MSc Dorota Kaniewska - Physiotherapist

Poznań on 10.03.2021

Rehabilitation Laboratory

Opinion

Our Rehabilitation Laboratory has been using the LUNA EMG Robot for 3 months. It is an easy-to-use rehabilitation and diagnostic device from EGZOTech. With its help, we perform exercises with patients suffering from orthopedic, neurological and oncological conditions, with particular emphasis on women who underwent mastectomy. The LUNA device performs EMG diagnostics, as well as active and passive movement measurement. Diagnostics at the beginning and at the end of the therapy allow us to analyze the patient's progress.

EMG biofeedback based on measuring EMG and muscle contraction length, as well as EMG - tense-relax, tense-hold is one of the useful features offered by the device. With these functions, the patient has to control the muscle activity and consciously control the proper contraction and relaxation. The device demonstrates the advantage of the orthopedic program with the entire range of static and dynamic attachments for upper and lower limbs for passive, active and resistance exercises.

Games constitute an interesting addition, thanks to which the patient has to be focused on the task, and at the same time the exercises bring him or her a lot of joy and fun. Ability to set the attachment and the column according to the position from the last session is an extremely helpful function, which saves the therapist's time.

The device is aesthetically pleasing, easy to keep clean and it can be quickly moved to a convenient place thanks to its built-in wheels.

The LUNA EMG robot by EGZOTech is a very good device, proving useful in physiotherapist work. A number of functions and ease of use relieve the physiotherapist, diversify kinesiotherapy and accelerate the return of lost function. The robot is a state-of-the-art device improving kinesiotherapist work.

We believe that the LUNA EMG robot is worth recommending.

Head of the Rehabilitation Laboratory
MSc Marek Kaczmarczyk

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3.1.6.45. Center of Oncology of the Lublin Region St. Jana z Dukli

“Most of the time, we use the shoulder rehabilitation attachment due to the large number of patients with shoulder pain syndrome, and in women who underwent mastectomy, who suffer from limited mobility in the shoulder joint. In conclusion, our team unanimously recommends the Luna Rehabilitation Robot to work with both neurological, orthopedic and oncological patients.”

MSc Adrian Białas
MSc Paula Syty
MSc Ewelina Pytel

Lublin, 18.02.2021

Rehabilitation Center
Center of Oncology
of the Lublin Region St. Jana z Dukli

We have had the Luna rehabilitation robot in the Rehabilitation Center since December 2020. It is used in rehabilitation of orthopedic patients with pain syndromes of the shoulder, elbow, wrist, knee and foot joints, as well as in oncological patients. Luna EMG allows conducting active exercises, resistance and proprioception exercises.

It is also possible to perform a diagnostic test on it using electromyography, as well as training using electrodes for EMG. Thanks to the firmly placed attachment, the exercised part of the patient's body is well isolated and stabilized. Exercises can be performed in sitting, lying and standing positions. Biofeedback in the form of an on-screen graph allows to monitor and evaluate the patient's current task and progress. The ability to exercise while playing makes the exercise more enjoyable, as the patient focuses his or her attention on the game.

The programs we use most often are progressive CPM, elastic resistance, dynamic continuous reversal, visible and latent proprioception, as well as orthopedic games. Due to the small number of neurological patients, we use EMG occasionally to check the neural activity of muscles; we recommend reactive EMG trigger & hold as the exercise involving EMG.

Most of the time, we use the shoulder rehabilitation attachment due to the large number of patients with shoulder pain syndrome, and in women who underwent mastectomy, who suffer from limited mobility in the shoulder joint.

In conclusion, our team unanimously recommends the Luna Rehabilitation Robot to work with both neurological, orthopedic and oncological patients.

Head of the Rehabilitation Center
MSc Adrian Białas

MSc Paula Syty
Physiotherapist

MSc Ewelina Pytel
Physiotherapist

LINGUA LAB S.C.
W. Szyszkiewicz, M. Dembińska
ul. Sereno Fenn'a 6/14, 31-143 Kraków
NIP 677-235-31-15 REGON 121429650

Translated by www.lingualab.pl

Headquarters:

Center of Oncology of the Lublin
Region St. Jana z Dukli
Rehabilitation Center
Dr. K. Jaczewskiego 7 Street
20-090 Lublin

The person issuing the reference:

MSc Adrian Białas - Head of the Rehabilitation
Center
MSc Paula Syty - Physiotherapist
MSc Ewelina Pytel - Physiotherapist

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mgr Paula Syty
Paula Syty
fizjoterapeuta
709

Ewelina Pytel
mgr fizjoterapii
26481

3.1.6.46. M. Kopernik Voivodeship Multispecialist Center for Oncology and Traumatology in Łódź

“This solution makes "Luna" a very versatile and universal device. Each facility or therapist will find functions and possibilities thanks to which the work will provide more satisfaction and improve the effects of therapy.”

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M. Kopernik Voivodeship Multispecialist Center
for Oncology and Traumatology in Łódź
Voivodeship Rehabilitation Outpatient Clinic

Voivodeship Rehabilitation Outpatient Clinic at M. Kopernik Voivodeship Multispecialist

Center for Oncology and Traumatology in Łódź has been using Egzo Tech "Luna" device for six months.

Without a doubt, Luna has become a useful tool in the therapy of patients with various diseases and dysfunctions encountered by physiotherapists in our facility. According to unanimous opinion of therapists using the device, it has many possibilities and functions which make the therapy more attractive and less monotonous.

The appearance of the device, as well as its small size, are its advantages, also arousing interest and curiosity among patients. The possibility of using "Luna" for both guided, as well as EMG stimulation exercises is its extremely positive feature. The option to easily change the functional attachments, thus adapting the device to the needs of different types of patients, is also a big plus.

The option of therapeutic games provides a variety eagerly used by our patients. Thanks to those games, you can see much greater commitment and willingness to exercise in patients.

The device has a very extensive database of therapeutic programs in its menu. This solution makes "Luna" a very versatile and universal device. Each facility or therapist will find functions and possibilities thanks to which the work will provide more satisfaction and improve the effects of therapy.

Head of the Medical Technician Team
MSc Katarzyna Szymankiewicz

MSc Tomasz Błaszczyk
Physiotherapist

Headquarters:

M. Kopernik Voivodeship
Multispecialist Center for Oncology
and Traumatology in Łódź
Voivodeship Rehabilitation
Outpatient Clinic
Pabianicka 62 Street
93-513 Łódź

LINGUA LAB S.C.
W. Szyszkiewicz, M. Dembińska
ul. Sereno Fenn'a 6/14, 31-143 Kraków
NIP 677-235-31-15 REGON 121429850

Translated by www.lingualab.pl

The person issuing the reference:

MSc Katarzyna Szymankiewicz - Head of the
Medical Technician Team
MSc Tomasz Błaszczyk - Physiotherapist

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3.1.6.47. M. Specialist Hospital in Kościerzyna

“In addition to the obvious effects of the rehabilitation process, the Luna robot also offers the possibility of comparing trainings. It is useful for observing the progress of individual therapy and contributes significantly to the attractiveness of physiotherapy and its modernity.”

MSc Zbigniew Kucki

Kościerzyna, 13 April 2021

Opinion

Since January 2021, the Rehabilitation Centre in the Specialised Hospital in Kościerzyna has been using the Luna EMG rehabilitation and diagnostic robot.

This robot assists in the therapy of cancer patients. To date, it has been most commonly used to improve the quality of life and self-care of mastectomy patients. The goal of robotic kinesitherapy for this group of patients was most often to improve the range of motion of the shoulder joint. A significant effect of using Luna EMG is an improvement in muscle strength in the chest and limb on the operated side, and an increase in motor coordination of the torso and upper limb.

Patients used the following programmes:

- CMP - passive exercise
- progressive CMP - the exercise gradually increases the range of motion until it reaches the maximum for the day
- apparent proprioception
- hidden proprioception
- games

In addition to the obvious effects of the rehabilitation process, the Luna robot also offers the possibility of comparing trainings. It is useful for observing the progress of individual therapy and contributes significantly to the attractiveness of physiotherapy and its modernity.

Headquarters:

Specialist Hospital in Kościerzyna
Department of Rehabilitation
Piechowskiego 36
83-400 Kościerzyna

The person issuing the reference:

MSc Zbigniew Kucki - Head of the
Rehabilitation Department

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3.1.6.48. Provincial specialist hospital in Biała Podlaska

Biała Podlaska, 13 June 2021

“We believe that this device is very helpful in the rehabilitation process and we intend to gradually expand its use in our centre.”

Dr Anna Polańska

OPINION

Our experience with the LUNA robot by EGZOTech in the Neurological Rehabilitation Subunit has been relatively short-lived – from September 2020. The unit itself is also very small (6 beds) and it is focused on early post-stroke rehabilitation, hence our patients are overwhelmingly geriatric.

Therefore, we face the limitation of the robot's application because we can use this device for only those patients who cooperate and communicate in a simple manner. Another limitation that we encountered is the very severe spasticity of the paretic limbs of rehabilitated patients.

In spite of the above-mentioned problems, staffing limitations and a little experience with this device, we use the LUNA robot when working with minor stroke patients, younger age groups and those with peripheral nervous system damage (2 patients with GBS).

To date, the LUNA's training programmes used by us include CPM, biofeedback, proprioception exercises and games applied in occupational therapy.

Unfortunately, we rarely use the EMG-assisted movement, however, the enthusiasm of the physiotherapy team suggests an improvement in this area.

We are aware of the fact that there is an amazing range of options offered by the LUNA robot and at the moment we minimally use them. We believe that this device is very helpful in the rehabilitation process and we intend to gradually expand its use in our centre.

Deputy Head of Rehabilitation
in the Department of Neurology
with Stroke Subunit and
Neurological Rehabilitation Subunit

Headquarters:

Provincial specialist hospital in Biała
Podlaska
Department of Neurology
with Stroke Subunit and
Neurological Rehabilitation Subunit
Terebelska 57-65
21-500 Biała Podlaska

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The person issuing the reference:

Dr Anna Polańska - Deputy Head of
Rehabilitation

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Anna Polańska

3.1.6.49. St. Queen Jadwiga's Clinical Regional Hospital No. 2 in Rzeszów

"Thanks to this robot, patients can achieve their goals, namely, they can obtain an increase in the range of motion, better coordination, muscle strength and thus they can slow down the course of chronic disease or they can improve their health. Even patients with severe muscle weakness can actively exercise, which has a psychotherapeutic effect."

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Headquarters:

St. Queen Jadwiga's Clinical Regional Hospital No. 2 in Rzeszów
Rehabilitation Clinic with a Subunit of Early Neurological Rehabilitation
Lwowska 60
35-301 Rzeszów

The person issuing the reference:

Dr Grzegorz Przysada - Deputy Head of Rehabilitation

St. Queen Jadwiga's Clinical Regional Hospital No. 2 in Rzeszów
Address: ul. Lwowska 60, 35-301 Rzeszów

Rzeszów, 22 July 2021

Opinion

The Clinical Rehabilitation Unit with the Early Neurological Rehabilitation Subunit is in possession of the LUNA EMG device. The robot by EGZOTech plays an increasingly important role in our rehabilitation process.

Thanks to this robot, patients can achieve their goals, namely, they can obtain an increase in the range of motion, better coordination, muscle strength and thus they can slow down the course of chronic disease or they can improve their health. Even patients with severe muscle weakness can actively exercise, which has a psychotherapeutic effect.

Having completed the training, we are fascinated by the wide range of options of this device and we attempt to use it to the full extent of its possibilities. Therapeutic programmes, such as orthopaedic and neurological programmes, or even games add variety to the rehabilitation process and they are an interesting exercise alternative for both neurological and orthopaedic patients.

The Luna EMG also provides diagnostic capability for patients, which is extremely helpful information for the entire therapy planning team.

The analysis of therapy progress is a useful feature of this device, which is appreciated by patients since it gives the possibility to compare the effects of training at the beginning and at the end of the rehabilitation process. Luna is a mobile device, which is a great advantage of this robot.

For our team, the Luna robot is a very helpful complement to the rehabilitation process of the most demanding patients.

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3.1.6.50. "Leśna Polana" Care and Rehabilitation Center

"Luna EMG is an innovative, multi-functional device. It allows training of patients with different levels of fitness: from weak, lying patients to patients requiring strengthening therapy with additional load."

MSc Aleksandra Badtke

"Leśna Polana"
Care and Rehabilitation Center
ul. Leśna Polana 1
72-300 Rzęskowo

Opinion

Luna EMG is used at the "Leśna Polana" Care and Rehabilitation Center primarily among patients with neurological deficits. Therapy is provided using a variety of interchangeable extensions. In patients with post-stroke pain syndrome, a shoulder joint extension has been successfully used to allow very precise positioning of the patient's limb during therapy. The ankle extension, which is used to train patients with a foot drop, also works very well. In addition, the use of EMG-activity-based assisted programs allows for active patient participation in the exercises and better motor control when exercising in cases of foot drop.

At our center, the Luna EMG robot is also used to successfully provide pelvic floor muscle therapy based on Biofeedback programs.

Luna EMG is an innovative, multi-functional device. It allows training of patients with different levels of fitness: from weak, lying patients to patients requiring strengthening therapy with additional load.

In conclusion, we are pleased with the possibilities provided by Luna EMG in our daily therapeutic work. It perfectly complements and diversifies the therapy conducted at the center.

Headquarters:

"Leśna Polana" Care and
Rehabilitation Center
Leśna Polana 1
72-300 Rzęskowo

LINGUA LAB S.C.
W. Szyszkiewicz, M. Dembińska
ul. Sereno Failla 8/14, 31-143 Kraków
NIP 877-235-31-16 REGON 121429850

Translated by www.lingualab.pl

The person issuing the reference:

MSc Aleksandra Badtke - Head of the
Rehabilitation Department

MTP Gryfice Sp.z o.o.
ul. Leśna Polana 1, 72-300 Gryfice/Rzęskowo
tel.91 385 67 00 fax.91 385 67 02
KRS: 0000471947 NIP: 5313652893 REGON: 146799493

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3.1.6.51. "Medico" Rehabilitation Center

"Task-based training with Luna is not only an engaging therapy, but also a nice change from traditional exercises based on dozens of repetitions. Patients are satisfied and enjoy exercising using the device."

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Opinion

The Luna EMG rehabilitation robot is an excellent tool for working with neurological and orthopedic patients with varying degrees of functional limitations. Diverse training programs allow for passive, assisted, active and active-resistance exercises. The unique feature of assisted movement triggered by the EMG signal from the patient's muscles allows active participation of the patient in exercises. With a real-time graphical representation of the activity being undertaken, patients are engaged in therapy, and seeing their results makes them more motivated for daily therapy.

We also use rehabilitation games implemented in the Luna EMG robot to add variety to the rehabilitation process among our patients. Task-based training with Luna is not only an engaging therapy, but also a nice change from traditional exercises based on dozens of repetitions. Patients are satisfied and enjoy exercising using the device.

In addition, we appreciate the ability to generate reports on our trainings in our therapeutic work. They are a clear supplement to the patient's record, allowing an objective assessment of the patient's progress in therapy.

We are pleased to work with the device in our center, it is an indispensable part of daily therapy sessions.

LINGUA LAB s.c.
W. Szyszkiewicz, M. Dąbłowska
ul. Sereno Fanni'a 6/14, 31-143 Kraków
NIP 677-235-31-15 REGON 12142650

Translated by www.lingualab.pl

Headquarters:

"Medico" Rehabilitation Center
Komorowicka 23
43-300 Bielsko-Biała

The person issuing the reference:

Małgorzata Winczowska - Owner of the
Rehabilitation Center "Medico"

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3.1.6.52. Voivodeship Hospital in Poznań

Poznań, 29.09.2021

“With its unique feature of supporting movement based on the patient's own muscle activity (Reactive Electromyography T&R, T&H), Luna EMG can be used in muscle re-education according to modern knowledge in neurophysiology.”

Piotr Nowicki

Voivodeship Hospital in Poznań,
Department of Neurological Rehabilitation of Branch No. 3
ul. Uzdrowskowa 2, Poznań - Kiekrz

EgzoTech Sp. z o.o.
ul. Romualda Traugutta 6H
44-100 Gliwice

Opinion

In our Department, the Luna EMG rehabilitation robot is used for patients with neurological deficits, such as: muscle strength weakness, coordination disorders, sensory disorders, muscle control disorders.

Luna EMG allows to perform both upper limb and lower limb exercises. Trunk activation is also possible. Thanks to the variety of training programs, it is used in patients with different fitness levels (weak, unable to perform any movement, with reduced muscle tension, as well as those with increased tension, requiring assistance in performing the movement). With its unique feature of supporting movement based on the patient's own muscle activity (Reactive Electromyography T&R, T&H), Luna EMG can be used in muscle re-education according to modern knowledge in neurophysiology.

All the training sessions completed with the device are recorded and can be compared. This allows us to assess the patient's functional improvement over time. The detailed reports generated after each exercise are a great addition to the patient's therapy records.

In summary, Luna EMG is a device that is an effective and engaging adjunct to therapy for patients with neurological deficits. We are pleased to be able to use the robot in our Department.

LINGUA LAB S.C.
W. Szyzkielwicz, M. Dambinowska
ul. Sereno Fanni'a 8/14, 31-143 Kraków
NIP 677-235-31-16 REGON 121428650

Translated by www.lingualab.pl

Headquarters:

Voivodeship Hospital in Poznań
Department of Neurological
Rehabilitation of Branch No. 3
Uzdrowskowa 2
Poznań - Kiekrz

The person issuing the reference:

Piotr Nowicki - Director of the Voivodeship Hospital in Poznań

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wn

3.1.6.53. Białystok Oncology Center

“Luna EMG robot is used primarily for exercises in pain syndromes, mobility restrictions in the shoulder joint and lymphedema within the entire upper limb. As a result of our rehabilitation, we achieve functional status improvement in our patients.”

Dr Joanna Filipowska

Białystok, 10.12.2021

Department of Rehabilitation
Białystok Oncology Center
Ogrodowa 12 Street
15-027 Białystok

Information on the use of equipment

The Department of Rehabilitation of the Białystok Oncology Center has been using Luna EMG and Stella BIO oncological rehabilitation sets for comprehensive rehabilitation of oncological patients for a year. We use them in female and male patients during or after cancer treatment. The largest group includes patients with

- malignant tumors of the breast,
- cancers of the reproductive organs,
- urinary tract cancers,
- melanomas,
- brain tumors,
- and many more.

When dealing with our patients, the most commonly encountered functional disorders manifest as tissue and joint mobility limitations, muscle strength weakness, muscle imbalance, lymphatic distribution disorders and lymphedema (upper limbs, lower limbs, head and neck tissues), as well as pain syndromes.

Luna EMG - The rehabilitation and diagnostic robot is used primarily for

- exercises with the CPM upper limb splint for pain syndromes, mobility restrictions in the shoulder joint and lymphedema within the entire upper limb,
- CPM rotational splint exercises and games for lymphedema in the forearm and arm,
- CPM steering wheel exercises and games for edema in the trunk and upper limbs,
- exercises with CPM ankle joint tip and games for edema in the ankle joint and calf, as well as limited mobility in this joint.

As a result of our rehabilitation, we achieve functional status improvement in our patients.

Best regards

dr n. med. Joanna Filipowska
Head of Department of Rehabilitation
Białystok Oncology Center
Ul. Ogrodowa 12 15-027 Białystok
e-mail: jfilipowska@onkologia.bialystok.pl phone 85 6646813

Translated by www.lingualab.pl

LINGUA LAB s.c.
W. Szyszkiewicz, M. Dembińska
ul. Piastowska 8A/33, 30-211 Kraków
NIP 677-235-31-15 REGON 121429850

Headquarters:

Białystok Oncology Center
Department of Rehabilitation
Ogrodowa 12
15-027 Białystok

The person issuing the reference:

dr n. med. Joanna Filipowska - Head of Department of Rehabilitation

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e-mail: jfilipowska@onkologia.bialystok.pl tel. 85 6646813

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Dr n. med. Joanna Filipowska

3.1.6.54. University Clinical Center in Gdańsk

“Working in virtual reality allows for greater patient involvement in the therapeutic process, and robot-assisted work using the EMG signal is possible even in a patient with severe paresis.”

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The Luna EMG device is used in the Rehabilitation Clinic of the GUMed University Clinical Center. It is primarily used for patients after stroke and craniocerebral trauma, but also those with spinal cord or peripheral nerve injuries. Luna EMG constitutes a beneficial addition to the physiotherapy of patients after orthopedic or spinal surgeries. Therapy programs provide training for strength, endurance, motor control or proprioception. Working in virtual reality allows for greater patient involvement in the therapeutic process, and robot-assisted work using the EMG signal is possible even in a patient with severe paresis. Most of the tasks have different forms of difficulty level scaling, therefore you can successfully use the device both in working with a patient with severe disabilities and with an athlete. The treatment solutions and programs are well thought out and clearly developed in collaboration with physiotherapists with clinical experience. Intuitive menu and beautiful design are additional advantages of the device. Extremely good contact with the manufacturer - EgzoTech, which also provides very reliable training on equipment use, is also a great strength.

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3.1.6.55. The Murcki Hospital

“Selecting optimal operational parameters according to the needs and capabilities of the patient, as well as – importantly – graphically transmitted information on the progress of therapy and the effects of the patient's work, support well the work of physiotherapists for the benefit of rehabilitated patients ”
Lek. Jolanta Kulawik-Szwajca

The Luna EMG device is used in the Department of Neurological Rehabilitation of Szpital Murcki Sp. z o.o. (The Murcki Hospital) in Katowice.

It is used primarily in the physiotherapy of neurological patients after stroke, patients with spasticity and Guillain-Barre syndrome, as well as orthopedic patients – to improve the upper and lower limbs after shoulder, hip or knee arthroscopy.

The capabilities of this highly specialized robot allowing the use of various therapeutic programs, and thus selecting optimal operational parameters according to the needs and capabilities of the patient, as well as – importantly – graphically transmitted information on the progress of therapy and the effects of the patient's work, having a positive effect on his involvement and motivation, well support the work of physiotherapists for the benefit of rehabilitated patients of the Department.

Head of the Department of Neurological Rehabilitation
Lek. Jolanta Kulawik-Szwajca
Specialist neurologist
Specialist in medical rehabilitation

Headquarters:

The Murcki Hospital
Department of Neurological
Rehabilitation
Sokołowskiego 2
40-749 Katowice

Translated by www.lingualab.pl

LINGUA LAB s.c.
W. Szyszkiewicz, M. Dembińska
ul. Piastowska 8A/33, 30-211 Kraków
NIP 677-235-31-15 REGON 121429850

The person issuing the reference:

lek. Jolanta Kulawik-Szwajca - Head
of the Department of Neurological Rehabilitation

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3.1.6.56. Voivodeship Specialist Hospital in Legnica

“In the absence of movement and palpable contraction of the muscle, we can evaluate and observe its activity using diagnostic EMG, which constitutes a great advantage of the Luna system. Luna's mobility is its advantage – thanks to this we can work with it at different positions, using many positions and alignments.”

Department of Neurological Rehabilitation
Voivodeship Specialist Hospital in Legnica

The Luna EMG robot is used for diagnostic and therapeutic purposes in the Department of Neurological Rehabilitation of the Voivodeship Specialist Hospital in Legnica. It complements ongoing therapies in most cases. It is used to work with patients after CNS damage and peripheral nerve injuries.

In the absence of movement and palpable contraction of the muscle, we can evaluate and observe its activity using diagnostic EMG, which constitutes a great advantage of the Luna system. Our most commonly used program is EMG with biofeedback, which allows our patients to observe and control muscle tension when experiencing sensory disorders. We use it most often in positions that normalize muscle tension. Programmed games that allow patients to actively train and relax at the same time is an interesting element combining entertainment and therapy.

We qualify people to work with Luna, provided that their intellectual impairment does not limit the understanding of instructions and task performance, therefore its user group is somewhat narrowed down.

Luna's mobility is its advantage – thanks to this we can work with it at different positions, using many positions and alignments. The chair included in the set unfortunately does not provide a feasible position for many people with mobility limitations. When working with Luna, we use height-adjustable tables and beds, which makes work much more efficient and easier.

With real-time EMG imaging, we can selectively train muscles based on specific needs and therapy plans. A global view of the patient is important to us, therefore we like to use two circuits. Knowing that entire muscle groups are engaged when performing many functions and activities, we would love to use more circuits for training. This will certainly complement the therapy and provide the opportunity for global, functional physiotherapy.

We are open to using the included heads, although it is necessary for our patients to select the most appropriate programs for working with them. When facing CNS injuries (both flaccid and spastic paralysis), passive or strongly assisted movement requires great precision and care due to the instability of the peripheral joints or the characteristics of the spastic muscles. Motor exercises performed in a single plane with assistance, resistance, or change of direction, allows you to prepare individual muscles for further work in multi-plane movements.

We see the Luna EMG robot as an innovative device that complements our work in an interesting way and puts diagnostic possibilities at our fingertips. With the connection to the screen, it compensates for sensory disturbances by imaging the work in real time. We expect that both our work efficiency and patient satisfaction will increase with Luna.

Headquarters:

Voivodeship Specialist Hospital in
Legnica
Department of Neurological
Rehabilitation
Iwaszkiewicza 5
59-220 Legnica

Team at the Department of Neurological Rehabilitation
of the Voivodeship Specialist Hospital in Legnica

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3.1.6.57. SPZOZ Lubartów

“LUNA EMG allows you to learn patient-controlled muscle contractions through electromyography. It allows active motor activity to a degree individually adapted to the patient's abilities.”

PT Mariusz Romaniuk

Lubartów, 25.11.2021.

Romaniuk Mariusz
Physiotherapist
Neurology Department
with the Stroke Unit
with the Neurological Rehabilitation Unit
SPZOZ Lubartów
ul. Cicha 14
21-100 Lubartów

Opinion

The LUNA EMG device was used in the Neurological Rehabilitation Unit during the months of October-November 2021 complementing physiotherapy in patients with multiple sclerosis and those who suffered a stroke.

LUNA EMG allows you to learn patient-controlled muscle contractions through electromyography. It allows active motor activity to a degree individually adapted to the patient's abilities. It works well in obtaining an increase in active mobility of peripheral joints. It also provides the possibility of repetitive exercise programs, which can be updated and recorded, and thus enables rehabilitation progress analysis.

LUNA EMG allows you to actively train patients with neurological conditions. With the ability to use a variety of heads, the device can be used for concentric exercise, which involves activating the muscle and relaxing/maintaining tension.

By using this device, the physiotherapist is able to perform the following exercises with the patient: isometric, isotonic, isokinetic, proprioceptive and elastic resistance exercises. It also allows performing passive exercises.

The device generated a lot of interest among patients, especially among the elderly. It awakened a desire to overcome one's own weaknesses. At the end of the session, the patients felt a kind of dissatisfaction, but when they saw their clearly visualized results, they felt that their progress could be measured, which resulted in increased motivation to work harder.

Physiotherapist
Mariusz Romaniuk

Headquarters:

SPZOZ Lubartów
Neurology Department
with the Stroke Unit
with the Neurological Rehabilitation
Unit
Cicha 14
21-100 Lubartów

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3.1.6.58. Dr. K. Jonscher Hospital

“The programs that the device offers ensure effective work with patients who have upper and lower limb plegia (complete paralysis). Individual adjustment of the parameters to the patient's capabilities enables conscious cooperation of the patient during the therapy session.”

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The person issuing the reference:

prof. nadzw. dr hab. n. med. Elżbieta Miller - Head of the Department:
of Rehabilitation and the Department of Neurological Rehabilitation

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3.1.6.59. Nicolaus Copernicus Independent Voivodeship Hospital in Piotrkow Trybunalski

“The robot catches the eye with its modern and stylish design. Its operation, software and layout make it easy and fast to use the robot. Its thoughtful exercises that provide a wide range of patient activities are a great asset.”

Team at the Department of Neurological Rehabilitation

Piotrków Trybunalski 22.12.2021.

Nicolaus Copernicus Independent Voivodeship Hospital
in Piotrkow Trybunalski
ul. Rakowska 15
97-300 Piotrków Tryb.

Opinion

We would like to welcome the entire EgzoTech Team.

First of all, we would like to congratulate you on the idea of creating a device like Luna EMG. We have recently had the pleasure of using the robot in our daily work with stroke patients. We are happy to share our first insights, conclusions in terms of technical and functional aspects with you.

The robot catches the eye with its modern and stylish design. Its operation, software and layout make it easy and fast to use the robot. Its thoughtful exercises that provide a wide range of patient activities are a great asset. The designed tips and accessories, the ease of changing them and the ability to customize them to the patient makes Luna functional. When working with it, one may get the impression that the robot reflects the direct "therapist - patient" work.

A few things caught our attention in terms of possible improvement:

- increasing the adjustment of the robot's lifting and lowering range,
- greater adjustment of the tablet arm (some positions during exercise prevent the patient from using games),
- protruding loose power plug
- change of the fastening strap system in the ends (they unfasten, difficulty in adjustment)
- change to Luna's platform locking system
- no change in the order of selected exercises
- no possibility to go back to previous exercise during training
- no exercise list view during training

Our idea is to create an integrative lower limb tip in which we would have a connection between the hip and knee joint, providing a functional tri-flexion movement in different positions, with adjustable locking of the knee joint.

We are pleased to work with your Company. The service personnel, sales representative and training instructor met our expectations.

Thank you very much. We would like to wish you Merry Christmas and many happy customers in the New Year.

Team at the Department of Neurological Rehabilitation

Headquarters:

Nicolaus Copernicus Independent
Voivodeship Hospital
Department of Neurological
Rehabilitation
Rakowska 15
97-300 Piotrków Trybunalski

Translated by www.lingualab.pl

LINGUA LAB s.c.
W. Szyszkiewicz, M. Dembińska
ul. Piastowska 8A/33, 30-211 Kraków
NIP 677-235-31-15 REGON 121429850

The person issuing the reference:

Magdalena Mijas - Specialist in medical rehabilitation
Physiotherapists:
Jakub Banaszczyk
Krzysztof Celej

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3.1.6.60. Poviát Health Center in Opole Lubelskie

“The possibility of rehabilitation through games effectively diversifies the tedious rehabilitation process by increasing focus, concentration and involvement during exercises. Patients exercising with the help of Luna rated their exercises as more effective and resulting in satisfaction with the completed task.”

MSc Joanna Jaszczak

Poniatowa 15.11.2021

Powiatowe Centrum Zdrowia
Sp. z o.o (Poviát Health Center) in Opole Lubelskie
Rehabilitation Center

Opinion LUNA EMG Rehabilitation Robot

During the four-week period of using the Luna-type device, our physiotherapists appreciated working with it in terms of:

- possibility to monitor rehabilitation progress of patients with CNS and PNS injuries, in terms of examining developing paresis before and rechecking the muscle strength after the training period, providing additional patient record information
- the possibility of rehabilitation through games effectively diversifies the tedious rehabilitation process by increasing focus, concentration and involvement during exercises
- during the activity of stopping the movement (steering wheel) and crossing the center line, we noticed effective coordination and involvement of the right and left cerebral hemispheres
- thanks to EMG imaging, the patient exercises with a greater awareness of movement and this allows us to control the effect of movement or its absence
- LUNA is also useful for examining selected muscle groups to determine the muscle balance, e.g. the external and internal rotators of the shoulder joint. It is therefore very useful in orthopedics in the field of arthroscopy and endoprosthetics.
- Patients exercising with the help of Luna rated their exercises as more effective and resulting in satisfaction with the completed task, despite the lack of verbal support, compared to working individually with a physiotherapist.

In addition, our physiotherapists would like to point out some technical details of the device. We feel that the casters and locking mechanism are too hidden under the shelf, which makes them difficult to lock with your foot. In addition, we did not have a lower limb exercise chair which made it difficult and lengthy for us to adapt the device to the rehabilitation couch. When setting the parameters and range of motion after the exercise has started, it is not possible to change the range of motion parameters by stopping, without leaving the program. Thank you for the opportunity to work with this highly specialized device.

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3.1.6.61. "Vita" Medical Center

"The LUNA EMG perfectly complements the work of physiotherapist at every stage of improvement. Working with the device makes it easy to work with each patient individually, thanks to different types of programs and specifications. LUNA EMG provides a useful device for improving patients with a variety of motor function disorders."

MSc Grzegorz Piechota
MSc Wioletta Mazurek

Urszulin, 15.12.2021

"Vita" Medical Center - REHABILITATION

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Opinion

We tested the LUNA EMG device in the rehabilitation center. The device has been used for people who suffered strokes, had brain tumor resections, and orthopedic injuries (mostly knee injuries.). The LUNA EMG perfectly complements the work of physiotherapist at every stage of improvement, both in the acute phase and in the chronic phase. Assisted by the use of electromyography and proper positioning of the arms, the LUNA design, in various starting positions, allows the patient to effectively learn controlled contractions of a muscle that previously had profound paresis. Working with the device makes it easy to work with each patient individually, thanks to different types of programs and specifications. The LUNA system allows for daily monitoring of the patient's progress in rehabilitation. Patients who worked with LUNA achieved much better and faster results in noticeable way. LUNA EMG provides a useful device for improving patients with a variety of motor function disorders, regardless of age, cause or general condition.

Head of the physiotherapy department
mgr Grzegorz Piechota

Deputy head of the physiotherapy department
mgr Wioletta Mazurek

Headquarters:

"Vita" Medical Center
Physiotherapy Department
Modrzewiowa 1/1
22-234 Urszulin

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The person issuing the reference:

MSc Grzegorz Piechota - Head of the physiotherapy department
MSc Wioletta Mazurek - Deputy head
of the physiotherapy department

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3.1.6.62. Powiat Hospital in Iława

“Luna allows for a variety of training such as: passive exercises, active exercises with resistance, active exercises with elastic resistance, biofeedback method, and games which are a combination of fun and training for the patient.”

Lek. med. Stefan Szpruch

Iława, 29.12.2021

OPINION

The LUNA EMG device is used at the Department of Neurological Rehabilitation in Iława, mainly with patients after ischemic and hemorrhagic stroke. Patients with spinal cord dysfunction, MS, MPD, Parkinson's disease and after orthopedic surgeries also work with the equipment.

Luna EMG allows us to work in the department, regardless of the degree of paresis. Patients who suffered a stroke recently can actively participate in the assessment of muscle activity using EMG. Training records help you compare the effects of your therapy using upper and lower limb tips.

Luna allows for a variety of training such as: passive exercises, active exercises with resistance, active exercises with elastic resistance, biofeedback method, and games which are a combination of fun and training for the patient.

At the first demonstration the equipment seems complicated, but everything about the operation of the equipment is much clearer after detailed training.

Luna EMG is a mobile, state-of-the-art rehabilitation equipment that enhances the patient's functional abilities.

Headquarters:

Powiat Hospital in Iława
Department of Neurological
Rehabilitation
gen. Wt. Andersa 3
14-200 Iława

Translated by www.lingualab.pl

LINGUA LAB S.C.
W. Szyberk-Wawrzyniak, M. Domalik
ul. Seneca 10, 01-643 Warszawa
NIP 677-245-31-15 REGON 121425850

The person issuing the reference:

Lek. med. Stefan Szpruch - Head of the Department of Rehabilitation

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3.1.6.63. University Hospital No. 2 in Bydgoszcz

“Reactive electromyography is a definite advantage, which allows diagnostics and assessment of the patient's functional status. The complexity of the functions and the large number of dedicated programs allow customized therapies for patients from neurological, neurosurgical or orthopedic departments.”

dr hab. n. med.
Mariusz Baumgart

Bydgoszcz, 9.12.2021

Opinion

The Luna EMG device is a useful addition to the neurological rehabilitation process for patients in both acute and chronic phase of CNS injury. Reactive electromyography is a definite advantage, which allows diagnostics and assessment of the patient's functional status. The complexity of the functions and the large number of dedicated programs allow customized therapies for patients from neurological, neurosurgical or orthopedic departments. Possibilities offered by LUNA EMG perfectly fit in with current trends in therapy preceded by examination of the patient and enable objectivization of the progress of therapy. The Luna device has been used not only for therapy, but also for teaching and scientific purposes of the Department of Neurosurgery and Neurology at the Dr. Jan Biziel University Hospital No. 2 in Bydgoszcz

Headquarters:

University Hospital No. 2 in
Bydgoszcz
Department of Neurosurgery and
Neurology
Ujejskiego 75
85-168 Bydgoszcz

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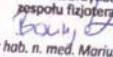
3.1.6.64. Voivodeship Specialist Hospital in Częstochowa

“The wide range of programs available helps in matching exercises to the capabilities and needs of our patients. The app's clear interface and UX allows therapists to use the equipment in an intuitive way and helps patients understand the various exercises, as well as adding motivation to their work (beating their previous results).”

MSc Kamil Alijew

Częstochowa 28.02.2022

Opinion

Department of Neurological Rehabilitation of the Blessed Virgin Mary Voivodeship Specialist Hospital in Częstochowa has and uses the Luna EMG machine for therapy and diagnostics. In our department, we use the device mainly when working with patients after stroke, cranio-cerebral injuries, and complications after surgeries. We primarily use CPM, CPM+, reactive EMG, proprioception training and elastic resistance training to rehabilitate neurological patients. Luna EMG is also useful for orthopedic patients. For patients with these types of conditions, we use all of the programs available on the device. Training to mimic elastic resistance and lifting weights help increase muscle strength. Proprioception training programs improve coordination of movement, and the CPM program setting helps maintain and increase range of motion of individual joints.

When working with patients with plegia, the greatest therapeutic effects are achieved using Trigger and Release, and Trigger and Hold programs, using 2 EMG channels in different configurations (antagonistic/agonistic muscles). Thanks to EMG electrodes, we can accurately check the occurrence of muscle contraction, unnoticeable to the "naked eye" or not felt by palpation.

Games are a great addition with therapeutic use. In our day-to-day work, we use EMG and orthopedic games to supplement therapy. They allow patients to relax a bit while training proprioception and motor coordination. Patients were particularly fond of "Brick Pirates" and "Bubble Game".

At the Department of Neurological Rehabilitation in the Blessed Virgin Mary Voivodeship Specialist Hospital in Częstochowa, we use the Luna EMG robot every day. The wide range of programs available helps in matching exercises to the capabilities and needs of our patients. The app's clear interface and UX allows therapists to use the equipment in an intuitive way and helps patients understand the various exercises, as well as adding motivation to their work (beating their previous results). Insights from post-training reports allow us to assess the progress of therapy and the rehabilitation process in detail. We are very pleased to work on such an advanced device.

Headquarters:

Voivodeship Specialist Hospital in
Częstochowa
Neurological Rehabilitation
Department
Bialska 104/118
42-200 Częstochowa

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The person issuing the reference:

MSc Kamil Alijew - Physiotherapist
Lek. med. Dagmara Soluch-Egeman - Head of the Neurological Rehabilitation Department
MSc Renata Pluta - Department Nurse

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3.1.6.65. Jelenia Góra Valley Voivodeship Hospital Center

“The LUNA EMG is a very useful device for working when patients with all types of conditions. It provides the possibility to use various exercises selected appropriately to the patient, allows the selection of treatment parameters and undoubtedly makes the work more attractive for both the patient and the staff working with Luna EMG.”

MSc Katarzyna Czubala

JELENIA GÓRA VALLEY VOIVODESHIP HOSPITAL CENTER

We have and have been using the LUNA EMG device in the Department of Physiotherapy, Ambulatory Rehabilitation Unit of the at the Jelenia Góra Valley Voivodeship Hospital Center from December 2021. Our patients are mostly people with orthopedic conditions, neurological conditions and degenerative joint changes. We use all of the programs that LUNA EMG offers, but this is often determined by the patient's psycho-physical condition. Reactive EMG using electrodes is not used in patients who cannot coordinate basic activities and have difficulty following simple commands. In case of these patients, we use programs that do not require as much concentration and commitment on their part. Nevertheless, the aforementioned program is effectively used for orthopedic patients. LUNA gives us the ability to assess the extent of damage and the progress of improvement. Isometric EMG and orthopedic games are very popular to make the improvement process more interesting.

The device is very modern, but the use of working tips does not quite work. In some positions, the patient's focus is more on the stability of holding the limb in the extremity than on the exercise itself (e.g., shoulder in supine position). Besides, the tip for the lower extremity (exercising the hip joint in the supine position) can only be used when working with a physiotherapist (supporting the lower leg).

The LUNA EMG is a very useful device for working when patients with all types of conditions. It provides the possibility to use various exercises selected appropriately to the patient, allows the selection of treatment parameters and undoubtedly makes the work more attractive for both the patient and the staff working with Luna EMG.

Translated by www.lingualab.pl

Headquarters:

Jelenia Góra Valley Voivodeship
Hospital Center
Physiotherapy Department
Ogińskiego 6
58-506 Jelenia Góra

The person issuing the reference:

MSc Katarzyna Czubala - Head of Physiotherapy Department

LINGUA LAB S.C.
W. Szyszkiewicz, M. Dembińska
ul. Szrena Fenn'a 6/14, 31-143 Kraków
NIP 677-235-31-15 REGON 121426851

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tel. sekretariat: (075)-75-37-201, tel. centrala: (075)-75-37-100

fax: (075)-75-42-335

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3.1.6.66. Local Government Hospital in Kutno

“Exercises on this device stimulate the patient to be more involved in the therapy and to perform them independently, which makes the therapy more attractive. The mobility of this device increases the accessibility to almost every patient, relieves the physiotherapists, improves the comfort of work and at the same time gives better results.”

Lek. Katarzyna Medyńska

Kutno 10.01.2022

Rehabilitation Department with Neurological
Rehabilitation Unit
of the Local Government Hospital in Kutno
ul. T. Kościuszki 52
99-300 Kutno

Opinion

We received the LUNA EMG in late November 2021. After familiarizing with the operation of the device, we started treating patients with it. LUNA is an instrument adapted for patients with stroke or peripheral nerve damage. Physiotherapists can choose from many options, from diagnosing the patient to monitoring their improvement process. Electromyography, biofeedback, and a variety of therapeutic programs may be used. It is also used by trauma and orthopedic surgery patients to increase the range of motion in damaged joints. Exercises on this device stimulate the patient to be more involved in the therapy and to perform them independently, which makes the therapy more attractive. The mobility of this device increases the accessibility to almost every patient, relieves the physiotherapists, improves the comfort of work and at the same time gives better results.

Yours sincerely

Headquarters:

Local Government Hospital in Kutno
Rehabilitation Department with the
Neurological Rehabilitation Unit
Kościuszki 52
99-300 Kutno

Translated by www.lingualab.pl

LINGUALAB S.C.
W. Szyszkiewicz, M. Dambin
ul. Sereńo Fenn'a 6/14, 31-143 Kraków
NIP 877-215-31-15 REGON 12142999

The person issuing the reference:

Lek. Katarzyna Medyńska - Head of Rehabilitation Department with the Neurological Rehabilitation Unit

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3.1.6.67. Powiat Hospital in Mikołów

Mikołów, 22.12.2021

“The device presents an unquestionable advantage of the ability to use the EMG signal in combination with a robotic arm that relieves the limb and enables passive or assisted movement, which provides effective therapy for patients with muscle strength defined as 1° on the Lovett scale.”

MSc Anna Brzezina

Luna, the neuro-rehabilitation robot, has been used in the Department of Neurological Rehabilitation at the Centrum Zdrowia w Mikołowie sp. z o.o. since October 2021.

Due to the profile of the Department, the patients covered by the neurophysiological therapy with substitute feedback using the Luna EMG robot mainly include people with damage to the upper motor neuron (stroke, craniocerebral trauma), neurodegenerative diseases (Parkinson's disease, multiple sclerosis) and damage to the peripheral nervous system (traumatic and autoimmune). Patients in the Department are in the sub-acute or chronic stages of illness.

The device presents an unquestionable advantage of the ability to use the EMG signal in combination with a robotic arm that relieves the limb and enables passive or assisted movement, which provides effective therapy for patients with muscle strength defined as 1° on the Lovett scale. Using the EMG signal to produce substitute feedback, combined with games, ensures that focus on the task at hand is maintained for longer, thus increasing the therapeutic potential of the robot. The device's software allows for the creation of an individual account for each patient, which, combined with the available diagnostic programs, makes it possible to evaluate the applied therapy on a quantitative scale, thus optimizing the rehabilitation process.

The Luna EMG robot provides task variability, promotion of the affected limb, implicit feedback, increasing difficulty level, task specificity of the exercises and direct feedback to satisfy generally accepted principles of neurorehabilitation.

We can see the potential of using the system not only as a complement to traditional therapy, but also as a completely new concept of working with patients after damage to the upper motor neuron.

Translated by www.lingualab.pl

LINGUA LAB s.c.
W. Szyszkiewicz, M. Dembińska
ul. Serego Farni 6/14, 31-143 Kraków
NIP 877-235-31-15 REGON 121429850

Headquarters:

Powiat Hospital in Mikołów
Department of Neurological
Rehabilitation
Waryńskiego 2
43-190 Mikołów

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3.1.6.68. Voivodeship Hospital in Łomża

Łomża 18.03.2022

“Patients are very eager to work with Luna EMG, because of its latest technology, the attractiveness and painlessness of the exercises, and the availability of biofeedback that provides feedback to both patient and therapist.”

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Rehabilitation Division of the Voivodeship Hospital in Łomża

Opinion

We have had the Luna EMG rehabilitation robot since January 2022.

After training, getting familiar with the operation and capabilities of the device, we started working with it.

We use Luna EMG in the Rehabilitation Department and the Neurological Rehabilitation Department mainly for neurological, oncological and orthopedic patients. So far, we have used CPM programs, reactive EMG, and therapy games.

The variety of available programs and the multitasking capability of the device allow you to adjust trainings to physical and functional abilities of the patient.

Patients are very eager to work with Luna EMG, because of its latest technology, the attractiveness and painlessness of the exercises, and the availability of biofeedback that provides feedback to both patient and therapist. Physiotherapists also praise working with the device because of the ability to control progress, adjust training loads, the variety of programs and therapeutic games.

Due to the fact that we had Luna EMG for a relatively short time, we had no opportunity to use all of its features.

Headquarters:

Voivodeship Hospital in Łomża
Rehabilitation Division
Piłsudskiego 11
18-403 Łomża

Translated by www.lingualab.pl

LINGUA LAB
W. Szyszkewicz, M. Dan
ul. Sereno Faenza 6/14, 31-14
NIP 677-235-31-15 REGON 121-235-31-15

The person issuing the reference:

Lek. med. Anna Katarzyna Matys - Coordinator of Rehabilitation Department
MSc Agnieszka Głodkowska - Head of the Therapists Team

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3.1.6.69. Voivodeship Combined Hospital in Konin

Konin, 15.03.2022

"Luna EMG is a modern device for neurorehabilitation, providing our department with the opportunity to improve patients in accordance with the assumptions of modern neurorehabilitation for the 21st century."

Lek. Joanna Pieńkowska

Opinion

The Rehabilitation Department and Neurological Rehabilitation Unit of the Voivodeship Combined Hospital in Konin have been using the Luna EMG device for several months.

There are patients with neurological, orthopedic and rheumatological diseases in our department, and we use the the neuro-robot's capabilities when working with them. The largest pool of patients working with Luna include those who had strokes and central neurological trauma. This device supports the work of physiotherapists through Luna's beneficial effects on brain plasticity and the formation of new connections between neurons. Our patients enjoy exercising with Luna. They emphasize its modern nature, variety of therapeutic and training possibilities.

The neurorehabilitation robot Luna provides great help for our rehabilitators. The device is also an excellent diagnostician. Thanks to electrodes placed on the limbs, we obtain an analysis of muscle tone and strength. Luna can pick up even the slightest muscle contraction and support its work, and the patient, focusing on a given movement, actively participates in the rehabilitation process.

Patients often refer to Luna as a modern-day "neurorehabilitator." The device picks up signals from the muscles and helps with movement. Even the smallest contraction initiated by the patient is registered and based on it the limb movement is performed. Thanks to this possibility Luna takes part in passive, active and passive exercises up to resistance exercises supporting the patient's work, but not replacing it. Thanks to the exercises performed with the Luna, we also conduct deep sensory training and improve patients' motor coordination.

The Luna neurorehabilitation robot is a diagnostic device. We diagnose muscles in our department: their activity and generation of any contraction, even such a delicate one that cannot be noted by physiotherapists, and therefore we have the opportunity to quickly and accurately select exercises for the patient.

Trainings with the patient and diagnostic data are recorded on the Luna device, giving us the ability to track and measurably assess the patient's rehabilitation progress and functional recovery.

Luna EMG is a modern device for neurorehabilitation, providing our department with the opportunity to improve patients in accordance with the assumptions of modern neurorehabilitation for the 21st century.

Translated by www.lingualab.pl

Headquarters:

Voivodeship Combined Hospital in
Konin
Szpitalna
62-504 Konin

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The person issuing the reference:

Lek. Joanna Pieńkowska - Head of Rehabilitation Department with the Neurological Rehabilitation Unit

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3.1.6.70. Centre of Physiotherapy in Zgorzelec

Zgorzelec, 13.06.2022

Centre of Physiotherapy in Zgorzelec
ul. Konarskiego 5B
59-900 Zgorzelec
75 77 558 941

“The diagnostic capabilities of the device are a great convenience. The Luna EMG device generates reports from training sessions, making the monitoring of the rehabilitation progress objective.”

Beata Michalak, MD, PhD

We have been using the modern Luna EMG rehabilitation robot at the Centre of Physiotherapy in Zgorzelec for several months. The device is applied for diagnostic and therapeutic purposes, mainly in neurological patients. They include people after stroke, with multiple sclerosis, or cerebral palsy. In these patients, we successfully use the CPM programs, proprioception training, reactive electromyography and neurological games.

We also use the Luna EMG device in orthopaedic patients, especially the programs to increase the range of motion and muscle strength, as well as to improve motor coordination.

The Luna EMG robot is a very good complement to the work of our physiotherapists. The variety of programs allows for precise and individual training which is tailored to the patient's needs on each day of therapy.

Thanks to the biofeedback system, the device gives feedback to the patient and the physiotherapist, which motivates them to continue working.

The diagnostic capabilities of the device are a great convenience. The Luna EMG device generates reports from training sessions, making the monitoring of the rehabilitation progress objective.

Luna EMG is an attractive and modern device that is eagerly used by both patients and physiotherapists. It is a comprehensive device for passive, active and assisted exercises. We can work with patients using games, which engages them even more in therapy and helps them regain lost functions. An additional advantage of the device is painlessness of exercises which motivates them to push their limits even further.

Our physiotherapists value the Luna EMG device for its diagnostic objectivity, the possibility of controlling the progress of rehabilitation and individual selection of therapy tailored to the patient's needs.

Headquarters:

Multidisciplinary Hospital -
Independent Public
Healthcare Team in Zgorzelec
Konarskiego 5B
59-900 Zgorzelec

The person issuing the reference:

Rehabilitation Department
Beata Michalak, MD, PhD

LINGUA LAB s.c.
W. Szyszkiewicz, M. Dembińska
ul. Sereno Fenn'a 6/14, 31-143 Kraków
NIP 677-236-31-16 REGON 121429850

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3.1.6.71. Centre of Physiotherapy in Świętochłowice.

“The device allows for the rehabilitation of patients regardless of the degree of paresis. Patients after stroke and orthopaedic surgery are actively involved in the assessment of the activity of the muscles using EMG.”

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Healthcare Centre in Świętochłowice Sp. z o.o.
County Hospital
Department of Neurological Rehabilitation
41-605 Świętochłowice, ul. Chorzowska 36

Świętochłowice, 21.06.2022

Opinion

The LUNA EMG device has been used at the Department of Neurological and General Rehabilitation in Świętochłowice in patients after ischemic and haemorrhagic stroke and in those after orthopaedic surgeries. The device allows for the rehabilitation of patients regardless of the degree of paresis. Patients after stroke and orthopaedic surgery are actively involved in the assessment of the activity of the muscles using EMG. Training records are kept to evaluate the effects of the therapy. In the Luna EMG, we use end-parts for the upper and lower limb. Luna allows for a variety of training tailored to the patient's needs, i.e. passive exercises, active exercises with resistance, active exercises with elastic resistance, the biofeedback method and games that perfectly combine fun with training. The operation of the equipment is clear and transparent. The Luna EMG device is modern rehabilitation equipment successfully used in the work with patients in our Department.

Headquarters:

Healthcare Centre in Świętochłowice
Sp. z o.o.
County Hospital
ul. Chorzowska 36

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3.1.6.72. The Cardinal Stefan Wyszyński Provincial Specialist Hospital

“In neurological patients, the LUNA EMG device gives numerous opportunities to work with paretic limbs, to strengthen weakened muscles, and to improve deep sensation. We also use games that make the therapy session more attractive.”

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Lublin, 27.05.2022

OPINION

Since December 2021, we have been using the LUNA EMG device at the Department of Rehabilitation with the Subunit of Neurological Rehabilitation of the Provincial Specialist Hospital in Lublin.

Our department treats patients with diverse diseases. These include neurological patients - after stroke, with multiple sclerosis, Guillain-Barré syndrome and other neurological diseases, orthopedic patients - after fractures, hip and knee arthroplasty, etc., as well as oncological patients.

The LUNA EMG device is widely used in each of the above-mentioned groups of patients. It makes the process of diagnosing very quick and pleasant (using diagnostic EMG to check with high accuracy the range of motion in the joint and deep sensation). Various endings attached to the device allow for the work with almost any part of the body.

In neurological patients, the LUNA EMG device gives numerous opportunities to work with paretic limbs, to strengthen weakened muscles, and to improve deep sensation. We also use games that make the therapy session more attractive.

The LUNA EMG device also gives very good results in patients with limitations due to pain. For instance, we can use the progressive CPM function to quickly increase the painless range of motion in the joint, for example, in patients with frozen shoulder.

We have received very positive feedback from our patients who observe the benefits and pleasure of working with this equipment. It is also a relief for our physiotherapist who, after connecting the patient to the device and setting the rehabilitation program, can work with another person or just relax ...

Translated by www.lingualab.pl

Headquarters:

The Cardinal Stefan Wyszyński
Provincial Specialist Hospital

The person issuing the reference:

Anna Mazur-Maj - Head of the Department of Rehabilitation

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3.1.6.73. Non-Public Health Care Centre Sanus by Jacek Madej

“The device is perfect for the treatment of the limited mobility of peripheral joints. The possibility of accurate monitoring and evaluating the effectiveness of therapy is also the advantage of the device.”

MSc Elżbieta Kasperska

Lublin, 14.04.2022

Elżbieta Kasperska
Head of the Department of Rehabilitation
Non-Public Health Care Centre Sanus Jacek Madej
ul. Magnoliowa 2
20-143 Lublin

20-143 Lublin, ul. Magnoliowa 2
NIP 946-125-41-05, REGON 060102640

Opinion

Our physiotherapists had the opportunity to test the LUNA EMG device. It is a very modern robot which, thanks to the use of electromyography, offers great diagnostic and therapeutic opportunities, especially in orthopedic and neurological patients. The device is perfect for the treatment of the limited mobility of peripheral joints. The possibility of accurate monitoring and evaluating the effectiveness of therapy is also the advantage of the device. Thanks to the various games installed in the equipment, it is perfect for the work with children.

The vast majority of our patients suffer from chronic back pain. Therefore, we could not use the LUNA EMG device on a large group of patients.

With kind regards,

Translated by www.lingualab.pl

Headquarters:

Non-Public Health Care Centre
Sanus Jacek Madej
ul. Magnoliowa 2
20-143 Lublin

The person issuing the reference:

MSc Elżbieta Kasperska - Head of the Department of Rehabilitation

NZOZ Sanus Jacek Madej
20-143 Lublin, ul. Magnoliowa 2
NIP 946-125-41-05, REGON 060102640

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3.1.6.74. Provincial Specialist Hospital named L. Rydygier

Krakow, 24.05.2022

“By using the LUNA EMG device, we significantly increase the efficiency of our work. The appeal of the robot and the variety of its programs make the process of rehabilitation attractive for patients”.

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From December 2021, we have been using a modern rehabilitation robot, LUNA EMG, at the Department of Neurological Rehabilitation with the Sub-unit of General Rehabilitation.

Most of our patients are in the subacute phase after stroke. It is at this time that the body regenerates the most and exercises bring the best results.

The LUNA EMG device can be used for passive, active and assisted exercises. In this way, we help patients overcome their limitations at all stages of rehabilitation. As a result, patients have a chance to regain the functions they have lost. We follow the principle of neuroplasticity, which is crucial in neurological rehabilitation.

Thanks to the use of the superficial EMG, the robot gives the opportunity to objectively and repeatedly assess the activity of the muscles, as well as their reactions to various commands.

The training with the biofeedback system is usually used in patients with flaccid paresis and the impaired muscle tone regulation. It is a long and demanding process. The LUNA device allows the patient to observe how the activity of the muscles is influenced by their will to contract or release muscle tone, and this often becomes a turning point in regaining control over the activity of the muscles.

Some of our patients suffer from orthopaedic diseases, e.g. healing after fractures, but there are also people after endoprosthesis surgery. In these cases, we generally use the CPM programs, as well as progressive passive movement. This is to help increase the range of motion in the joints. At a later stage of rehabilitation, we work on the muscle strength using the programs based on resistance exercises, such as lifting "weights" and elastic resistance exercises. The program of isokinetic exercises allows for adjusting the training to the capabilities of the skeletal muscular system and the level of pain and fatigue. Working in a painless way makes patients use the device willingly.

Thanks to the multifunctionality of the device, we can treat patients with various pathologies. By using the LUNA EMG device, we significantly increase the efficiency of our work. The appeal of the robot and the variety of its programs make the process of rehabilitation attractive for patients. In addition, all data and reports are collected in the system, thanks to which we can present the patients their progress, which greatly improves their motivation to work.

Translated by www.lingualab.pl

Headquarters:

Provincial Specialist Hospital named L. Rydygier
31-826 Krakau,
Złotej Jesieni 1

The person issuing the reference:

MD Jadwiga Sułowska - Head of the Rehabilitation Team

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3.1.6.75. Institute of Psychiatry and Neurology in Warsaw

Warsaw, 17 August 2018

*Institute of Psychiatry and Neurology in Warsaw
ul. Sobieskiego 9, 02-957 Warsaw
2nd Neurology Clinic
Associate Professor Maciej Krawczyk, MD, Ph.D.*

Reference

The LUNA EMG device was used in the 2nd Neurology Clinic of the Institute of Psychiatry and Neurology in the period from 17 July to 17 August 2018 as part of physical therapy and occupational therapy in patients with a history of stroke and spinal cord disorders. The device was analysed in terms of the requirements for physical therapy in the above-mentioned patients under scientific, technical and practical supervision.

The Luna EMG device allows for exercises of both upper and lower limbs as well as activation of the torso. It is characterised by high flexibility and the possibility of practical use when dealing with neurological patients having various degrees of dysfunction. It enables rehabilitation in various positions – from lying in bed to standing. One of the greatest benefits is the use of electromyography during robot-assisted exercise. This allows for the active rehabilitation of a patient and induction of any muscle contraction, even if significant paresis is concerned. The LUNA EMG device employs the principles of movement learning similar to biofeedback, repeated active repetition of a particular movement, focusing on the goal, grading difficulty and exercise on the edge of capability. Both patients with decreased and increased muscle tone can be included in the rehabilitation. Thanks to the integrated programmes and substitute biological feedback, patients become highly motivated and willing to train. Daily kinesiotherapy with the use of the LUNA EMG device lead to the improvement in all patients. In conclusion, as results from the above statements, in our opinion the LUNA EMG device can be a very useful element in the process of medical rehabilitation and affect the motor improvement in neurological patients.

Translated by www.fingualab.pl

“One of the greatest benefits is the use of electromyography during robot-assisted exercise. This allows for the active rehabilitation of a patient and induction of any muscle contraction, even if significant paresis is concerned.”

Associate Professor
Maciej Krawczyk, MD, Ph.D.

Headquarters:

Institute of Psychiatry and Neurology in Warsaw
2nd Neurology Clinic
Sobieskiego 9, 02-957 Warsaw

The person issuing the reference:

Associate Professor Maciej Krawczyk, MD, Ph.D. - President of the National Council of Physiotherapists (2016 - 2022), physiotherapist, physiotherapy specialist

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3.1.6.76. Central Clinical Hospital of the Ministry of Interior and Administration in Warsaw

“In addition, in difficult cases such as patients with a history of stroke and traumatic brain injuries, rehabilitation games offered by the Luna device, are invaluable support in the rehabilitation of neurocognitive disorders.”

Justyna Zielińska-Turek, MD, Ph.D.

Central Clinical Hospital
of the Ministry of Interior and Administration
in Warsaw

Warsaw, 6 July 2022

Clinical opinion

The use of the Luna EMG device at the Department of Neurological Rehabilitation of the Central Clinical Hospital of the Ministry of Interior and Administration in Warsaw, has resulted in high flexibility and practicality of physiotherapists' work with patients. A great advantage are static and dynamic ends for upper and lower extremities, used for passive, active and resistance exercises. The Luna device allows for the rehabilitation in various starting positions, helps a patient complete the movement, so that even patients with advanced muscle atrophy can actively exercise. Thanks to the biofeedback option, the patient has to control the work of the muscles, and consciously regulate their contraction and relaxation.

The device enables the possibility of transdiagnostic therapy of a patient, measuring range of motion, muscle strength and providing electromyographic measurements results, while increasing the patient's muscle strength, range of joint motion as well as improving motor coordination and range of motion. By analysing the condition of a patient, the Luna robot provides the opportunity to compare trainings at the beginning and end of rehabilitation.

In addition, in difficult cases such as patients with a history of stroke and traumatic brain injuries, rehabilitation games offered by the Luna device, are invaluable support in the rehabilitation of neurocognitive disorders.

In our opinion, the Luna EMG robot, is an innovative device that is an important element of physical therapy, measurably affecting the speed and quality of improving the condition of patients.

Headquarters:

Central Clinical Hospital of the
Ministry of Interior and
Administration in Warsaw
Wołoska 137,
Warsaw 02-50

Translated by www.lingualab.pl

The person issuing the reference:

Justyna Zielińska-Turek, MD, Ph.D. - Deputy Chef of the Neurology Clinic

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3.1.6.77. John Paul II Subcarpathian Provincial Hospital in Krosno

“Physical therapists appreciate the use of the above-mentioned equipment, because it provides them the opportunity to control the therapy and decrease their workload. ”

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John Paul II SUBCARPATHIAN PROVINCIAL HOSPITAL in Krosno
38-400 KROSNO, ul. Korczyńska 57

Krosno, 29 June 2022

Summary of work on the LUNA EMG device

I would like to inform that the LUNA EMG device was installed at the Rehabilitation Unit with the Neurological Rehabilitation Sub-unit in December 2021. As a result of the COVID-19 pandemic and the reorganisation of the work at the Unit related to the transformation into the Covid Unit, the use of the above-mentioned device started in April 2022.

Launching the LUNA EMG robot was preceded by training and familiarising with a wide range of tasks and functions offered by the device. After the possibilities of the application of the Luna device were mastered, the staff of the Unit included regular training on the above-mentioned equipment in the process of rehabilitation of patients hospitalised in the Unit. Patients who undergo rehabilitation at the Unit originate also from other hospital units including: neurology, orthopaedics, surgery and rheumatology.

Luna allows for a precise selection of parameters for a patient as well as for tracking the progress in therapy. Painless exercises increase the range of motion, improve proprioceptive sensation and motor coordination at a quick pace. In rehabilitation, a growing number of new opportunities to make the therapy more attractive is used, e.g. orthopaedic games and biofeedback. Because of the fact that the biofeedback system provides both the therapist and the patient with feedback, the progress in rehabilitation or the possible possibility of its optimisation are visible. This information mobilises patients and motivates them to further effort and therapeutic development, therefore it is observed that the LUNA EMG robot is a very attractive device for patients and staff. Physical therapists appreciate the use of the above-mentioned equipment, because it provides them the opportunity to control the therapy and decrease their workload. While getting familiar with the above-mentioned robot, the staff has not so far managed to fully apply all the functions offered by the device.

The staff of the Rehabilitation Unit with the Neurological Rehabilitation Sub-unit appreciates the possibility of further development, and at the same time is open to further training to improve the comfort of their work and take full advantage of the multitasking offered by the LUNA EMG device.

Translated by www.lingualab.pl

Headquarters:

John Paul II Subcarpathian Provincial
Hospital
Korczyńska 57
38-400 Krosno

The person issuing the reference:

Janusz Boczar, MD - Head of the Rehabilitation
Unit with the Neurological Rehabilitation Sub-unit

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3.1.6.78. Iwonicz Spa Resort

“The programmes available as part of the LUNA EMG device that increase the muscle strength and the range of motion in the joints work very good for patients e.g. with a history of knee arthroplasty and a history of orthopaedic procedures as well as injuries of the upper and lower limbs.”

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Iwonicz Spa Resort S.A.
Al. Torosiewiczza 2
38-440 Iwonicz-Zdrój
Subcarpathian Voivodeship

Opinion concerning work using the LUNA EMG rehabilitation and diagnostic robot

The LUNA EMG device has been used in the “Excelsior” Spa and Rehabilitation Hospital of the Iwonicz Spa Resort S.A. since April 2022.

The “Excelsior” Spa and Rehabilitation Hospital accommodates mainly patients with a history of stroke and suffering from multiple sclerosis and Parkinson’s disease. There are also patients with back pain as well as orthopaedic and rheumatology ones who benefit from a curative stay.

The LUNA EMG device is perfect for working with neurological and orthopaedic patients. Patients are eager to exercise with the LUNA EMG robot.

The programmes available as part of the LUNA EMG device that increase the muscle strength and the range of motion in the joints work very good for patients e.g. with a history of knee arthroplasty and a history of orthopaedic procedures as well as injuries of the upper and lower limbs.

On the other hand, neurological patients, appreciate rehabilitation with the robot in particular for the possibility of checking the results of their work on an ongoing basis, which is possible thanks to the biofeedback system available in the device. Thanks to the CMP passive exercise programme, training of the weakest patients is possible.

The LUNA EMG is a device willingly applied by our physical therapists and patients themselves.

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3.1.6.79. L. Zamenhof University Children's Clinical Hospital in Białystok

L. Zamenhof

UNIVERSITY CHILDREN'S CLINICAL HOSPITAL in Białystok

Director: Prof. Anna Wasilewska, MD, Ph.D.

Białystok, 8 June 2022

Reference – summary of work using the Luna EMG device

We have been using the Luna device at the Children's Rehabilitation Clinic of the University Children's Clinical Hospital since February 2022. After the necessary training and becoming familiar with working options, we included training with the use of the device in the rehabilitation process of our patients. Considering the age of the patients of the Rehabilitation Clinic, the programme is used by older children. People training on the Luna device include mainly neurological and orthopaedic patients. The robot complements the therapy, enables accurate diagnostic measurements and their monitoring during and after the training cycle. The possibility of saving parameters and therapeutic progress is a great convenience. It significantly facilitates completing the medical record documentation kept by physical therapist.

In orthopaedic patients, we use all the programmes available in the device to increase muscle strength as well as improve joint mobility, coordination and the efficiency of the body. Orthopaedic games are very popular among young patients.

In neurological patients, Luna is perfect for Trigger Hold/ Release REACTIVE EEM using 2 EEM channels for antagonist or agonist muscles. We also use passive as well as dynamic continuous manoeuvrability programmes with monitoring of muscle action potential (EMG).

The Luna EMG device constitutes a perfect complement to the therapy with the possibility of adjusting the training to the needs of a patient. Biofeedback provides feedback to the therapist and the patient, which to a great extent motivates people to further efforts and their continuation at home.

Translated by www.lingualab.pl

Headquarters:

L. Zamenhof University Children's
Clinical Hospital
Jerzego Waszyngtona, 15-274 Białystok

The person issuing the reference:

Ewa Renata Rakowicz, M.Sc. - Head of the team of physical therapists

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“Thanks to advanced technology, it enables: diagnosis of innervation with the use of surface electromyography, assessment of the range of motion in isolated joints and planes, assessment of maximum muscle strength, assessment of susceptibility to fatigue.”

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Reference

The NeuroRehab Centre is a specialist rehabilitation centre, offering a wide range of state-of-the-art equipment. In our work, we rely on individually selected rehabilitation tailored to the current needs of the patient and complement it by using specialised equipment.

The LUNA EMG device provided by EgzoTech fits into the base of our equipment. The Luna EMG robot is a robotic rehabilitation device combining passive movement, active movement, resistance movement, EMG Biofeedback and robotic support training.

Thanks to advanced technology, it enables:

- Diagnosis of innervation with the use of surface electromyography,
- Assessment of the range of motion in isolated joints and planes,
- Assessment of rigidity and spasticity (muscular disorders) in the limbs,
- Assessment of maximum muscle strength,
- Assessment of susceptibility to fatigue,
- Wide application in neurological and orthopaedic patients as well as in the treatment of lower back pain.

In addition, it plays a significant role in training that:

- Increase muscle strength,
- Increase the range of limb motion,
- Increase coordination.

Yours faithfully
Dorota Pala, M.Sc.

Translated by www.lingualab.pl

“In technical terms, the device offers enormous possibility of various forms of supporting training with patients, at the same time complementing the work of a physical therapist. A significant benefit is an option to monitor and document the course of the training and the daily progress of a patient.”

Anna Zaremba-Ner, M.Sc.

Headquarters:

Mazovian Hospital in Garwolin
Legionów 11,
08-400 Garwolin

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3.1.6.82. SPSK No.4 in Lublin

Lublin, 25 August 2022

Clinical Rehabilitation Unit
SPSK No. 4 in Lublin

Reference
for the Luna EMG device

“The Luna EMG robot is not only a therapeutic device, but also a diagnostic one. The therapy of a patient is recorded at each stage of rehabilitation, which enables an objective assessment and analysis of its progress.”

Magdalena Sobiech, MD, Ph.D.

The Luna EMG device has been owned by our rehabilitation unit since June this year.

The Luna EMG robot has enabled us to expand the scope of physiotherapy treatments with new forms of rehabilitation of neurological, geriatric and orthopaedic patients. It provides the opportunity to exercise the whole body, not only the lower and upper extremities, using the biofeedback system and EMG. EMG employs the active effort of patients to the extent that is optimally suited to their capabilities. On the other hand, biofeedback supports the work on concentration and association while performing tasks. It complements the actions of a therapist both in the acute and chronic phase of a disease.

The Luna EMG robot is not only a therapeutic device, but also a diagnostic one. The therapy of a patient is recorded at each stage of rehabilitation, which enables an objective assessment and analysis of its progress.

We express a positive opinion concerning the device and the offered therapeutic options.

Magdalena Sobiech, MD, Ph.D.
Coordinator of the team of physical therapists
of the Clinical Rehabilitation Unit

Translated by www.lingualab.pl

Headquarters:

SPSK No.4
Dr. K. Jaczewskiego 8,
20-954 Lublin

The person issuing the reference:

Magdalena Sobiech, MD, Ph.D.
- Coordinator of the team of physical therapists
of the Clinical Rehabilitation Unit

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3.1.6.83. Polish Association for Persons with Intellectual Disability

“Thanks to rehabilitation games, the involvement and motivation of our patients to perform exercises increases. With the help of the robot, we obtain better neuromuscular priming and an improvement in proprioception. The Luna EMG enables active work with patients from “1” on the Lovett’s scale.”

Polish Association for Persons with Intellectual Disability
BRANCH IN SKARSZEWY

Polish Association
for Persons with Intellectual Disability
Branch in Skarszewy
ul. Kościerska 11A
83-250 Skarszewy

REFERENCE

The LUNA EMG robot – a device used in the Polish Association for Persons with Intellectual Disability, Branch in Skarszewy from 8 July 2022 till 9 August 2022. Physical therapists working in our Association successfully complemented the physical therapy of patients with the possibilities provided by the LUNA EMG device. Physical therapy covered people with various disease entities. These were mainly adults with ataxic tetraparesis, with a history of spinal cord injury, hemiparesis as well as upper extremities and lower extremities paresis. People with a history of stroke were also included.

The Luna EMG robot allows, among others, for robotic passive training, weight training, training with dynamic resistance: isokinetic, isotonic, flexible exercises, with rehabilitation games and in selected configurations – with reactive electromyography. Thanks to rehabilitation games, the involvement and motivation of our patients to perform exercises increases. With the help of the robot, we obtain better neuromuscular priming and an improvement in proprioception. The Luna EMG enables active work with patients from “1” on the Lovett’s scale.

Physical therapists reported an issue and suggested that the improvement of tablet angle tilt regulation in order to increase the visibility and brightness of the screen would significantly improve the comfort of working with the patient.

Translated by www.lingualab.pl

Headquarters:

Branch in Skarszewy
Kościerska 11A
83-250 Skarszewy

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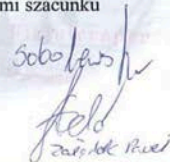
3.1.6.84. SAWIMED Centers for Medical Care and Rehabilitation

Sawice Wieś, 29.09.2022

“Functionality, ease of adjustment to the patient, simple and intuitive operation are the advantages offered by LUNA. Being able to plan various forms of activity – from passive activity, isometric contractions, contractions with resistance to virtual games – is also important.”

MSc Kamil Smyl

OPINION

The LUNA EMG device used at the Rehabilitation Center has been an excellent addition to physiotherapy for patients staying at our facility. The device has been used mainly with neurological patients with past strokes, as well as those with peripheral paresis. It is also a great aid for orthopedic patients after knee and hip endoprotheses. Working with the Luna EMG allows controlled movement of the patient in their current range of motion, and it enforces the patient's muscular activity by connecting the EMG. The device allows maximum patient involvement in the rehabilitation process and provides control over the actual work done by the patient.

Functionality, ease of adjustment to the patient, simple and intuitive operation are the advantages offered by LUNA. Being able to plan various forms of activity – from passive activity, isometric contractions, contractions with resistance to virtual games – is also important. LUNA EMG is a great support in physiotherapist work.

Luna EMG provides an objective and realistic assessment of therapy effects in accordance with EBM.

Translated by Lingua Lab
Nowogrodzka 50 lok. 515
00-695 Warszawa

Headquarters:

SAWIMED Centers for Medical Care and Rehabilitation
Sierakowska 8
00-709 Warszawa

The person issuing the reference:

MSc Kamil Smyl - Head of the Rehabilitation Department

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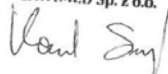
3.1.6.85. SP ZOZ Łapy

Łapy, 26.09.2022

“Physiotherapists at our facility have significantly improved patient rehabilitation process using the Luna EMG device, particularly in case of patients with peripheral nerve palsy, and after knee and shoulder joint surgeries.”

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Physiotherapists at our facility have significantly improved patient rehabilitation process using the Luna EMG device, particularly in case of patients with peripheral nerve palsy, and after knee and shoulder joint surgeries.

Exercises using electromyography have produced a visible effect in patients with sagittal nerve and facial nerve palsy. In orthopedic rehabilitation, the Luna EMG device works well for increasing the range of motion in a joint, and electromyography works well for rebuilding muscle strength and mass. Luna EMG also allows tracking the patient's progress, which helps in planning further rehabilitation. The system for operating the device is intuitive, and after just a few days of use, one can take full advantage of the benefits of Luna EMG, and modify exercises accordingly.

Translated by Lingua Lab
Nowogrodzka 50 lok. 515
00-695 Warszawa

Headquarters:

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The person issuing the reference:

Agnieszka Kulesza - Head of medical rehabilitation

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“We use Luna as an additional tool in the improvement process, improving "eye-hand" coordination through orthopedic games and proprioception training - programs that have gained the approval of patients qualified for this.”

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Opinion - Summary of Work with LUNA EMG

For the past 10 months, we have been using the LUNA EMG device for diagnostic and therapeutic purposes in the Rehabilitation Department with Neurological Rehabilitation Subunit of the Silesian Hospital in Cieszyn.

Our patients include mainly people after stroke, craniocerebral injuries, those with MS, Parkinson's disease, spinal cord injuries, polyneuropathies, those undergoing neurological diagnosis, as well as people after spinal surgery, hip and knee endoprosthesis insertion, cerebral vascular embolization and many others.

We use Luna as an additional tool in the improvement process, improving "eye-hand" coordination through orthopedic games and proprioception training - programs that have gained the approval of patients qualified for this. We also use Luna as a diagnostic tool to measure muscle strength at various stages of rehabilitation.

Our problem is the conditions of the premises - LUNA found a place in a small gym hall, with many patients working at the same time, therefore there is a problem with attention and focus, along with the lack of ability to use tips that require a larger area to work. Because of that, some of the capabilities of the device have not been applied at the moment, but we plan to increase the scope of device use and monitor the progress of improvement after changing the premises.

Translated by Lingua Lab
Nowogrodzka 50 lok. 515
00-695 Warszawa

Headquarters:

Silesian Hospital in Cieszyn
Bielska 4
43-400 Cieszyn

The person issuing the reference:

Dr Małgorzata Kowala-Koziół - Head physician of the rehabilitation ward with sub-ward for neurological rehabilitation
MSc Lechosław Chmielniak - Physiotherapy specialist, coordinator of physiotherapists

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NIP: 548-20-01-181
REGON: 000313348
KRS: 0000143545

3.1.6.87. University of Rzeszow

“The LUNA EMG device offers diagnostic tests to assess muscle tension, proprioception, ranges of motion, and muscle strength, among others.”

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Opinion on working with the LUNA EMG rehabilitation and diagnostic robot

The Department of Physiotherapy of the Institute of Health Sciences of the College of Medical Sciences at the University of Rzeszow conducts research using the LUNA EMG rehabilitation and diagnostic robot.

The LUNA EMG device offers diagnostic tests to assess muscle tension, proprioception, ranges of motion, and muscle strength, among others. Research conducted at the University of Rzeszow focuses on using Luna EMG in healthy people of different age groups, as well as people with neurological disorders.

In addition, as part of their coursework, Physiotherapy students conduct examinations, as well as arrange rehabilitation programs using the latest technologies offered by Luna EMG to increase muscle strength and joint range of motion. Furthermore, they conduct exercises using the visual and auditory biofeedback method found in Luna EMG's built-in games.

LUNA EMG is a device that is eagerly used in scientific research, as well as in teaching.

Translated by Lingua Lab
Nowogrodzka 50 lok. 515
00-695 Warszawa

Headquarters:

University of Rzeszow
Rejtana 16C
35-959 Rzeszów

The person issuing the reference:

Professor Mariusz Druźbicki - Head of the Department of Physiotherapy at the University of Rzeszow

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3.1.6.88. Manual-Med

Lublin, 23.11.2022

“It is a very innovative and modern device that completely changes the standards for working with mostly trauma and neurological patients.”

MA Mateusz Daktera

Manual-Med. sp. z o.o.
Al. Warszawska 54
20-803 Lublin

FEEDBACK ON THE LUNA EGZOTECH DEVICE

Luna EGZOTECH has been used in our facility since 26.10.2022. It is a very innovative and modern device that completely changes the standards for working with mostly trauma and neurological patients. Luna EGZOTECH is designed as an intuitive and easy to use device, while opening up a very wide range of therapeutic possibilities. Furthermore, it makes the therapy and the patient's arduous work in the rehabilitation process more attractive and thus easier and more enjoyable for the therapist and the patient.

The very form of the device's skeleton, the mechanism for connecting and disconnecting the handpieces, setting up and adjusting to the patient's needs demonstrates an extremely high standard. Given the increased number of strokes, paralysis, traffic accidents in recent years, we can confidently say that Luna can be the future of any rehabilitation facility.

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3.1.6.89. Independent Public Clinical Hospital named after prof. Adam Gruca CMKP

Otwock, 28.11.2022

Opinion on Luna EMG device

“In conclusion, we consider the Luna EMG device to be very useful in the rehabilitation process of orthopedic, neurological patients. For an orthopedic patient, however, a different range of muscle force sensors and a greater range of resistance generated by the device are required. Working with it supplements therapy with effective proprioception exercises and improves the effectiveness of therapy by activating the muscles involved in the correct movement pattern.”

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We have been working with the Luna EMG device at the Rehabilitation Clinic of Adam Gruca CMKP SPSK in Otwock since September 2022. We use the device to work with patients in the Social Insurance Institution disability prevention department and the rehabilitation department. The department's patients mostly include those who have undergone orthopedic surgeries performed at the A. Gruca Hospital, such as primary and revision endoprosthesis insertion of the hip and knee joints, arthroscopic procedures in the joints of the upper as well as lower extremity, victims of multi-organ trauma as a result of traffic accidents, and neurological patients.

We have found uses for Luna EMG equipment among patients with postoperative complications related to neurological deficits associated with peripheral nerve damage. Another group of patients in case of whom the LUNA robot can find great use include patients who have developed abnormal motor patterns and compensation mechanisms as a result of their disease. EMG-assisted re-education of the correct movement path has significantly improved the effectiveness of rehabilitation, such as patients after ligament reconstruction.

Among the patients in our department, the resistance offered by resistance training possible with the Luna device has proved insufficient for the lower extremity muscles. We work with orthopedic patients in the early period after surgical treatment who have greater muscle strength than neurological patients, and the device is extremely necessary for such patients, due to the activity correction for the muscles involved in the movement pattern. In addition, the stabilization of the limb for this type of exercise is too weak and patients have reported straps loosening, in addition to buttons at the shoulder end pads becoming worn out.

The Luna EMG device's software seems unintuitive. It is not possible to change the handpiece in any position you choose, such as one in which the patient has finished the exercise. It can only be mounted in the vertical position.

Furthermore, the proper positioning of the device is not convenient for the physiotherapist because of the 4 separate brakes - a better solution is to lock all wheels, which can be activated and deactivated with the foot. The device requires precise adjustment of the knee rotation axis for each patient.

In conclusion, we consider the Luna EMG device to be very useful in the rehabilitation process of orthopedic, neurological patients. For an orthopedic patient, however, a different range of muscle force sensors and a greater range of resistance generated by the device are required. Working with it supplements therapy with effective proprioception exercises and improves the effectiveness of therapy by activating the muscles involved in the correct movement pattern.

Translated by Lingua Lab
Nowogrodzka 50 lok. 515
00-695 Warszawa

Headquarters:

Independent Public Clinical Hospital named after prof. Adam Gruca CMKP
Stanisława Konarskiego 13
05-400 Otwock

The person issuing the reference:

Professor Małgorzata Łukowicz - Head of the Rehabilitation Department

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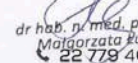
NIP: 532-16-62-948

REGON: 000290156

3.1.6.90. Mazovian specialized hospital in Ostrołęka

Doctor Aneta Wekwejt
Head of Department of Rehabilitation
MSS in Ostrołęka

Ostrołęka 19.12.2022

“Working with Luna has confirmed that it is a very helpful and professional device for upper limb, lower limb or trunk stabilization exercises. The therapeutic effect on the patient can be achieved faster by working with Luna. ”

Doctor Aneta Wekwejt

Opinion

The Luna EMG device has been available in the resources of our Rehabilitation Department since June 2022. Despite such a short time of working with Luna, our team (consisting of physiotherapists and doctors) has noticed that the equipment is perfect for the therapy of both neurological patients, e.g. after strokes, craniocerebral or spinal injuries, with multiple sclerosis, or after spinal surgeries with loss symptoms, and orthopedic patients, e.g. after prosthetics especially of the knee joint or arthroscopic surgeries of joints, e.g. knee, shoulder, and after limb injuries.

Working with Luna has confirmed that it is a very helpful and professional device for upper limb, lower limb or trunk stabilization exercises. The therapeutic effect on the patient can be achieved faster by working with Luna. As a diagnostic robot, Luna EMG enables evaluating the range of motion, assessing muscle activity or muscle strength, and thus allows a real assessment of physiotherapy effectiveness. Working with limb paresis can be more effective thanks to using electromyography. It is also worth noting that Luna provides biofeedback training capabilities.

The Luna EMG is an innovative yet uncomplicated device that complements physiotherapist work.

With best regards,

Aneta Wekwejt

Translated by Lingua Lab
Nowogrodzka 50 lok. 515
00-695 Warszawa

Headquarters:

Mazovian specialized hospital in Ostrołęka
Jana Pawła II 120A
07-410 Ostrołęka

The person issuing the reference:

Doctor Aneta Wekwejt - Head of the Rehabilitation Department

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3.1.6.91. Regional hospital in Toruń

“The biofeedback system further enhances the beneficial effects of motor therapy. The various programs available on the device make it possible to tailor training to patients at different stages of improvement. Training monitoring allows you to observe the rehabilitation progress, and simple mechanical solutions make it easy to operate the device.”

Doctor Jadwiga Nowak
MSc Dorota Zastawna

The LUNA EMG device is used at the Voivodeship Rehabilitation Outpatient Clinic of the Regional Hospital in Toruń for outpatients to supplement physiotherapy. We started rehabilitating our patients with it about three months ago.

To date, beneficial effects of the therapy have been observed in both orthopedic patients (after fractures, injuries, surgeries) and neurological patients, such as those with Guillain-Barre syndrome. Programs: CPM, elastic resistance, weightlifting, contribute to improving the range of motion in the joints, increasing the muscle strength of the extremities, and proprioception exercise helps provide better joint control, resulting in improved gait function. The biofeedback system further enhances the beneficial effects of motor therapy. The various programs available on the device make it possible to tailor training to patients at different stages of improvement. Training monitoring allows you to observe the rehabilitation progress, and simple mechanical solutions make it easy to operate the device. The only drawback of the device is the complicated wheel lock.

We will begin training with electrodes and an EMG system in the near future.

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3.1.6.92. WAM-CSW Clinical University Hospital in Łódź

“To summarize, the LUNA EMG is one of the most innovative and practically applicable devices we have had the opportunity to work with at our Clinic in years. Observations to date allow us to conclude that in addition to patients highly praising working with Luna EMG, it is the therapists who also recognize its qualities as a complement to the basic rehabilitation program.”

Professor Robert Irzmański
Doctor Tomasz Nowakowski

Department of Internal Medicine, Rehabilitation and Physical Medicine

USK im. WAM - CSW in Łódź

Pl. Hallera 1,90-647, Łódź

Clinical Opinion

The LUNA EMG device has been used in the Department of Internal Medicine, Rehabilitation and Physical Medicine at the University Clinical Hospital of WAM in Łódź from May 2022. It operates as a part of the program for physiotherapy treatments and diagnostics, and is used in teaching activities with students. LUNA EMG is used to work with neurological patients (patients after stroke incidents, with MS, after spinal cord injuries, after neurosurgeries, after craniocerebral injuries, with Parkinson's disease and others), as well as with orthopedic patients (after fractures, after hip endoprosthesis insertion, after knee replacement and others). Our greatest experience with Luna EMG to date is based on working with patients who had strokes, and thus have impaired muscle strength.

The use of LUNA EMG makes it possible to work on both the lower and upper extremities. Due to its large number of both neurological and orthopedic programs, it is used in a wide range of applications (for patients with reduced as well as excessive muscle tension). It is characterized by great flexibility in working with patients. Selecting the right tip or starting position for the exercises is intuitive, which undoubtedly makes it easier to work with the patient. Adjustability within the device itself, as well as within the handpieces, allows training to be tailored to the patient with a specific improvement program.

Without a doubt, electromyography and biofeedback programs are the biggest advantage offered by LUNA EMG. Patients with muscle strength of 1 or even 0 on the Lovett scale can undergo rehabilitation thanks to the reactive EMG in programs, such as Trigger - Release or Trigger - Hold. A patient, performing the tasks instructed to them through muscular activation of the relevant muscle groups, not only improves their strength, but also has the impression of real influence on what the machine performs instead of them. This has a very positive effect on the motivation and improvement of the mental state of patients benefiting from these programs. The compensation control program in two-channel operation, which is sometimes crucial in neurological patients, in case of whom I very often observe the compensation phenomenon, is also extremely useful.

To summarize, the LUNA EMG is one of the most innovative and practically applicable devices we have had the opportunity to work with at our Clinic in years. Observations to date allow us to conclude that in addition to patients highly praising working with Luna EMG, it is the therapists who also recognize its qualities as a complement to the basic rehabilitation program.

Translated by Lingua Lab
Nowogrodzka 50 lok. 515
00-695 Warszawa

Headquarters:

WAM-CSW Clinical University
Hospital in Łódź
Hallera 1
90-647 Łódź

The person issuing the reference:

Professor Robert Irzmański - Head of the Department of Internal Medicine, Rehabilitation and Physical Medicine
Doctor Tomasz Nowakowski - Physiotherapy specialist

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3.1.6.93. Bonifraterskie Physiotherapy Center

Bonifraterskie Physiotherapy Center in Piaski
Physiotherapy Department

“ The LUNA EMG device has given us the opportunity not only for therapy, but importantly for muscle diagnostics. In case of neurological patients, LUNA EMG complemented the rehabilitation process brilliantly, using biofeedback and EMG..”

MSc Barbara Piotrowska

Piaski 23.01.2023.

OPINION FOR THE LUNA EMG DEVICE AT BONIFRATERSKIE PHYSIOTHERAPY CENTER IN PIASKI

The LUNA EMG device has given us the opportunity not only for therapy, but importantly for muscle diagnostics. In case of neurological patients, LUNA EMG complemented the rehabilitation process brilliantly, using biofeedback and EMG.

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We would like to thank EGZO TECH for the opportunity to work with this equipment. LUNA EMG device received my positive opinion.

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mgr Barbara Piotrowska

3.1.6.94. Pojda Rehabilitation and Physiotherapy

“ The EMG system, interface, appearance of Luna and training selection are clear and intuitive. Based on our short experience, we are most satisfied with motor facilitation in the peripheral joints”

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Pojda Rehabilitation and
Physiotherapy
A. Napieralskiego 33
43-267 Rudziczka

The person issuing the reference:

MSc Przemysław Pojda- Owner of the Pojda Rehabilitation and Physiotherapy

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3.1.6.95. Lower Silesian Specialist Hospital named after T. Marciniak

“ An important functionality of the device is the ability to quickly generate reports, which gives the patient and the physiotherapist information about the progress of therapy”

MSc Denis Moskiewicz

Opinion on the LUNA EMG device.

The LUNA EMG rehabilitation robot has been used in our Rehabilitation Department for over a year. Before starting work with patients, physiotherapists were trained in the use of the device. This allowed for proficient use of the robot. LUNA EMG is equipped with interchangeable extensions, each of which is correctly constructed in terms of ergonomics. The ability to adjust the parameters in each of them allows the robot to be used in various starting positions.

The tablet used to operate the device is intuitive and easy to use. This allows the robot to be used quickly.

A great advantage of the device is the use of electromyography, which allows the use of the robot with little muscle activity, and biofeedback gives the possibility of constant control of the performed movement task by the patient and the physiotherapist.

An important functionality of the device is the ability to quickly generate reports, which gives the patient and the physiotherapist information about the progress of therapy. Electromyography combined with a system of extensions, a variety of training programs and the ability to generate reports gives the possibility of good use of the robot for scientific research, both for measurements and as a method of working with the patient, which also takes place in our Rehabilitation Department.

To sum up, LUNA EMG is a very good diagnostic and therapeutic device, willingly used by patients due to its modernity, and for physiotherapists it is a perfect complement to therapy with the patient.

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Headquarters:

Lower Silesian Specialist Hospital
named after T. Marciniak - Center for
Emergency Medicine
ul. Fieldorfa 2
54-049 Wrocław

The person issuing the reference:

MSc Denis Moskiewicz- teaching assistant

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3.1.6.96. National Residential and Rehabilitation Center for people suffering from MS Dąbek

“Luna EMG supports kinesitherapy focused on neurological patients. The advantage of the device is the possibility of active work with people with muscle strength on the Lovett scale "1". A great solution is the possibility of diagnostics, in particular the measurement of the range of motion or muscle strength.”

Dariusz Sowiński

Our Center deals with the rehabilitation of people suffering from multiple sclerosis. For a month we had the opportunity to work on the Luna EMG device from EGZOTech.

The main goal of our work was to observe the increase in muscle strength, improvement in range of motion and coordination. After a month-long rehabilitation cycle, an improvement in the above-mentioned parameters could be observed.

Luna EMG allows you to perform a variety of exercises, including passive exercises, exercises based on dynamic variable resistance or EMG Biofeedback. Luna EMG supports kinesitherapy focused on neurological patients. The advantage of the device is the possibility of active work with people with muscle strength on the Lovett scale "1". A great solution is the possibility of diagnostics, in particular the measurement of the range of motion or muscle strength. Importantly, it allows you to objectively compare the initial trainings to the final trainings by saving the history of exercises. Rehabilitation games aroused great interest, thanks to which daily exercises became more attractive and interesting.

At the same time, we used the Stella-Bio electromyograph. Work on this device was limited due to problems with the Internet coverage. Despite the small number of therapies, we evaluate the equipment positively. Thanks to its extensive functions, Stella - Bio could work well in working with people with multiple sclerosis.

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Headquarters:

National Residential and
Rehabilitation Center for people
suffering from MS Dąbek
06-651 Stupsk

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Dariusz Sowiński

3.1.6.97. Voivodeship Hospital Complex named after Ludwik Perzyna in Kalisz

Kalisz, 30.08.2023

“The Luna EMG device is a valuable supplement primarily for patients with neurological diseases.”

MSc Aleksander Kubiacyk

The Luna EMG device is a valuable supplement primarily for patients with neurological diseases.

Therefore, most programs used during therapy are based on EMG.

In these programs, even minimal muscular activity of patients is used.

We use these programs as part of the re-education of neurological patients.

I would like to mention that understanding the essence of the operation of the Luna EMG device would not be possible without additional training by Mrs. Alicja Nogala.

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Headquarters:

Voivodeship Hospital Complex
named after Ludwik Perzyna
st. Poznańska 79
62-800 Kalisz

The person issuing the reference:

MSc Aleksander Kubiacyk - team leader of physiotherapists

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NIP 618 20 25893
REGON 300224440

3.1.6.98. Rehabilitation Sanatorium-Hospital Complex for the Musculoskeletal System 'Gwarek'

Opinion

“Luna EMG has proven itself as a versatile device, offering the possibility of working with patients of all ages with various motor dysfunctions and serving as a relief in the physiotherapist's work. It adds attractiveness and variety to the rehabilitation process.”

MSc Anna Bojdoł

The Luna EMG Robot has a broad range of applications in the rehabilitation of patients with neurological disorders and in orthopedic postoperative rehabilitation. It enables the performance of exercises for both upper and lower limbs in lying, sitting, and standing positions. Therapy conducted with the robot using EMG is tailored to the patient's current functional abilities. Luna is a compact, mobile device that allows for bedside rehabilitation. The device requires cognitive ability from the patient, significant engagement, precise tension of the relevant muscle group, and minimizing tension of antagonistic muscles.

Luna EMG is highly sensitive, detecting minimal muscle tension generated by the patient to initiate movement. It facilitates active movement of the paralyzed limb through the full range of motion. With the Luna EMG Robot, we can record each patient's training session, monitor rehabilitation progress, generate PDF reports, and print the collected documentation.

A challenge for neurologically impaired patients during exercises with biofeedback and games seems to be the small screen of the tablet. In such situations, a television screen can also be connected to Luna. One limitation of the robot is that it operates only in one plane, and changing the plane is time-consuming.

Luna EMG has proven itself as a versatile device, offering the possibility of working with patients of all ages with various motor dysfunctions and serving as a relief in the physiotherapist's work. It adds attractiveness and variety to the rehabilitation process.

Translated by
Clinical Specialist @ EGZO Tech
Alicja Nogala

Headquarters:

Rehabilitation Sanatorium-Hospital
Complex for the Musculoskeletal
System "Gwarek"
Uzdrowskowa Street 51
43-230 Goczałkowice Zdrój

The person issuing the reference:

MSc Anna Bojdoł - Head of the Rehabilitation/Spa Facility

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3.1.6.99. District Hospital in Wyrzysk

Opinion

“Certainly, LUNA EMG is a very good complement to the patient's therapy through its wide application.”

MSc Daria Goła

LUNA EMG is a device used in the Neurological Rehabilitation Department. It is used by patients after stroke, polyneuropathy, multiple sclerosis and spine surgery. Thanks to the possibility of connecting electromyography to specific muscle groups, the patient takes an active part in performing movements despite significant paresis. During therapy, muscle contraction is shown on the monitor, which encourages the patient to become more involved in the movement process. This work can be made easier or more difficult at any time. Thanks to the possibility of turning on the game, the patient's work on a given movement becomes easier more pleasant.

The device has many therapeutic programs, both with or without the use of EMG, if the patient is able to control muscle tension on his own. The tips that are attached to the device allow for the work of the upper limb, lower limb, precise hand movements and activation of the torso.

Certainly, LUNA EMG is a very good complement to the patient's therapy through its wide application.

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The person issuing the reference:

MSc Daria Goła - coordinating physiotherapist of the Neurological Rehabilitation Department

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3.1.6.100. Sosnowiec Municipal Hospital

Opinion

“The device allows for combining traditional rehabilitation methods with the highly advanced technology offered by Luna EMG to a very large extent. Therapies using the Luna EMG device are very effective and efficient because they take into account the individual needs of the patient.”

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Physiotherapists working in the Neurological Rehabilitation Department at the Sosnowiec City Hospital supplement patient rehabilitation using modern rehabilitation robots. As part of the funding from the Ministry of Health, Luna EMG with the Mezos SIT chair was purchased. The rehabilitation device is a very effective tool in the rehabilitation process of neurological patients in our department. The robot allows for the introduction of individual therapy, monitoring of progress, motivating patients. The device allows for combining traditional rehabilitation methods with the highly advanced technology offered by Luna EMG to a very large extent. Therapies using the Luna EMG device are very effective and efficient because they take into account the individual needs of the patient. We are very pleased with the purchased device and the professionalism and commitment of the manufacturer and representatives of EGZOTech.

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The person issuing the reference:

MD Edyta Łebek - Medical rehabilitation specialist, Head of the Neurological Rehabilitation Department

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3.2.1. China

3.2.1.1. Nanjing Pukou Central Hospital

“Even for patients with little muscle contraction can work actively and get better results quicker”

Yesi, Rehabilitation Department
Nanjing Pukou Central Hospital

Nanjing Pukou Central Hospital
No.166,Jiangpu Shanghe Street,Pukou District, Nanjing

Experience with Luna EMG

Nanjing Pukou District Central Hospital, founded in 1949, is a comprehensive hospital integrating prevention, medical treatment, teaching and scientific research. The hospital undertakes the task of being a medical, teaching and scientific research center in the district and provides medical services to nearly 400,000 people in neighboring Hean,Quanjiao and Laian County in Anhui Province. The hospital covers an area of 110,000 square meters with 957 beds.

Luna EMG is mainly used in neurological rehabilitation and orthopedic rehabilitation in our hospital. Luna is portable and flexible, which can be applied in the ward or placed in the PT room. For bed-bound patients, it is important to use Luna for muscle training at an early stage. In the traditional treatment, the bedridden patients with very weak muscle strength used to be treated with simple passive training and manipulation treatment. Luna fills the gap in active training. Luna also helps us treat patients in other ways:

- set up easily, save the time of the therapist
- visual interface and digital report, let patients kept in training with the confident
- Luna EMG is equipped with exchangeable extensions, which allow patients to perform the training of all major joints, cost-effective
- with passive, active and active assistive/assisted training mode, It is suitable for patients with muscle strength 0-5 , wide application scope

Luna is a relatively new device and technology in the field of rehabilitation in China. It brings more possibilities for muscle training .**Even for patients with little muscle contraction can work actively and get better results quicker.** Hopefully there will be more and more good equipment and technology which can help more patients.

Rehabilitation department,
Nanjing Pukou District Central Hospital
Yesi

Headquarters:

Nanjing Pukou Central Hospital
No. 166 Jiangpu Shanghe Street,
Pukou District, Nanjing

The person issuing the reference:

Rehabilitation Department Nanjing
Pukou District Central Hospital Yesi

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3.2.1.2. Shanghai Jinhui Rehabilitation Hospital

“Luna is a relatively new device in China that combines EMG and isokinetic training for severe patients. Luna EMG is used as a holistic device with passive, active and active assistive/assisted training especially for weak patients ”

Jiang Yanju

Shanghai Jinhui rehabilitation hospital
No. 3739, Wenchuan Road,
Baoshan District, Shanghai

Experience with Luna EMG

The rehabilitation department of Shanghai Jinhui Rehabilitation Hospital, which integrates medicine, teaching and research, has all kinds of rehabilitation facilities. The rehabilitation treatment area covers 3,100 square meters. It is a key specialty in Baoshan District and has a number of characteristic core technologies, all of which are leading in the district. There are nearly 50 rehabilitation doctors and therapists in rehabilitation team. It is a training base for postgraduates of Medical College of Wuhan University of Science and Technology, as well as a rehabilitation practice and teaching base for many universities. At the same time, he has undertaken several projects and won many awards. At present, the department of rehabilitation medicine of our hospital has established close relationship with 7 community health service centers.

Luna EMG has been introduced into our hospital for nearly a year, mainly for the hospitalized elderly people. Luna is a robot with many functions and is suitable for many patients in different stages with rehabilitation needs. Especially for patients with weak muscle strength, trigger and release, trigger and hold these two functions are kind of innovation. It is particularly important for the neural remodeling of patients that they can actively participate in movement. Luna is a good example use of this. After the use of Luna, we also found that Luna has the following characteristics:

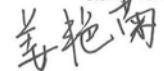
- equipment is cabinet, convenient to move to wards for patients with bed
- patient is able to work independently, the therapist can look after other patients at the same time
- the visual interface of the training and games let patients have more confidence and interests to participate in the training
- training has a clear and detailed assessment report, which you can access to at any time
- Patients in different rehabilitation period can use Luna, high cost performance

Luna is a relatively new device in China that combines EMG and isokinetic training for severe patients. Luna EMG is used as a holistic device with passive, active and active assistive/assisted training, especially for week patients. We are still studying more usage scenarios and functions of Luna EMG, hoping to benefit more patients.

Headquarters:

Shanghai Jinhui Rehabilitation
Hospital
No. 3739 Wenhuan Road,
Baoshan District, Shanghai

Rehabilitation department
Shanghai Jinhui rehabilitation hospital
Jiang Yanju

The person issuing the reference:

Rehabilitation Department
Shanghai Jinhui Rehabilitation Hospital
Jiang Yanju

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3.2.1.3. People's Hospital of Shiping County, Yunnan Province

“Before the introduction of Luna, many patients were easy to fall asleep during the passive training, and the training effect was not good.

With LUNA, we can get patients to participate an active exercise as early as possible, which is a very meaningful innovation.”

Peng Ying

Headquarters:

People's Hospital of Shiping County,
Yunnan Province
No. 3739 Wenhuan Road,
Baoshan District, Shanghai

The person issuing the reference:

People's Hospital of Shiping County, Yunnan Province
Peng Ying

People's Hospital of Shiping County, Yunnan Province
No. 3739, Wenchuan Road,
Baoshan District, Shanghai

Experience with Luna EMG

Shiping County People's Hospital was founded in 1951. After 65 years of construction, it has now become a comprehensive third-class B hospital integrating medicine, education, prevention and first aid. There are more than 300 hospital beds. At present, the rehabilitation department is the section that the hospital wants to develop vigorously, and its foundation was relatively weak in the past. Therefore, LUNA isokinetic assessment and training system was introduced in the hope of bringing better experience to patients with this advanced rehabilitation method.

Before the introduction of LUNA, many patients were easy to fall asleep during the passive training, and the training effect was not good. With LUNA, we can get patients to participate in active exercise as early as possible, which is a very meaningful innovation. At the same time, the therapist can observe the patient's muscle strength and contraction during the training process, and provide real-time and correct guidance to the patient.

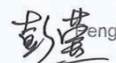
There are also many features in Luna's operation, such as:

- It is very easy to replace the joints extension, and the therapist can complete the replacement in a few seconds
- The double emergency brake button provides multiple protection for the safety of patients
- It is easy to move and can be used in wards and PT room
- With touch screen, patients can see their training situation intuitively

It is believed that after we are more and more familiar with Luna, we will be able to find more advantages of this device and deliver it to more suitable patients for treatment.

Rehabilitation department

People's Hospital of Shiping County,

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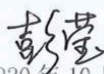
3.3. Malaysia

3.3.1. Perkeso Medical Rehabilitation Centre

“The assessment and training reports are useful to provide evidence on the patients progress and recovery”

Muhammad 'Irfan bin Azman

Opinion

Luna EMG is a versatile device that provides a wide range of clinical applications to our patients ranging from orthopedics to neurological conditions. The special feature of Luna EMG that uses the reactive EMG to train the clinically weak patients is beneficial to the stroke or spinal cord injury patients in our centre. The different extensions and positionings with various exercise modes allow the therapists to apply the device on different patients across different stages of recovery. In addition, the assessment and training reports are useful to provide evidence on the patients' progress and recovery.

Headquarters:

Pusat Rehabilitasi Perkeso Melaka,
Hang Tuah Jaya
75450, Bemban, Malacca
Malaysia

The person issuing the reference:

Muhammad 'Irfan bin Azman,
Coordinator Program of Rehabilitation Services

MUHAMMAD 'IRFAN BIN AZMAN
PEGAVALI PROGRAM PERAKSIAN REHABILITASI
Jabatan Rehabilitasi
Pusat Rehabilitasi PERKESO

3.3.1.1. Gleneagles Hospital Penang

“Physiotherapists are amazed with the fundamental elements from Luna EMG device, especially the electromyography and visual biofeedback for patients with muscle weakness and lack of motor control”

Lee Jih Yen

OPINION / FEEDBACK

Firstly, thank you for the time spent with us for the guidance throughout the training. We are pleased with the wide usage of Luna EMG device especially for the patients with neurological problems.

Physiotherapists are amazed with the fundamental elements from Luna EMG device, especially the electromyography and visual biofeedback for patients with muscle weakness and lack of motor control. Most of the patients also given positive feedback on the effectiveness after the individualized treatment session with Luna EMG.

Unfortunately we have yet to try it on most patients with musculoskeletal problems as we were having increased workload during that period as the setting up of the device was time-consuming for beginners.

In short, we in the Neuro Rehabilitation team are pleased with the wide usage function of the Luna EMG device.

LEE JIH YEN (L000427)
BSc (Hons) in Physiotherapy, MPPSA University
Physiotherapist
12 | 05 | 2023.

Headquarters:

Gleneagles Hospital Penang
1, Jalan Pangkor,
10050 George Town,
Pulau Pinang, Malaysia

The person issuing the reference:

Ms. Lee Jih Yen
Senior Physiotherapy

3.3.1.2. Asian Institute of Medicine, Science and Technology (AIMST University)

“The device combines several important elements that can be personalised to meet our patient’s needs. Our patients found that the built in games were interactive and challenging, thus enhancing their involvement during rehabilitation.”

Sonita Ashvni Raj

Feedback on Luna EMG

We are thankful to be given the opportunity to run a trail using the Luna EMG device. The device was loaned to our department for 2 weeks. The device was kept at the Stroke Center of our hospital where our acute and chronic cases were able to explore the benefits of the device. Our physiotherapists and occupational therapists were able to explore the wide use of the above mentioned device. The device combines several important elements that can be personalized to meet our patient’s needs. Our patients found that the built in games were interactive and challenging thus enhancing their involvement during rehabilitation. The reports generated at the end of the session were in detailed description and can be utilized for tracking patients’ progression. On behalf of Pantai Integrated Rehab, we are pleased to be given the opportunity to explore the device and would like to thank EGZO Tech for the training and guidance.

SONITAASHVNI RAJ
BSc (Hons) Physiotherapy,
AIMST University
Physiotherapist

Headquarters:

AIMST University
Batu 3 1/2, Bukit Air Nasi,
Jalan Bedong—Semeling
08100 Bedong, Kedah, Malaysia.

The person issuing the reference:

Sonita Ashvni Raj - Physiotherapist
Senior Physiotherapy

3.3.1.3. Hospital Universiti Malaysia Sabah

“One of the most remarkable features of Luna EMG in rehabilitation is their ability to provide targeted and customizable therapy. These machines can be programmed to adapt to each patient's unique needs, allowing for tailored exercises and treatments that focus on specific muscle groups and movement patterns. This level of customization not only accelerates the rehabilitation process but also minimizes the risk of overexertion or strain, ensuring a safer and more efficient recovery journey. ”

Norshafinie Binti Abu Samah

Physiotherapist Hospital Universiti Malaysia Sabah

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7th August 2023

I am writing to express my opinion on the Luna EMG particularly their transformative impact on the field of healthcare and patient recovery.

We had the opportunity to have Luna EMG device at the Pusat Rawatan Rehabilitasi Hospital Universiti Malaysia Sabah since 7th July till 27th July 2023. We use the device for neurological patient stroke mainly.

The integration of Luna EMG in rehabilitation represents a groundbreaking advancement that holds immense promise for improving the quality of life for individuals recovering from various medical conditions, injuries, and surgeries. These machines, designed with cutting-edge technology and engineering precision, have the potential to significantly enhance the rehabilitation process and contribute to better patient outcomes.

One of the most remarkable features of Luna EMG in rehabilitation is their ability to provide targeted and customizable therapy. These machines can be programmed to adapt to each patient's unique needs, allowing for tailored exercises and treatments that focus on specific muscle groups and movement patterns. This level of customization not only accelerates the rehabilitation process but also minimizes the risk of overexertion or strain, ensuring a safer and more efficient recovery journey.

Furthermore, it offers the advantage of consistent and repetitive movements, which are often essential for rebuilding motor skills and strengthening weakened muscles and deliver a controlled, consistent regimen, resulting in more predictable progress tracking and better-informed adjustments to the rehabilitation plan.

The integration of real-time feedback and EMG integrated is another notable aspect of Luna EMG. Patients can receive instant feedback on their performance, and progress, empowering them to actively engage in their own recovery. This not only fosters a sense of autonomy but also encourages a higher level of motivation and compliance with the prescribed rehabilitation regimen.

However, there's few things that need to be highlight while using the Luna EMG, its taking quite some time to set up the machine for example when installing the extension and setting up the patient position when there's only bed and chair was the only option we have that time.

I am also wondering if whether Luna EMG can be functioned to powered battery in case of any power cut at that time.

In conclusion, the incorporation of Luna EMG in rehabilitation represents a remarkable stride forward in the healthcare industry. The potential benefits it offers in terms of customization, consistency, and real-time feedback are undeniable. As we continue to witness the evolution of this technology, it is my hope that the collaboration between robotics and human expertise will usher in a new era of rehabilitation that maximizes recovery potential and enhances the overall well-being of patients.

Thank you for considering my perspective on this important matter. I look forward to further advancements in this field and the positive impact they will undoubtedly bring.

Sincerely,

Headquarters:

Faculty of Medicine and Health Science,
Jalan UMS, 88400 Kota Kinabalu, Sabah, Malaysia

The person issuing the reference:

Norshafinie Binti Abu Samah - Physiotherapist

3.4. Middle East & Africa Clinics

3.4.1. Saudi Arabia

3.4.1.1. Health Care Department In Mecca Health Department At-Taif King Faisal Medical Complex

“Of the many advantages of Luna EMG, we can mention that the patients has positive attitude towards the device; [...] the active work can be started even when the muscle activity is very low; [...]; Our close cooperation with Egzotech is also another point on which we are extremely satisfied.”

Sami Abdu Akuur

The Rehabilitation Center of King Faisal Medical Complex Taif, KSA has been applying the Luna robot for the therapy of patients for more than one year. To date, patients with neurological conditions (multiple sclerosis, strokes, and Guillain Baré syndrome), but also orthopedic patients at early stages, fully benefit from this innovative approach.

To start with, Luna EMG is an assessment tool particularly suitable for predicting the responsiveness of the rehabilitation. Then, the device provides an individualized therapy specifically tailored to the patient needs.

Of the many advantages of Luna EMG, we can mention:

- The patients positive attitude towards the device.
- A minimize setup time compared with other robotic systems.
- The active work can be started even when the muscle activity is very low.
- The ability for the patient to train autonomously. This specific feature is essential given that scientific research has demonstrated that the rehabilitation time is largely in short supply. By increasing the number of repetitions, Luna allows to bridge this gap while therapists do less physical labour.
- An increased cost-effectiveness (therapists can work with more patients at once).
- Data are recorded in the software and can be easily transferred in any formats.

Our close cooperation with Egzotech is also another point on which we are extremely satisfied. Because a steady evolution is what matters the most for us, we can assert that our hand-on experience is taken into great consideration.

Sami Abdu Akuur
Head Of Rehabilitation

Dr. Sami Abdu akuur
Head of Rehabilitation
2725911

Headquarters:

Health Care Department In Mecca
Health Department At-Taif King
Faisal Medical Complex.

The person issuing the reference:

Sami Abdu Akuur; Head of
Rehabilitation

4. Mezos SIT

4.1. European Clinical Facilities

4.1.1. Germany

4.1.1.1. Aviva Berlin

“The advantage the Mezos therapy seat offers lies in that patients can be secured in such a manner that thanks to ongoing movements they can concentrate on the treatment itself without fearing that it may be painful or the pain-free area may be exceeded by accident.”

Franziska Schlagwein

Assessment opinion on MEZOS SIT

Impressions of the combination of MEZOS SIT therapy couch with Luna EMG and Stella BIO

Franziska Schlagwein | aviva.berlin

The AVIVA Health Centre in Berlin Adlershof is a multidisciplinary centre that is focused on health, exercising and socialising in a welcoming and friendly atmosphere.

We offer specific therapeutic measures in the fields of ergotherapy, physiotherapy and logopedics, coupled with sports courses and medical training therapy with a rehabilitative and preventative aim.

The Luna EMG complete therapy system along with the Mezos SIT therapy couch/therapy seat and the Stella BIO therapy robot have been complementing the range of therapies we offer for some time now.

The Mezos SIT therapy couch, which is equipped with an electronic height adjustment system and may additionally be reclined, complements the therapy robots that initiate movement in one and three space dimensions.

In addition to an individually adjustable seat position and depth, the couch offers individually adjustable footrests and armrests at the sides. A safety belt and handles provide additional safety for patients during procedures.

It is a challenge for therapists to treat their patient on the border of painful areas and ongoing movements in order to act and intervene as effectively as possible without provoking pain or secondary after-effects.

- The advantage the Mezos therapy seat offers lies in that patients can be secured in such a manner that thanks to ongoing movements they can concentrate on the treatment itself without fearing that it may be painful or the pain-free area may be exceeded by accident. Different positioning and support pads make it possible to transfer weight from the affected extremities. Furthermore, the extremities can be secured with straps and loops to prevent further movement.
- The size and stability of the seat, which is an advantage for anxious, immobile patients, is an additional treatment challenge. A single therapist may find it difficult to operate the device, move it from A to B, while small therapy rooms may create space limitations. We believe that reducing the size and weight and thus enhancing the manoeuvrability of the device through the possible abandonment of some safety components would make it more user friendly.

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5.1.1. Germany

5.1.1.1. Physiotherapie Berten Nachf. Tobias Hamann

“The parameters and settings of the device can be individually adjusted to meet the needs of a particular female or male patient. With the ability to graphically display activities performed in real time, patients find it easier to engage in therapy and, with visible results, are more committed to continuing therapy on a daily basis.”

Tobias Hamann

ASSESSMENT REPORT

The goal of the Tobias Hamann’s physiotherapy practice is to create a modern, competent therapeutic solutions for the health of the patients entrusted to us.

Since 2010, our office has been providing health care to the players of FC Hansa Rostock. The demands placed on our therapists by professional football players mean that we have to continuously follow the innovations and developments in modern (sports) medicine. This allows us to use the experience we have gathered in the field of competitive sports directly in our daily work.

We tested EGZO Tech’s Stella BIO device under trial conditions for treating orthopedic and surgical diseases and disorders. The device is equipped with 4 channels for superficial muscle training and 2 channels for treating pelvic diaphragm dysfunction in both men and women. Various programs for electrical stimulation of the muscular system and EMG Biofeedback programs can be selected for the treatment. In addition, the device provides three different EMG-controlled games. The parameters and settings of the device can be individually adjusted to meet the needs of a particular female or male patient. With the ability to graphically display activities performed in real time, patients find it easier to engage in therapy and, with visible results, are more committed to continuing therapy on a daily basis. Games have an additional positive effect on increasing motivation. The ability to activate specific parts and groups of muscles and the ability to visualize their activation for the exercising patient as well are particularly noteworthy, being essential for recovery in sports.

This small, handy device is suitable specifically for home visits, and it features automatic recording and documentation of treatment and therapy, making it easy for therapists to both keep records and monitor progress and results.

In summary, one may say that Stella BIO is an effective device and an interesting addition to the therapy of patients with various orthopedic and neurological deficits.

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5.1.1.2. Medical Park Chiemsee

“The small, handy device allows the option to document treatment and therapy processes automatically, thus simplifying both documentation for the therapist, as well as any inspections of the treatment course.”

Sebastian Stemmler

Assessment letter: „Stella BIO by EGZOTech“

Respected Ladies and Gentlemen,

I am attaching the requested assessment letter:

The Medical Park Chiemsee is a rehabilitation clinic that operates the following wards: orthopaedics, traumatology and applied sports medicine. We have applied the device Stella BIO by EGZOTech as part of a trial, and used it to treat orthopaedic and surgical complaints. The device operates four channels for topical treatment of muscles, as well as two channels for the treatment of functional deficiencies of the pelvic floor, in women as well as in men. The treatment be based on programmes covering electric muscle stimulation as well as EMG biofeedback schemes. Furthermore, there are three EMG-triggered games available. The parameters and settings of the device may be adapted individually to the patients. The real-time graphic display of the conducted activities allows the engagement of the patients in the therapy, they are motivated by their results to continue the daily therapy. The games improve the motivation further. Within the scope of the test scheme, we have utilised this device for our patients together with targeted muscle development training. We particularly stress the possibility of our patients to activate specific muscle groups or areas with a visual representation of this activation for the patients, something that is necessary for regeneration in sport. The small, handy device allows the option to document treatment and therapy processes automatically, thus simplifying both documentation for the therapist, as well as any inspections of the treatment course.

To summarise, I may say that the Stella BIO is a device that constitutes an effective and interesting amendment to therapy of patients with diverse deficiencies of orthopaedic and surgical nature.

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5.1.1.3. Aviva Berlin

“As therapists, we see the invaluable advantage of the precision and sensitivity that the device ensures. The highly sensitive EMG signals allow us to capture the kind of muscle activity and use it for exercise - a feature that the therapists have not been able to see before.”

Franziska Schlagwein

Aviva Berlin
Pfarrer-Goosmanstr. 11

12489 Berlin

The opinion on the Stella BIO device made by AVIVA Berlin in Berlin-Adlershof

AVIVA Berlin in Berlin-Adlershof is a multi-specialist facility focused on health, movement and contact in a friendly and welcoming atmosphere. We offer dedicated therapeutic procedures in the fields of occupational therapy, physiotherapy and speech therapy as well as sports courses and medical exercise therapy of a rehabilitative and preventive nature.

The Stella BIO device by EGZOTech has been supplementing our offerings for some time now in the form of a therapeutic robot with an electrostimulation function that can additionally work using EMG. In addition, the device offers functional patterns for the upper limbs, which can be applied using EMS and EMG.

In our multidisciplinary team of therapists, we use the Stella BIO mainly with patients affected by neurological disorders as well as a wide range of injuries and hand disorders.

As therapists, we see the invaluable advantage of the precision and sensitivity that the device ensures. The highly sensitive EMG signals allow us to capture the kind of muscle activity and use it for exercise- a feature that the therapists have not been able to see before.

This shows us a whole new spectrum of therapeutic possibilities at a very early stage of rehabilitation. It is not uncommon for new therapeutic solutions to also be opened up to chronically ill patients, which demonstrates the great motivational and uprising potential.

In addition, a great benefit is provided by functional patterns for the upper limbs- using them we can do exercises that consist in performing daily activities with our highly motivated patients, both through stimulation and based on EMG, in order to stimulate neuroplasticity.

It is also worth mentioning the great suitability of Stella for everyday use - its handy size and the fact that Stella can be controlled wirelessly and attached to the patient allows for versatile use of the device.

We are very pleased to have Stella BIO on our team and recommend purchasing the device to be able to use it in many ways!

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Bankverbindung
Commerzbank
DE33 1204 0000 0072 4531 00

5.1.1.4. Theraphysia Hellersdorf

“For us, an additional advantage is the patient’s own activity, which may be initiated by using the device. This makes the entire therapy more active and saves the therapist’s resources, while also gently guiding them towards taking initiative and responsibility.”

Therapy Centre Hellersdorf in Spreecenter
81 Hellersdorfer Street
12619 Berlin

Established in 2016, THERAPHYSIA GmbH is an interdisciplinary centre operating several locations and cooperating with hospitals, retirement and care facilities, day-care centres for children and schools in order to be able to offer and design therapy services for patients in a targeted and practical manner. Our patients suffer from various disorders, including the neurological, orthopaedic and paediatric ones. We have been using Stella BIO to treat neurological and orthopaedic patients for several months now. The diseases we have been treating by using the device cover post-stroke, multiple sclerosis and wrist disorders.

We are using EMS programmes, EMG-derived EMS and playing functionalities of both programmes. We believe that a great added value of using the therapy robot lies in that we are able to visually present our patients where and which capacities they have, and, above all, which capacities even severely affected and chronically ill patients may use.

This has a motivational effect that could hardly be achieved if it weren’t for the device. On one hand, the therapist may use the diagnostic functionality to determine his or her patient’s condition in an objective, detailed and already documented manner. If the therapist finds it beneficial, the results may be shared with the patient.

For us, an additional advantage is the patient’s own activity, which may be initiated by using the device. This makes the entire therapy more active and saves the therapist’s resources, while also gently guiding them towards taking initiative and responsibility.

As part of the therapy, the targeted default offers are particularly beneficial, because an appropriate measure may be selected quickly on the basis of a precise diagnosis, and the reappointment is just as pleasant.

We are highly satisfied with using Stella BIO and are more than ready to use it as a complementary tool in our therapy.

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5.1.1.5. Human Motion

“Furthermore, muscles may be stimulated by triggering muscle activity. Offering a multitude of diagnostic and rehabilitative functionalities in the area of the muscular system, the device becomes an indispensable tool for customised therapy, recovery and effective training.”

Sacha Lubach

Assessment opinion on Stella Bio

Advantages of the combination of modern diagnostics & therapy devices

Sacha Lubach - human motion

Using the example of **StellaBIO** EMG/FMS device and an anti-gravity treadmill

Both therapy devices offer many advantages, which may be used in prevention and rehabilitation. The following opinion will in particular focus on the possible combinations for orthopaedic and neurological indications. Combining the two devices implies that Stella BIO can be used in an even more targeted, effective and functional manner.

StellaBIO 4-channel EMG/FES System (electromyography/functional electrical stimulation)

The 4-channel EMG/FES therapy robot StellaBIO can not only visualise muscle activity by means of the world's most sensitive real-time EMG (superficial electrodes and/or pelvic floor probes), but can also activate the muscle. This is achieved through electrical muscle stimulation, which can also be operated with the use of 4 superficial EMG channels and 2 channels for internal pelvic floor probes.

Furthermore, muscles may be stimulated by triggering muscle activity.

Offering a multitude of diagnostic and rehabilitative functionalities in the area of the muscular system, the device becomes an indispensable tool for customised therapy, recovery and effective training.

Weightless treadmill

In this respect, Stella BIO is combined with a weightless treadmill. Referring to our target group, which also includes professional football players, we design the therapy to be as active and functional as possible. In case of rehabilitation or preparatory preoperative fitness training, the weightless treadmill should ensure freedom from pain and fall safety while walking and running with no body weight felt. The joint-friendly treadmill includes exercises with a closed kinetic muscle chain and allows to gradually reduce body weight by 1% up to 80% as a maximum. This enables longer, more functional and more intensive training and therapy sessions and an increase in the volume of training and therapy while reducing the risk of strain-related injuries.

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5.1.1.6. Therapeuticum Gehrden

LunaEMG and StellaBIO evaluation letter from the Therapeuticum Gehrden team

“Stella BIO is also very versatile. We often use this device as an adjunct to therapy with Luna EMG in patients with flaccid hemiparesis or, through the use of vaginal and anal electrodes, to treat pelvic floor problems.”

Alisa Barthel

Therapeuticum Gehrden is a rehabilitation centre based in Gehrden. Our concept combines physiotherapy, ergotherapy and speech therapy using computer-assisted, research-based methods and robotics. Our practice focuses on the treatment of neurological conditions that occur, for example, after strokes or in the course of Parkinson's disease.

Because of our specialisation, we purchased LunaEMG and StellaBIO equipment. Both devices immediately made a great impression on us.

For patients with flaccid hemiparesis of the upper limbs, the LunaEMG with its built-in and highly sensitive muscle EMG measurement function has simply become indispensable for us. Patients feel extremely motivated if they can see on the screen exactly how much they can activate their muscles. They then focus on the screen rather than the limb itself. Thanks to the games installed on the LunaEMG device, patients do not even notice how often they repeat particular movements and how intensely they exercise.

The StellaBIO is also distinguished by its EMG-triggered electrostimulation function. StellaBIO is also very versatile. We often use this device as an adjunct to therapy with LunaEMG in patients with flaccid hemiparesis or, through the use of vaginal and anal electrodes, to treat pelvic floor problems. Thanks to acoustic and visual signals, patients feel the triggering of muscles, such as the pelvic floor, very precisely.

Thanks to the very good integration of the individual elements of motor learning and thus the good activation of specific brain areas, we would never want to give up the use of LunaEMG and StellaBIO in the treatment of our patients with neurological disorders.

Last but not least, the professional support of the EgzoTech team should be further emphasised. We always received answers to all our questions as quickly as possible.

Alisa Barthel and team

Translated by
Biuro tłumaczeń
Lingua Lab

Headquarters:

Therapeuticum Gehrden
Dammstraße 18
30989 Gehrden, Germany

The person issuing the reference:

Alisa Barthel - Physiotherapeutic
management, Owner

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Alisa Barthel und Team

5.1.1.7. Move forward Ergotherapie

“The straightforward and visualised interface makes it easy for therapists to seamlessly integrate the device into daily practice. The multiple uses of the electrodes also contribute to environmentally friendly practices. The compact size of the device allows for easy portability during home visits.”

Katharina Vöst

EgzoTech's evaluation letter "Stella und Luna"

We have been using EgzoTech's modern and innovative equipment for therapeutic purposes every day in our practice for more than four years. LUNA and STELLA devices used in the treatment of neurological and orthopaedic patients.

Luna is proving to be an excellent therapeutic device for patients with varying degrees of disease. Patients receive immediate visual and auditory feedback in response to their motor actions. The Luna device also offers a passive movement function, which makes the therapist's work much easier. Particularly important is the ability to use the device for upper and lower limb exercises, which makes it indispensable for practice.

Stella enables the use of electricity as part of therapy. We use the Stella device to improve perception, control abilities, enhance strength, reduce muscle tension and for many other purposes. Patients love this kind of news! Simple and visualised operation makes it easy for therapists to optimally integrate the device into their daily work. Reusable electrodes make the device environmentally friendly. Thanks to its compact dimensions, the device can be conveniently taken to home visits.

Move forward Ergotherapie
Katharina Vöst

Move forward
Ergotherapie Katharina Vöst

Peter-Dörfler-Str. 30
86199 Augsburg
www.move-forward-ergotherapie.de

Phone 0821 470 12733
Fax 0821 470 12734

Headquarters:

Move forward Ergotherapie
Katharina Vöst
Peter-Dörfler-Straße 30, 86199
Augsburg, Germany

The person issuing the reference:

Katharina Vöst - Occupational therapist,
owner

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5.1.2. Czech Republic

5.1.2.1. University Hospital Brno

“EMG reading is an added advantage, and thus allows for further new possibilities for motor re-education in both neurological and orthopedic patients.”

Marcela Nováková

Simona Šrubařová

DRO Nový Lískovec
Zoubkova 18
623 00 Brno, Czech Republic

Evaluation of the STELLA BIO device

We borrowed EGZO Tech's STELLA BIO device for a month.

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The device can be used in classic electrotherapy with an indication of pain reduction – although without offering such a wide range of currents, but with the possibility of connecting more electrodes. Electrostimulation programs offer already pre-set combinations, the choice of possible pulse types matches the indication group, as well as the diagnostics and training statistics.

This offers the advantage of having more electrodes connected, thus involving more muscle groups in the motor pattern activation.

EMG reading is an added advantage, and thus allows for further new possibilities for motor re-education in both neurological and orthopedic patients.

The Internet connection is a disadvantage.

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5.1.3. Poland

5.1.3.1. Functional Rehabilitation Center ORTHOS

“Monitoring the work of selected superficial muscles allows for more accurate work with the patient, matching the exercises to his or her individual needs in a better way.”

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Warszawa, 01.03.2021

ORTHOS
Functional Rehabilitation Center

OPINION

We have had the Stella BIO EMG device from EGZO Tech in our Orthos Functional Rehabilitation Center since 01.01.2021. I personally use Stella to work with children and adolescents with neurological problems, specifically with myopathies and dystrophies. Stella BIO EMG present in our facility is equipped with 4 channels of superficial EMG + 2 attachment channels for pelvic muscle stimulation. There are many readily available muscle stimulation and EMG Biofeedback programs, as well as 3 EMG games. It is also possible to adjust the parameters of the device to the individual needs of the patient and the planned therapy.

Monitoring the work of selected superficial muscles allows for more accurate work with the patient, matching the exercises to his or her individual needs in a better way. Stella BIO EMG is able to record muscle activity, even with little bioelectrical activity in the muscles. Each training session is recorded in the patient's individual record, allowing to monitor the rehabilitation process and generate reports necessary for medical files.

In my work, I mainly use EMG Biofeedback during exercises and various patient activities. I work with younger patients using available EMG games, which are very motivating. The device is very easy to use, and scheduling therapy for the patient who, after being instructed, can have EMG Biofeedback training sessions at home provides great control over the prescribed exercises.

Based on our experience to date, I recommend the Stella BIO EMG device as an additional tool for working with individual patients, as well as continuing the prescribed exercises at home.

MSc Hubert Grycza
Physiotherapist

Headquarters:

Functional Rehabilitation Center
ORTHOS
Belgradzka 18 Street
02-793 Warsaw
Poland

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5.1.3.2. COZL Rehabilitation Centre

“Stella Bio allows for neuromuscular re-education, muscle strengthening and relaxation, improved coordination and EMG biofeedback. The effects of the therapy are visible as early as after 2-4 weeks of use, which is why the device enjoys positive opinions among patients.”

MSc Adrian Białas

MSc Paula Syty

MSc Ewelina Pytel

Lublin 23.04.2021

OPINION ON STELLA BIO

We have had the Stella Bio device at the COZL Rehabilitation Center since January 2021. We use the device to work with orthopedic, neurological, and oncological patients, as well as those with pelvic floor muscle disorders.

Stella Bio allows for muscle and nerve electrostimulation with surface electrodes, electromyography and biofeedback, as well as pelvic floor muscle stimulation and biofeedback. The device is small and easy to use; it has a special app that is intuitive to use. There are five categories available at Stella Bio: neuro, ortho, pain, pelvic and sport. Each of these categories contains different EMG biofeedback and electrostimulation programs with already defined current parameters and a program for manual settings. The most common programs we use are neuro, ortho and pelvic.

Pelvic floor muscle training is often provided in cases of urological, gynecological and oncological patients. We use both electrostimulation and EMG biofeedback software. The procedure is hygienic and intimate thanks to an individual vaginal or rectal probe. The patient, when tensing the pelvic floor muscles, sees the effect on the computer screen in the form of a graph showing muscle tension over time, and can also control a computer game – such a biofeedback increases the effectiveness of the exercises and provides actual image of the training effect. In addition to the obvious strengthening of these muscles, electrostimulation of the pelvic floor muscles also helps to localize them, making it easier to exercise.

For surface electrode treatments, we most commonly use the Ortho programs for muscle recovery and strengthening after orthopedic injuries, and Neuro for improving nerve and muscle function in cases of partial denervation. After selecting an exercise category and program, you select the muscle(s) to be stimulated. You can see the placement of the electrodes and their polarity next to each muscle in the program, making the procedure much easier to perform.

Stella Bio allows for neuromuscular re-education, muscle strengthening and relaxation, improved coordination and EMG biofeedback. The effects of the therapy are visible as early as after 2-4 weeks of use, which is why the device enjoys positive opinions among patients.

The only disadvantages of using Stella Bio are related to server problems that occur once every 2 weeks on average. A game like Brick Pirates often stutters or does not work properly, without responding to muscle tension. During pelvic floor muscle biofeedback exercises, the graph sometimes speeds up twice and the patient cannot keep up (the exercise goes twice as fast as it should). Then again, the voltage-time dependence scale has too extensive values for higher intervals, and there are no intermediate values to better visualize muscle contraction during exercise. Nevertheless, in case of technical problems, Egzotech always reacts very swiftly and repairs the defect, which we appreciate very much.

In conclusion, our team would like to recommend the Stella Bio device for working with orthopedic, neurological, and oncological patients, as well as those with pelvic floor muscle disorders.

Translated by www.lingualab.pl

Headquarters:

COZL Rehabilitation Center
Jaczewskiego 7 Street
20-090 Lublin

The person issuing the reference:

MSc Adrian Białas - Head of the Rehabilitation Center
MSc Paula Syty - Physiotherapist
MSc Ewelina Pytel - Physiotherapist

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Ewelina Pytel
mgr fizjoterapeuta
26481

5.1.3.3. Białystok Oncology Center

“Stella BIO is primarily used for electrostimulation for paralysis and paresis of the muscles and electrostimulation in lymphedema. As a result of rehabilitation, we are able to improve the functional status of our patients.”

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- malignant tumors of the breast,
- cancers of the reproductive organs,
- urinary tract cancers,
- melanomas,
- brain tumors, and others.

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Stella BIO - Mobile electromyograph with triggered electrostimulation for oncological rehabilitation, is primarily used for

- TENS in pain syndromes,
- electrostimulation for paralysis and paresis of the muscles,
- electrostimulation in lymphedema.

As a result of rehabilitation, we are able to improve the functional status of our patients.

Best regards

dr n. med. Joanna Filipowska
Head of Department of Rehabilitation
Białystok Oncology Center
Ul. Ogrodowa 12 15-027 Białystok
e-mail: jfilipowska@onkologia.bialystok.pl tel. 85 6646813

LINGUA LAB s.c.
W. Szyszkiwicz, M. Dąbnińska
ul. Szosza Fana 5/14, 31-143 Kraków
NIP 677-205-31-15 REGON 121429503

Translated by www.lingualab.pl

Headquarters:

Department of Rehabilitation
Białystok Oncology Center
Ogrodowa 12
15-027 Białystok

The person issuing the reference:

Dr. Joanna Filipowska - Head of Department of Rehabilitation

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Dr n. med. Joanna Filipowska

5.1.3.4. "Kwitnąca" Rehabilitation and Care Center

"The system allows for keeping records in the form of patient's card, thanks to which we can monitor their progress and use the reports from the trainings to supplement the medical documentation. Based on past experience, I would like to recommend Stella BIO in everyday work."

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Zagórze Dworskie, 31.12.2021

Opinion

We have the Stella BIO device in the Kwitnąca Center since 13.12.2021.

Stella Bio combines a mobile electromyograph and electrostimulation triggered by patient's muscle activity. In our everyday work, we use the device for the therapy of patients with flaccid muscle paralysis using the possibility of intramuscular electrostimulation, EMG Biofeedback diversifying the therapy with prepared EMG games.

The device is equipped with prepared therapy suggestions in the form of ready-made programs, which can also be modified according to the patient's choice and needs. The system allows for keeping records in the form of patient's card, thanks to which we can monitor their progress and use the reports from the trainings to supplement the medical documentation.

Based on past experience, I would like to recommend Stella BIO in everyday work.

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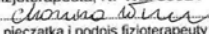
mgr. **Monika WILK**
fizjoterapeuta, NPWZ: 59914

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5.1.3.5. Desiderata Nursing Home

“We use it most often when working with people with urinary incontinence. After just 10 treatments, we see significant improvement in both symptoms and overall quality of life for our residents. Greater bladder control in our patients also translates into greater comfort for caregivers.”

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Opinion

Stella BIO device

We have been using the Stella BIO device at the Desiderata Nursing Home since 6.10.2021. We use it most often when working with people with urinary incontinence. After just 10 treatments, we see significant improvement in both symptoms and overall quality of life for our residents. We use both EMG Biofeedback and electrostimulation programs in our work. Greater bladder control in our patients also translates into greater comfort for caregivers. We plan to further develop our work using this device, gaining further valuable experience in the area of working with the pelvic floor. We recommend the Stella BIO device for working with residents in any nursing home.

Ewa Wączek
Desiderata Nursing Home Director

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The person issuing the reference:

Ewa Wączek - Director of the Desiderata nursing home

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5.1.3.6. UFMED Massage Studio

"On the screen, patients watched as they activated their muscles, which resulted in a better sense of their own bodies, and pelvic floor muscle training itself had a beneficial effect on symptoms of stress urinary incontinence. Therapy was more comprehensive and we were able to impact not only the physical health of our patients, but also their quality of life."

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Opinion

Stella BIO

We used the Stella BIO device at the UFMED Massage Studio in pelvic floor training for patients with symptoms of stress urinary incontinence, or as prevention of pelvic floor dysfunction. On the screen, patients watched as they activated their muscles, which resulted in a better sense of their own bodies, and pelvic floor muscle training itself had a beneficial effect on symptoms of stress urinary incontinence. Our therapists were also able to provide better assessment, if the patient was able to maintain muscle tone, if she was compensating using other body areas or if she was able to properly relax her pelvic floor.

By using the device in a holistic approach to the patient, the therapy was more comprehensive and we were able to impact not only the physical health of our patients, but also their quality of life.

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UFMED Massage Studio
Dobra 8/10
00-318 Warszawa

The person issuing the reference:

Małgorzata Bąk - Founder of the UFMED Studio

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5.1.3.7. Regional Center of Sports Medicine SPORTVITA

"Monitoring the effects of therapy in some neurological patients would be impossible without this device. It is extremely useful for patients who are assigned a value of 0 on the Lovett's muscle strength rating scale, as this device allows them to record a measurable value of the generated muscle tension."

Dr Andrzej Chęsy

Headquarters:

Regional Center of Sports Medicine
SPORTVITA
Rejtana 1
85-032 Bydgoszcz

The person issuing the reference:

Dr Andrzej Chęsy - Specialist in Physiotherapy

Bydgoszcz, 30.12.2021

OPINION

Exotech's Stella BIO EMG device became available to our facility for use in December 2021. We have orthopedic, neurology and urology outpatient clinics in the facility, and the patients of these clinics were able to use the aforementioned device. The possibility of using this device provided a lot of satisfaction to the patients, which was reflected in their subjectively declared opinions, as well as to the therapists, enabling them to make more complete medical records according to the latest standards.

Working with Stella allows for a very thorough analysis of the patient's problem and effective therapy, as well as measurable follow-up. Monitoring the effects of therapy in some neurological patients would be impossible without this device. It is extremely useful for patients who are assigned a value of 0 on the Lovett's muscle strength rating scale, as this device allows them to record a measurable value of the generated muscle tension.

The ability to record each exercise session makes it easy to keep records and generate reports for patients that show them the effects of their work with the device.

Two of our employees had the opportunity to work with EMG for the first time on patients after urological procedures and, thanks to the experience they gained with the device, they can no longer imagine working with these patients without EMG. Stella and the rectal and vaginal electrodes dedicated to urogynecological patients are recommended to all physiotherapists as an essential tool in therapy.

I would express my gratitude to Dr. Dominika Kozak for making the device available.

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5.1.3.8. Mazovian "Bródnowski" Hospital

"We work with adults in our facility. The device is used to treat patients with neurological and orthopedic conditions. The working range of the device allows for the selection of individual optimal therapy for patients with both muscle paralysis and muscle weakness from inactivity."

Iwona Lamprecht

Mazowiecki Szpital Bródnowski w Warszawie Sp. z o.o..
Department of Rehabilitation and Physical Therapy
ul. Kondratowicza 8, 03-242 Warszawa

OPINION

EGZOTech's Stella BIO EMG device is a part of MSB equipment as of November 9, 2020.

We work with adults in our facility. The device is used to treat patients with neurological and orthopedic conditions.

The working range of the device allows for the selection of individual optimal therapy for patients with both muscle paralysis and muscle weakness from inactivity.

The device allows you to monitor the progress of your patient's treatment with the ability to record parameters on an individual patient card and the ability to generate reports.

The Stella BIO EMG device is a state-of-the-art piece of equipment that has proven to work well in our facility.

Physiotherapy Department Coordinator
Iwona Lamprecht

LINGUA LAB s.c.
W. Szyszkiewicz, M. Dambirska
ul. Serego-Fam'a 5/14, 31-145 Kraków
NIP 277-225-31-15 REGON 121420823

Translated by www.lingualab.pl

Headquarters:

Mazovian "Bródnowski" Hospital
Kondratowicza 8
03-242 Warszawa

The person issuing the reference:

Iwona Lamprecht - Physiotherapy Department Coordinator

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5.1.3.9. Health Zone - Łochowo

Łochowo, 10.05.2022

“We have been using Stella Bio EMG device since January 2022. Since that time, our patients, mainly with orthopaedic and neurological diseases, had the procedures performed using the device.”

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Opinion

We have been using the Stella BIO EMG device since January 2022. Since that time, our patients, mainly with orthopaedic and neurological diseases, had the opportunity to benefit from using the device. Our patients expressed a positive opinion on the procedures performed using the device.

The Stella BIO EMG device allows for precise and effective therapy. In addition, thanks to the possibility of saving each treatment session, keeping records is easier and more complete. In the opinion of both patients and therapists, the generation of reports, that show the effects of training, is also an interesting solution.

The device is intuitive to use, the program control panel allows for an easy selection of a built-in program or the modification of the already designed programs in order to further individualize the therapy.

Many thanks to Dr. Dominika Kozak for the opportunity to work with the Stella BIO EMG device from Egzotech.

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Bydgoszcz, 12.05.2022

OPINION

“The possibility of saving each training session made the control of the course of treatment and monitoring of the progress much easier.”

Ph.D Grzegorz Sielski

We have been using the Stella BIO EMG device from Egzotech since April 2022. Our facility rehabilitates patients with various diseases, mainly people after stroke. Following the cycle of therapy, our patients reported satisfaction with the results, while the innovative solutions of the device aroused their interest and willingness to continue the treatments.

The Stella BIO EMG device allowed for an increase of the muscle strength and reduction of pain. Including so many procedures and treatments in one program made the planning of rehabilitation easier and the selection of treatments appropriate to a given disease. Thanks to the multifunctionality of the device, the time between various stages of the rehabilitation process has been reduced. The device also helped to control the effects of therapy.

The possibility of saving each training session made the control of the course of treatment and monitoring of the progress much easier. The data saved in the patient's card was a supplement to the standard medical records.

Both our physiotherapists and patients benefited from using the Stella BIO EMG device. Thanks to significant functional improvement of the rehabilitation procedure, they noticed a growing need to introduce new technologies in the therapy.

Many thanks to Dr. Dominika Kozak for cooperation and making the equipment available to our physiotherapists and patients.

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“The Stella BIO device has many functions and gives various opportunities. In the work with neurological patients, electrostimulation can be used - either alone or triggered by EMG, in this way We can improve the functions of the paretic side.”

MSc. Beata Zgierska

The Cardinal Stefan Wyszyński Provincial Specialist Hospital
Independent Public Health Care Centre
20-718 Lublin, al. Kraśnicka 100
DEPARTMENT OF REHABILITATION – SUBUNIT OF NEUROLOGICAL REHABILITATION
phone: 81 537 45 78, e-mail: rehabilitacja@szpital.lublin.pl

Lublin, 21.06.2022

OPINION

We have been using Stella BIO, a modern electrostimulation device, that also uses EMG, for half a year. During this short time, all our patients could benefit from using this equipment and enjoy its advantages.

The Stella BIO device has many functions and gives various opportunities. In the work with neurological patients, electrostimulation can be used – either alone or triggered by EMG, in this way we can improve the functions of the paretic side. There is an option of massage, relaxation, sports training, pain reduction, thanks to the use of TENS currents, and many more. The device is used not only in neurological patients, but also in people with orthopaedic and oncological diseases. It is a very interesting and useful variety in our work.

Translated by www.lingualab.pl

Headquarters:

Independent Hospital Public Health Care
Centre
20-718 Lublin, al. Kraśnicka 100

The person issuing the reference:

MD Anna Mazur-Maj - Head of the Department of Rehabilitation
MSc Beata Zgierska - physiotherapist

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5.1.3.12. University Hospital named by Karol Marcinkowski in Zielona Góra

Zielona Góra, 13.06.2022

“The device is very popular both among physiotherapists and patients, because it is practical in everyday therapy.”

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OPINION

Since December 2021, we have been using the Stella BIO EMG device from EGZOTech in the Department of Rehabilitation with the Subunit of Neurological Rehabilitation of the University Hospital in Zielona Góra.

The device is generally used in the neurorehabilitation of adult patients with diseases of the central nervous system and in the treatment of urogynecological diseases.

The built-in therapeutic programs, EMG biofeedback and the possibility of individual selection of parameters depending on the patient's needs allow for versatility and variety of therapies used among our patients.

The device is very popular both among physiotherapists and patients, because it is practical in everyday therapy.

Given the above, we recommend the device as an additional tool to relieve the work of the therapist and to provide favourable biofeedback to the patient.

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5.1.3.13. SPSK No. 4

Lublin, 25 August 2022

“It perfectly complements urogynaecological therapy. It can be used both in the acute and chronic phase of a disease.”

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**Reference
for the Stella BIO device**

The Stella BIO device has been owned by our rehabilitation unit since June this year.

The Stella BIO device has enabled us to expand the scope of physiotherapy treatments with new forms of rehabilitation of neurological and orthopaedic patients. It enables modern functional therapy of the upper extremity using the functional electrostimulation system, biofeedback and EMG. EMG employs the active effort of patients to the extent that is optimally suited to their capabilities. It perfectly complements urogynaecological therapy. It can be used both in the acute and chronic phase of a disease.

The Stella BIO device is not only a therapeutic device, but also a diagnostic one. The therapy of a patient is recorded at each stage of rehabilitation, which enables an objective assessment and analysis of its progress.

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5.1.3.14. Independent Public Clinical Hospital named after prof. Adam Gruca CMKP

Warszawa, 16.11.2022

“Stella BIO provides the opportunity to make objective assessments using EMG, which contributes to expanding the ability to determine patients' functional status and facilitating medical record keeping.”

Professor Małgorzata Łukowicz

OPINION

Egzotech's Stella BIO EMG device was made available for use at our facility in October 2022. The ability to use the latest equipment has helped increase the efficiency of the therapeutic process carried out in a group of patients with orthopedic conditions and peripheral nerve damage.

The wide range of available programs and the ability to individualize specific programs gives the opportunity to fully adapt the therapy to the functional needs of the patient, as well as to plan the rehabilitation process according to the results obtained. The combination of functional electrostimulation and EMG allows for varied programs to be carried out during a single appointment.

Stella BIO provides the opportunity to make objective assessments using EMG, which contributes to expanding the ability to determine patients' functional status and facilitating medical record keeping. An additional benefit for therapists is the ability to analyze reports to monitor the results obtained

The subjective opinions of patients, which mainly concern the pleasant sensory experience and the perceived motor effects of the stimulated structures, were a significant factor contributing to the positive assessment of the device. Subjects who received training based on EMG use indicated an increase in task participation, resulting in an increased sense of control over movement, satisfaction with performance, and enhanced motivation.

In addition, occupational therapists note the device's strengths through specifically designed functional programs that allow them to conduct training aimed at improving the performance of the activities of daily living.

Many thanks to Egzotech for the opportunity to use the innovative device and to Dr. Dominika Kozak for the comprehensive training.

Translated by Lingua Lab
Nowogrodzka 50 lok. 515
00-695 Warszawa

Headquarters:

Independent Public Clinical Hospital
named after prof. Adam Gruca CMKP
Stanisława Konarskiego 13
05-400 Otwock

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5.1.3.15. Manual-Med

23.11.2022, Lublin

“Working with Stella BIO yields very satisfactory results and opens up a range of possibilities for working with patients beyond neurological patients. A wide range of ready-made programs enables faster and highly promising therapy with visible results.”

MSc Mateusz Daktera

Manual-Med sp. z o.o.
al. Warszawska 54
20-803 Lublin

OPINION

We have used the Stella BIO device in our facility since 22.08.2022. The target patient was a pediatric neurological patient. Working with Stella BIO was supposed to enable us to rebuild muscle fibers using an electrostimulation program, evaluate the therapeutic effects to date, and support control and tension sensing with the help of an EMG Biofeedback program. After several weeks of daily work, the following results were achieved:

- improved tension and nutrition of soft tissues
- increased awareness of the paralyzed limb sensation
- increased motor control
- enabling the restoration of the normal movement pattern in the shoulder joint under suitably adapted conditions, i.e. working in a relieved position

Working with Stella BIO yields very satisfactory results and opens up a range of possibilities for working with patients beyond neurological patients. A wide range of ready-made programs enables faster and highly promising therapy with visible results. At the same time, it is worth noting that the idea behind the developing Stella BIO was, e.g., to create the possibility for the patient to work independently at home. Given the foregoing, it seems to us that it would be appropriate to simplify the setting of treatment parameters, possibly creating the remote operation capabilities for the therapist.

In summary, we recommend working with Stella BIO and believe that it is a very helpful therapeutic tool, yet one that requires subtle technical and program adjustments.

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5.1.3.16. Senior Villa-Strumiany

Strumiany, 21 11 2022 r.

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“This device is not only a good complement to therapy, but can also constitute its basis. Stella Bio gives us the opportunity to monitor the course of therapy and evaluate its progress in real time.”

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MSc Katarzyna Rynkiewicz - physiotherapist

Strumiany, 21 11 2022.

OPINION CONCERNING THE STELLA BIO DEVICE

For the past month, our center has had the opportunity to test the Stella Bio device. Stella Bio is a rehabilitation equipment that provided us with the opportunity to work with a diverse group of patients. This device is not only a good complement to therapy, but can also constitute its basis. Stella Bio gives us the opportunity to monitor the course of therapy and evaluate its progress in real time. The simplicity of the device's operation, which makes patient preparation much faster and allows the patient to move freely during the procedure, is particularly noteworthy.

Translated by Lingua Lab
Nowogrodzka 50 lok. 515
00-695 Warszawa

5.1.3.17. Bonifraterskie Physiotherapy Center

“ The Stella BIO device has given us the opportunity not only for stimulation, but importantly for muscle diagnostics. In case of neurological patients, Stella BIO complemented the rehabilitation process brilliantly, using functional electrostimulation, biofeedback and EMG.”

MSc Barbara Piotrowska

Bonifraterskie Centrum Medyczne sp. z o.o.
Bonifraterskie Physiotherapy Center in Piaski
Physiotherapy Department

Piaski 23.01.2023.

OPINION FOR THE STELLA BIO DEVICE AT BONIFRATERSKIE PHYSIOTHERAPY CENTER IN PIASKI

The STella BIO device has given us the opportunity not only for stimulation, but importantly for muscle diagnostics. In case of neurological patients, Stella BIO complemented the rehabilitation process brilliantly, using functional electrostimulation, biofeedback and EMG.

The recording of therapy at each stage of the patient's rehabilitation is a very important aspect of the device, allowing a clear analysis of its progress.

We would like to thank EGZO TECH for the opportunity to work with this equipment. Stella BIO device received my positive opinion.

Translated by Lingua Lab
Nowogrodzka 50 lok. 515
00-695 Warszawa

Headquarters:

Bonifraterskie Physiotherapy Center
Piaski Branch
Marysin 1
63-820 Piaski

The person issuing the reference:

MSc Barbara Piotrowska - Head of Hospital Physiotherapy Department

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mgr Barbara Piotrowska

5.1.3.18. Pojda Rehabilitation and Physiotherapy

“ The device aroused considerable curiosity and activity in patients, mainly those who suffered strokes. Using EMG during the therapeutic processes was undoubtedly and interesting and valuable experience in therapeutic processes.”

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Pojda Rehabilitation and
Physiotherapy
Przemysław Pojda

Żory 20.12.2022

Opinion

Courtesy of Exotech, we had the opportunity to test two devices at our facility - Luna EMG and Stella Bio.

We have tested the devices on many patients with diverse disorders. These included people with MS, post-stroke, children with cerebral palsy, athletes or people after surgeries.

Conclusions: Luna EMG:

- The EMG system, interface, appearance of Luna and training selection are clear and intuitive. Based on our short experience, we are most satisfied with motor facilitation in the peripheral joints (knee, ankle, elbow and wrist).
- We experienced significant difficulties using the shoulder joint tip, this applies to ranges of motion above 90/100,
- In young patients, e.g. children with cerebral palsy, we could not fully use the device for facilitating the hip extension, because the part of the tip to be placed on the femur was too big, and the starting position itself was difficult or impossible to set in many cases. Another difficulty we experienced were the frequent overload messages that occurred when trying to facilitate extension in growing patients. The weight of the limb, excluding spasticity, was too much of an overload for the mechanism that carries the moving tip.
- When facilitating ankle movements, lowering the Luna to a level lower than currently available would provide great help.
- A final note concerning the practical operation of the device: I would recommend using a different system for lifting and locking the wheels. Transporting Luna to different rooms and offices, as well as adjusting it to the patients that requires constant bending to reach the wheels, sometimes cause great difficulties.

Conclusions regarding Stella Bio:

- the software is clear and well-developed,
- the device is compact and easy to use,
- the device aroused considerable curiosity and activity in patients, mainly those who suffered strokes.

Using EMG during the therapeutic processes was undoubtedly an interesting and valuable experience in therapeutic processes. We see the point of investing in this type of equipment in the future. The comments we included concerned our difficulties in operating the equipment, and we hope they will be used to improve future equipment. Thank you for the good contact with our assistants Dominik and Katarzyna.

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5.1.3.19. Physioclinic Orthopedic Physiotherapy

“The device clearly enriches the therapeutic range in the physiotherapy office, allows to optimize and modernize the physiotherapy process, and is also a relief for the treating practitioner, as the patient does not require constant supervision during biofeedback exercises.”

MSc Dominik Buzalski

PHYSIOCLINIC Orthopedic Physiotherapy

We use the Stella Bio device at our facility to work with orthopedic and trauma patients. It is a functional tool used to monitor the ongoing activity of muscles undergoing kinesiotherapy. It works very well as biofeedback, making it easier for the patient to control selective muscle activity. In addition to biofeedback, the software allows for targeted electrostimulation in various modes, tailored to the patient's current condition, which definitely sets the device apart in the market.

The electrostimulation module is also extended with EMG-triggered EMS, which allows the patient to achieve full muscle contraction, which is often impossible to do unassisted, for example, after long-term immobilization. Games, which control muscle tension measured with an EMG module, are also an interesting solution - they make the treatment process more attractive and motivate the patient to put more effort into treatment.

The device clearly enriches the therapeutic range in the physiotherapy office, allows to optimize and modernize the physiotherapy process, and is also a relief for the treating practitioner, as the patient does not require constant supervision during biofeedback exercises.

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Physioclinic Orthopedic Physiotherapy
Nowotoruńska 8
85-839 Bydgoszcz

The person issuing the reference:

MSc Dominik Buzalski - owner of the Physioclinic Orthopedic Physiotherapy

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5.1.3.20. Provincial specialist Children's Hospital named after prof. Stanisław Popowski in Olsztyn

“What is most important in working with Stella is a quick and visible effect and improvement of functions, and at the same time the possibility of documenting patients' progress. Thanks to the reports, progress can be measured in a measurable way, which translates into better work motivation for the patient.”

MSc Danuta Krzywda

Opinion

In December 2021 Stella BIO EMG equipment from Egzotech was purchased for us. We work in the Rehabilitation Clinic of a children's hospital and we have a wide range of patients, however, we most often use Stella for neurological patients. What is most important in working with Stella is a quick and visible effect and improvement of functions, and at the same time the possibility of documenting patients' progress. Thanks to the reports, progress can be measured in a measurable way, which translates into better work motivation for the patient. Satisfaction of patients is also greater satisfaction of the physiotherapist's work. Stella Bio is the next step in the work of the Clinic, it extends the range of treatments performed while supplementing the medical documentation, which is a standard nowadays.

Translated by Clinical Specialist @ EGZOTech
Alicja Nogala

Headquarters:

Provincial specialist Children's
Hospital named after prof. Stanisław Popowski in Olsztyn
Żołnierska 18a
10-651 Olsztyn

The person issuing the reference:

MSc Danuta Krzywda - specialist in physiotherapy

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5.1.3.21. Columna Medica

“Stella BIO gives you the opportunity to stimulate and diagnose muscles. Both in the case of neurological and urogynecological patients, it complements the rehabilitation process very well, using electrostimulation, biofeedback and EMG”

MSc Karolina Hajdukiewicz

Opinion for the Stella Bio device in Columna Medica

The Stella Bio device gives you the ability to stimulate and diagnose muscles. Both in the case of neurological and urogynecological patients, it complements the rehabilitation process very well, using electrostimulation, biofeedback and EMG. Thanks to the possibility of recording therapy at each stage, it provides a reliable analysis of the patient's rehabilitation progress.

We would like to thank the EGZO TECH company for the opportunity to work on this equipment. I express a positive opinion about Stella Bio equipment.

Translated by Clinical Specialist @ EGZOTech
Alicja Nogala

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5.1.3.22. University Children's Clinical Hospital named after L. Zamenhof in Bialystok

“Working with the device guarantees a thorough analysis of dysfunctions and effective therapy as well as its control at every stage of the therapeutic process. Patients were satisfied with the effects of therapy: improvement in functioning and quality of life”

MSc Ewa Rakowicz

We had the opportunity to use the Stella BIO EMG device from EGZOTECH in January 2023. Our patients are children and teenagers up to 18 years of age. They present with many different conditions, most cases are neurological and orthopedic patients. Thanks to Stella, we were able to select parameters for optimal individual therapy.

The device was useful in patients whose muscle strength is rated at 0, because Stella's sensitivity allows the registration of minimal tension. Working with the apparatus guarantees a thorough analysis of dysfunctions and effective therapy as well as its control at every stage of the therapeutic process. Patients were satisfied with the effects of therapy: improvement in functioning and quality of life.

Saving parameters on an individual patient card allows you to quickly generate reports and monitor progress. Therapists were able to keep medical records according to the latest standards. We have made efforts to raise funds and purchase the device.

Translated by Clinical Specialist @ EGZOTech
Alicja Nogala

Headquarters:

University Children's Clinical Hospital
named after
L. Zamenhof in Bialystok
17 J. Washington Street
15-274 Bialystok

The person issuing the reference:

MSc Ewa Renata Rakowicz - team leader of physiotherapists

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5.1.3.23. Independent Public Health Care Center in Łapy

Opinion

“In working with neurological patients, we mainly used EMG programs. Thanks to this function, patients learned the correct tension of individual muscle groups and the movement pattern faster.”

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In our facility, we used Stella BIO mainly in working with patients with orthopedic injuries and with neurological problems.

In orthopedic patients, Stella BIO in combination with exercise gave very good results in rebuilding muscle strength and mass and improving muscle balance. In working with neurological patients, we mainly used EMG programs. Thanks to this function, patients learned the correct tension of individual muscle groups and the movement pattern faster.

The convenient and portable design of the device allows you to work in a gym without affecting the quality of the exercises performed by the patient.

Stella BIO well complements the work of a physiotherapist and accelerates the recovery of patients.

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Translated by Clinical Specialist @ EGZOTech
Alicja Nogala

Headquarters:

Independent Public Health Care
Center in Łapy
Department of Physiotherapy
st. Piaskowa 9
18-100 Łapy

The person issuing the reference:

MSc Mateusz Szyluk - Deputy Head of Medical Rehabilitation

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5.1.3.24. Mozaika Foundation

“Working with Stella BIO allows us to analyse the patient's problem very precisely, provide more effective therapy and precisely monitor progress. The therapist can obtain feedback on muscle activity even when their contraction is not visible to the naked eye.”

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Opinion
Stella BIO device

The Stella BIO device from Egzotech was made available to our facility in April 2023. The Mozaika Foundation's special school and kindergarten deals with the education and rehabilitation of children and adolescents with various diseases.

Working with Stella BIO allows us to analyse the patient's problem very precisely, provide more effective therapy and precisely monitor progress. The therapist can obtain feedback on muscle activity even when their contraction is not visible to the naked eye. Thanks to detailed reports, we can clearly present our work results to the parents of our clients and prepare specific goals in the therapeutic process. We mainly work on EMG Biofeedback programs, electrostimulation, improving circulation and games which are a big attraction for the youngest.

The program's operation is simple, clear, and allows you to modify your activity during a given session. A big advantage is the ready-made programs and for more individualization of therapy, the possibility of modifying it and adapting it to the patient's needs.

The Stella BIO device in our facility is the beginning of modern physiotherapy and we recommend it for both every physiotherapist and patient.

Translated by Clinical Specialist @ EGZO Tech
Alicja Nogala

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5.1.3.25. Municipal Hospital No. 4 in Gliwice

Opinion

“Working with the Stella device has enabled many patients to achieve better rehabilitation results in a shorter period of time.[...] The ability to generate reports on an individual patient card significantly facilitates maintaining medical records.”

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The Stella BIO device from Egzotech was used in the rehabilitation department in April this year.

In the department, we mainly deal with orthopaedic patients, but there are also people with neurological diseases (paresis, paralysis, people after strokes).

Working with the Stella device has enabled many patients to achieve better rehabilitation results in a shorter period of time. We were able to constantly monitor progress both in achieving better ranges of joint mobility and in increasing muscle strength. The ability to generate reports on an individual patient card significantly facilitates maintaining medical records and gives the patient the opportunity to view the achieved effects of therapy. Patients were happy to use the device and the possibility of selecting individual therapy for each of them was very well-received.

We are very grateful for the opportunity to use the Stella BIO device. Every centre should have the opportunity to use the device to see what a range of possibilities it offers for individual work with the patient.

Translated by
Clinical Specialist @ EGZOTech
Alicja Nogala

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ANNA REGULA
mgr fizjoterapii

PWZP nr 45040

5.1.3.26. University Clinical Hospital WAM-CSW in Łódź

Clinical Opinion

“STELLA BIO is one of the most innovative and practical devices that we have had the opportunity to work with in our Clinic. Observations so far allow us to conclude that not only patients highly appreciate working with STELLA BIO, but also therapists notice its advantages.”

Dr Tomasz Nowakowski

The STELLA BIO device has been used in the Department of Internal Diseases, Rehabilitation and Physical Medicine of the WAM University Clinical Hospital in Łódź since August 2023. It functions as part of the program of physiotherapy treatments, diagnostics and is used in teaching classes with students. STELLA BIO is used to work with various types of patients, mainly neurological patients (patients after strokes, MS, spinal cord injuries, neurosurgical operations, brain-cranial injuries, Parkinson's disease and others) as well as orthopaedic patients (after fractures, hip arthroplasty, knee arthroplasty and other injuries to the musculoskeletal system). The greatest experience so far in the work of STELLA BIO concerns stroke patients with weakened muscle strength and pain patients.

The use of STELLA BIO enables work both in the lower and upper limbs. Due to the large number of programs, neurological, orthopaedic and pain relievers, it is used in a wide range. It is characterised by high flexibility in working with the patient. Selecting the appropriate electrostimulation program is intuitive, which undoubtedly facilitates working with the patient. The software attached to the STELLA BIO electrostimulator guides the therapist in a clear and transparent way, step by step, on how to properly perform a given treatment. Recalibration possible even during the procedure, allows you to tailor the training to the patient with a specific rehabilitation program.

The greatest advantage of STELLA BIO are programs using electromyography, functional electrostimulation (FES) and biofeedback. Thanks to FES, in programs such as "Finger to Mouth", patients with muscle strength of 1 or even 0 on the Lovett scale can be improved. The patient performs the tasks assigned to him by muscle activation of appropriate muscle groups, through appropriately selected parameters. This has a very positive effect on motivation and improvement of the mental condition of patients using these programs. The EMG-triggered FES program is also extremely useful, as it allows for a more selective strengthening of a specific muscle group that is the source of the patient's functional impairment.

To sum up, STELLA BIO is one of the most innovative and practical devices that we have had the opportunity to work with in our Clinic. Observations so far allow us to conclude that not only patients highly appreciate working with STELLA BIO, but also therapists notice its advantages, which complement the basic rehabilitation program, including supplementing work with the LUNA EMG device that has been in use for over a year. Both devices are currently used as part of the so-called Combination therapy in some patients. The positive effects seem very promising.

Translated by
Clinical Specialist @ EGZOTech
Alicja Nogala

Headquarters:

Independent Public Health Care Center University Clinical Hospital Military Medical Academy -
Central Veterans Hospital
91-647 Łódź,
Haller's Square 1

The person issuing the reference:

Tomasz Nowakowski, MD, PhD - physiotherapist, physiotherapy specialist

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5.1.3.27. Masovian Rehabilitation Center STOCER

“STELLA BIO is one of the most innovative and practical devices that we have had the opportunity to work with in our Clinic. Observations so far allow us to conclude that not only patients highly appreciate working with STELLA BIO, but also therapists see its advantages.”

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In our Center, the priority is the skills of the therapeutic team. If we add the possibility of using the latest technologies in the rehabilitation process, then the experience of our staff is used even more effectively. Stella BIO is one such tool. Thanks to excellent motor assessment, interactive rehabilitation and the ability to monitor progress achieved, the rehabilitator receives help in developing individual therapeutic plans and the patient receives clear feedback. The ability to monitor the progress achieved allows the entire rehabilitation process to be linked into a logical and easy-to-interpret whole.

When exercising with children, this is an invaluable help. The feedback received and its interpretation enable us to guide young patients in such a way that the final result of their stay in the ward is as effective as possible for them. Our hospital provides rehabilitation for neurology, orthopaedics, postoperative patients, and correction of posture defects, and in each of these cases we are able to use the Stella BIO device. Thanks to it, we will undoubtedly influence the stimulation of proprioceptive sensations, the increase in muscle mass and the range of mobility. What is important to us is the fact that the patient, feeling safe, can achieve his goal through play. Performing activities that were often previously impossible or simply boring results in improvement of both the mental and physical condition of our patients. That is why we often reach for and use Stella BIO in therapy.

Translated by
Clinical Specialist @ EGZOTech
Alicja Nogala

Headquarters:

Masovian Rehabilitation Center
STOCER
Department of Postural Defects,
Scoliosis and Rehabilitation of Musculoskeletal Disorders for Children and Adolescents
05-510 Konstancin Jeziorna
street Długa 40/42

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5.1.3.28. Lunula Birthing School

“Both the assessment and exercises using the device enable the tracking of progress and serve as significant motivation for patients. Stella BIO is a valuable complement to my therapy.”

MSc Agata Bork-Ceszlak

Opinion

I have been using the Stella BIO in my clinic since July 2023. On a daily basis, I work with patients experiencing urogynecological issues, as well as pregnant and postpartum women. I use the device mainly in the treatment of urinary incontinence and weakened pelvic floor muscles.

I most often use biofeedback exercises. Thanks to the feedback information, patients have the opportunity to observe the performance of their muscles. This results in better control, increased muscle strength, and improved sensation within the pelvic floor. Additionally, I implement assessment programs, particularly the Glazer protocol. The ability to modify parameters allows me to tailor it individually to each patient and obtain precise information about their capabilities.

Both the assessment and exercises using the device enable the tracking of progress and serve as significant motivation for patients. Stella BIO is a valuable complement to my therapy.

Translated by
Clinical Specialist @ EGZOTech
Alicja Nogala

Headquarters:

Lunula Birthing Clinic and School,
street Wojska Polskiego 36E
77-100 Bytów

The person issuing the reference:

MSc Agata Bork-Ceszlak - owner, midwife, obstetrics and gynecological nursing specialist

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Bydgoszcz, 15.03.2021.

Opinion on the Stella BIO device

I have been using Stella BIO from EGZOTech since 15.01.2021. In the past, I have suffered a brain stem hemorrhagic stroke and I currently have epilepsy.

I use Stella BIO daily in combination with my existing rehabilitation. The device is equipped with both electrostimulation and EMG. Each workout is prepared remotely by a physical therapist, specifically for me.

I mostly use electrostimulation with individually adjusted parameters ("EMS User Program"), the combination of electrostimulation with EMG ("Reactive EMS (1 channel)") and various types of EMG programs ("Tension-relax" "5s/10s", "5s/10s"). I like "Burger Mania" and "Slice and Dice" EMG games in particular, during which I have to flex or relax my muscles appropriately to score points.

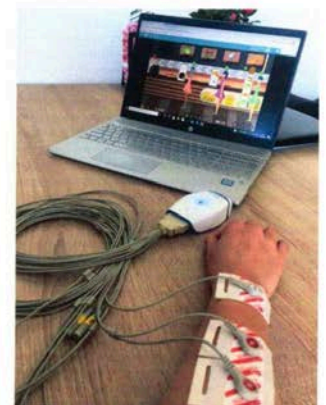
I observe the activity of my muscles on the screen of my laptop, and this allows me to adjust the moment of muscle contraction and relaxation, as well as monitor the entire rehabilitation process.

After a month of using Stella BIO, my arm was more relaxed and I could even put it behind my head, touching the other shoulder, something I could not do before. It is now easier for me to lift my arm up or straighten it at the wrist. Previously, I did not use my paretic hand or simply forgot about it. Since I already see the results, I use it whenever I can. I would like to improve my hand and fingers even more, so I continue exercising and look forward to further results.

Although it is not easy to achieve improvements after a hemorrhagic stroke, this is a device providing actually visible results. I also enjoy the fact that I can use it at home, with remote supervision by a physical therapist.

“Although it is not easy to achieve improvements after a hemorrhagic stroke, this is a device providing actually visible results. I also enjoy the fact that I can use it at home, with remote supervision by a physical therapist.”

Olimpia Czaplewska
patient after stroke

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Warschau, den 27. September 2021

Opinion on the Stella BIO device

“After 2 months of using the device, I can report a significant improvement in limb mobility, as well as some improvement in sensation.

I am positive about the effects of the Stella BIO device and would be happy to continue this treatment.”

Aleksander Sław
patient after stroke

I used the Stella BIO device for 13 months after I suffered a stroke. Over the course of the stroke, I lost 90% of the sensation in my limbs on the left side of my body. The sensations of touch, warmth and pain were disturbed. I underwent a therapy using PNF and Bobath methods for a year. As a result of the therapy, I was able to partially regain my motor skills and mobility. This was followed by treatment with EGZOTech's Stella BIO device. It involved the lower extremity. Electrode placement was not a technical problem. Treatment was done according to protocol, once a day. There were no problems with using the device. After 2 months of using the device, I can report a significant improvement in limb mobility, as well as some improvement in sensation.

I am positive about the effects of the Stella BIO device and would be happy to continue this treatment.

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Opinion

I had the opportunity to use the Stella BIO EMG device from EGZOTech. The Stella BIO EMG device was equipped with 4 (four) EMG channels and 2 (two) extension channels for pelvic muscle stimulation.

I used the device from December 2020 to May 2021. I completed exercises with with this device every day, using exercises planned specifically for me. This included electrostimulation of the tibialis anterior muscle, and a strength training program for the biceps muscle of both limbs. I also used EMG Biofeedback options for the tibialis, quadriceps, and abdominal muscles.

There is a noticeable increase in motor skills. In addition, there is also muscle tissue growth. I notice this mostly in cases of the muscle parts involved in training and stimulation on a regular basis. Exercises forced me to move regularly, to stimulate my muscles, to get out of my zone of stillness. My condition has improved significantly compared to before exercising with the Stella BIO device.

I have a very positive opinion of passive exercises: electrostimulation. These exercises made me feel better and had a positive effect on the activity and appearance of my muscles. During the exercise, and afterwards, I felt better, with more energy. I felt my own body more intensely. I definitely had more control over it. Electrostimulation has helped me recover faster and more efficiently from an exacerbation of my disease (relapse). After I left the hospital, I did not do full exercises. I focused only on passive exercises and they definitely accelerated my rehabilitation. I am sure that thanks to Stella BIO device I recovered faster, and most importantly – I regained my mobility.

I would like to recommend the Stella BIO EMG device. This is an additional important tool in the rehabilitation of the entire body. I will definitely choose rehabilitation institutions that have Stella BIO equipment. I find this device highly usable.

“I am sure that thanks to Stella BIO device I recovered faster, and most importantly – I regained my mobility. I would like to recommend the Stella BIO EMG device. This is an additional important tool in the rehabilitation of the entire body. I will definitely choose rehabilitation institutions that have Stella BIO equipment. I find this device highly usable.”

Krzysztof Man
patient with multiple sclerosis

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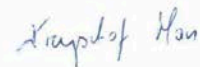
5.1.4.4. Sylwia Gdynia

6.10.2021 Knurów

“I have been exercising regularly at least once a day, and I have no problems with urinary incontinence during running or strength training at the moment. I recommend Stella Bio to every woman after childbirth, and also to those friends who have these "embarrassing" problems, as everything can be fixed.”

Sylwia Gdynia
patient with SUI

I gave birth to three children, two of them by natural means and the third by a C-section. My first pregnancy weakened and strained my pelvic floor and abdominal muscles, but I was able to exercise alone (without assistance) and get satisfactory results. After my third child (the difference between my older children and my youngest one is 11 years), as a 39-year-old person, it was definitely harder for me to get back to my condition from before the pregnancy. Ania Poświata and the Stella Bio device provided support with this. My pelvic floor muscles were severely weakened, because I had trouble maintaining my urinary continence, which was especially troublesome during longer runs and when playing with children, where movements were fast. The trouble occurred when jumping with a partially filled bladder. Ania conducted a detailed interview with me, took appropriate measurements (a test of muscle condition) and explained how the device operates and how the process will look like. And also what results can be expected.

She initially set the training plan for three months. In the beginning, the exercises were short to provide adaptation, and performed once a day, then the exercise time was extended, with the possibility of exercising twice a day. I noticed the first results already after a week, when after running 7 km my underwear was slightly damp. After a period of about three weeks, my underwear was completely dry after short runs – up to 10 km. After about 2 months, the incontinence during jumping or long runs completely subsided. I have been exercising regularly at least once a day, and I have no problems with urinary incontinence during running or strength training at the moment. I recommend Stella Bio to every woman after childbirth, and also to those friends who have these "embarrassing" problems, as everything can be fixed. I still exercise with great pleasure, not as regularly, but still often enough to maintain the effects I have achieved. Both the device and the team taking care of the patient are great.

Thank you for taking care of me and adjusting my training plan to both my physical and time capabilities.

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5.1.4.5. Agnieszka Flaszka

“Just 3 months of work, and the problem was gone. And working with Stella means evening entertainment with games. It is a lot of fun when you make a dinosaur jump using your pelvic floor muscles. That is exactly how it works, there is a huge selection of games that make exercise a hilarious and wonderful experience.”

Agnieszka Flaszka
patient with SUI

Opinion on the Stella BIO device

I used Stella after a long stay in the department of pregnancy pathology. My problems started after giving birth. Pelvic pain and incontinence. It was hard for me to find help, as this is an embarrassing problem. When I came across the pelvic floor muscle physiotherapy leaflet, I felt a huge sense of relief. It turned out that my problem was not unusual. After my initial examination and with plenty of good advice, I went home with Stella.

Just 3 months of work, and the problem was gone. And working with Stella means evening entertainment with games. It is a lot of fun when you make a dinosaur jump using your pelvic floor muscles. That is exactly how it works, there is a huge selection of games that make exercise a hilarious and wonderful experience.

There is also the support of the EGZO Tech team, which prepares an exercise plan every day, lets you know to do a progress test at the right moments and directs the work.

And the best outcome from this encounter is health, freedom from pain and embarrassment.

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Bardzo polecam!

5.1.4.6. Beata Król-Kowalik

“Stella provides you the opportunity to combine the pleasant with the useful. The child treats Stella as a game and a pleasant time, yet it still is an exercise tool.”

Beata Król-Kowalik
*mother of a patient with
dystrophy*

Opinion on the Stella BIO device

Stella provides you the opportunity to combine the pleasant with the useful. The child treats Stella as a game and a pleasant time, yet it still is an exercise tool. Games are controlled by tensing and relaxing muscles. The patient also learns about their body and how their muscles work through this process. Thanks to this device, it is possible to also work with muscle parts which are difficult to activate in a patient during active rehabilitation, e.g., when the patient is unable to perform a given movement. Stella also has typical exercise programs. A small, cool device that serves as additional form of stimulation and a supplement to traditional rehabilitation.

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5.1.4.7. Marcin Bartlewicz

“The "Stella" software is very intuitive. In a matter of minutes, I was able to navigate the app on my own, register new patients, and order personalized exercise sessions.”

Marcin Bartlewicz
*conducting research with the use
of Stella BIO*

I learned about StellaBIO when there was an opportunity to conduct scientific research with the device. The first impression after the presentation of "Stella" was very good, even when it comes to the very idea of developing a small, portable device allowing to receive feedback, both on whether the patient is using the correct muscle group during exercise, and whether the patient is exercising regularly. You can also see the effects of rehabilitation by collecting data from each exercise session.

The "Stella" software is very intuitive. In a matter of minutes, I was able to navigate the app on my own, register new patients, and order personalized exercise sessions. The ease of the software is important in terms of the patient performing the exercises at home by themselves. This allows patients to be cared for by physical therapists even during the COVID-19 pandemic.

As a new device of this type, StellaBIO is still undergoing dynamic development. Thanks to a well-tuned team, new ideas are quickly implemented, contributing to expanding the list of "Stella" advantages.

Marcin Bartlewicz
fifth-year Physiotherapy Student conducting research with the use of the Stella BIO device

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“The therapy addresses gluteal muscles that are weakened as a result of Muscular Dystrophy. I am positive about the results of the Stella Bio device so far, and I am eager to continue with further therapy.”

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Opinion on the Stella Bio device

I have been using the Stella device for 2 months. The therapy addresses gluteal muscles that are weakened as a result of Muscular Dystrophy. Using the device and placing the electrodes does not cause me any problems. Training sessions take place according to the prepared schedule - 3 times a week. I am noticing an improvement in the charts related to right gluteal tension.

I am positive about the results of the Stella Bio device so far, and I am eager to continue with further therapy.

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LINGUA LAB s.c.
W. Szyszkiewicz, M. Dembińska
ul. Sereno Fenni's 6/14, 31-143 Kraków
NIP 877-295-31-18 REGON 121429550

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29. 12. 2021

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5.1.4.9. Zygmunt Kaszyk

“My incontinence symptoms improved dramatically, including being in full control of my urination both day and night.”

Zygmunt Kaszyk
patient with SUI

13.12.2021 Katowice

Opinion on the Stella BIO device

“After my prostatectomy surgery, I attended rehabilitation where the training sessions were performed with an EMG machine, among other things. I learned how to properly tighten my muscles and watched my muscles contract on the screen. I really wanted to continue this type of exercise at home. I rented a Stella BIO machine from EGZOTech and trained 5 times a week. My incontinence symptoms improved dramatically, including being in full control of my urination both day and night.

My observations also suggest that with permanent incontinence, it is not advisable to stop exercising, because over time, the trained muscles weaken and the comfort associated with the bladder control deteriorates. Because of that, I strongly recommend regular training with Stella BIO.

I am glad I was able to practice on a professional piece of equipment, but still at home.”

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LINGUA LAB s.c.
W. Szyszkiwicz, M. Dembłńska
ul. Serego Fenia 6/14, 31-143 Kraków
NIP 577-235-31-16, REGON 121425533

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5.1.4.10. Katarzyna Kaczor

“Leon has been exercising with the Stella device 3 times a week since November. He is working on reinforcing his quadriceps femoris muscles. We are very pleased with the StellaBio device. Leon exercises eagerly, it provides a nice variety added to heavy physiotherapy exercises.”

Katarzyna Kaczor
mother of a patient with muscular dystrophy

Piaseczno 28.12.2021

My name is Katarzyna Kaczor and I am a mother of 9-year-old Leon Wiśniewski, who suffers from Ullrich muscular dystrophy. It is a rare, genetic disease leading to joint contractures and muscle weakness over the course of its progress.

Leon has been exercising with the Stella device 3 times a week since November. He is working on reinforcing his quadriceps femoris muscles, and while it is difficult to see any spectacular results with this condition, we are very pleased with the StellaBio device. Leon exercises eagerly, it provides a nice variety added to heavy physiotherapy exercises.

The only downside is that the adhesive at the electrodes is too strong, causing irritation to Leon's delicate skin, and peeling it off is quite painful.

Nonetheless, we are very pleased with the device and hope to continue the therapy, which will be able to show the device's effects more meaningfully.

LINGUA LAB s.c.
W. Szyszkiewicz, M. Dembińska
ul. Sereno Fanni'a 5/14, 31-143 Kraków
NIP 577-225-31-15 REGON 121426033

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5.1.4.11. Ewelina Rajewicz

“The possibility of using Stella BIO by the patient at home independently is a great advantage of the device. The physical therapist remotely records the patient's workouts, and assigns them to mutually agreed dates. They can then monitor the performance of the training session, according to the schedule, and modulate them in real time after receiving feedback from the user. ”

Ewelina Rajewicz, *physiotherapist*

Warsaw, 02.01.2022

Opinion on the Stella Bio device

I have been using the Stella BIO device in my work for over six months now. The main function I use is EMG Biofeedback.

The only requirements for using Stella BIO are Internet access and a computer with Bluetooth functionality. The device is intuitive and easy to use, however, on several occasions, patients have reported a problem configuring the Stella BIO with the user's computer.

Due to the large number of programs, it is possible to use it in both neurological and orthopedic diseases. Stella BIO has pre-set training protocols, but there is also the possibility for physiotherapist to select individual parameters.

The possibility of using Stella BIO by the patient at home independently is a great advantage of the device. The physical therapist remotely records the patient's workouts, and assigns them to mutually agreed dates. They can then monitor the performance of the training session, according to the schedule, and modulate them in real time after receiving feedback from the user. The program provides a concise EMG analysis that can be downloaded and included in your records in report form. This allows you to monitor the progress and effectiveness of your therapy. It is also possible to schedule training in the form of a game; this therapy type is liked, especially by younger patients.

The ability for the patient to observe their muscle activity in real time increases their sense of empowerment, increasing their motivation to exercise. This form of therapy positively influences the patient's involvement and co-responsibility in the treatment process.

In conclusion, I find Stella BIO to be a valuable therapeutic tool with a wide range of applications.

Ewelina Rajewicz
Master of Physiotherapy

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"I use the device to this day, in the first stage of rehabilitation I used factory programs for atrophy, improvement of circulation and regeneration. Currently, I use the option of "manual" defining training parameters (which was missing in other devices I familiarised with after injury). "

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Opinion on the EGZOTECH Stella Bio device

Warsaw, May 18, 2023

I got injured during the competition (August 2022) - rupture of the anterior cruciate ligament (ACL) and rupture of the medial meniscus. After the operation (August 10, 2022) usual muscle atrophy of the quadriceps muscle occurred (the operated leg was thinner by about 13 centimeters after 2 operations). Due to a several-week ban on weight-bearing on the leg, I couldn't perform exercises, after visiting the CRF Orthos, I was recommended electrostimulation, for which I used the Stella device.

I use the device to this day, in the first stage of rehabilitation I used factory programs for atrophy, improvement of circulation and regeneration. Currently, I use the option of "manual" defining training parameters (which was missing in other devices I familiarised with after injury). Stella has the ability to monitor the "contraction force" by registering the reception from the contracting muscle, thanks to which you can "operatively" control the contraction force of the right and left leg on the chart. In addition, it is a multi-channel device, i.e. using it on several muscles at the same time. The size of the device and the strap to attach it to the waist (and other places) allows you to take the device anywhere (where we have access to the application and wifi network). The device is easy to use and starts training in literally 3 minutes.

Additionally I noticed many other interesting options: for patients with neurological diseases, pain syndromes and for pediatric patients (from what I've seen, there are even games for the little ones). Currently, after 8 months of intensive rehabilitation and training, in which stella occurred frequently, the difference between the left and right leg is only 1 cm.

I definitely recommend the device to both athletes and other patients. It is a very interesting and modern equipment. Thanks to the equipment's knowledge of therapists I worked with, I could use it in my recovery.

Translated by Clinical Specialist @ EGZOTech
Alicja Nogala

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5.1.4.13. Karolina Kwiatek-Rolka

“The Stella BIO device for EMG and EMS stands out for its solid construction and functionality, making it an excellent choice for both physiotherapists and patients. Its compact dimensions ensure ease of use, and the simple control panel allows for quick adaptation to the needs of various users.”

Karolina Kwiatek-Rolka, *Master of Physiotherapy, lecturer at “Perinea - School of Perineology”*

Opinion - Stella BIO

The Stella BIO device for EMG and EMS stands out for its solid construction and functionality, making it an excellent choice for both physiotherapists and patients. Its compact dimensions ensure ease of use, and the simple control panel allows for quick adaptation to the needs of various users.

One of the device’s main advantages is its versatility. It can be used in a wide range of therapies, from orthopedic issues and post-injury rehabilitation to applications in urogynecology. This versatility makes it a valuable tool across different medical fields.

An additional benefit is the inclusion of games, which are particularly useful in pediatric therapy. These make treatment more engaging and less stressful for young patients, positively influencing their motivation to continue therapy.

Another key feature is its wireless design and comfortable strap, providing patients with freedom of movement—a frequent issue with other devices that rely on numerous cables. Reduced risk of tangled wires or delays in setup enhances the efficiency and flow of therapy sessions.

In summary, Stella BIO is a functional, user-friendly device with significant therapeutic potential, particularly in orthopedic and urogynecological therapy.

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Krzysztof Rolka-Kawaliński

5.1.4.14. Martyna Romanowska-Naimska

“What we find particularly outstanding about this device is its user-friendly and intuitive interface, highly sensitive EMG signal, convenient size for easy transport, and most importantly, the overwhelmingly positive feedback from our patients! Therapy results are achieved much faster, and the playful format of the built-in therapeutic programs makes Stella a favorite, even among our youngest patients”

Martyna Romanowska-Naimska,
*Master of Physiotherapy, lecturer
at “Perinea - School of
Perineology”*

We have been working with the Stella BIO device since 2022, primarily using it for the diagnosis and therapy of pelvic floor disorders such as urinary incontinence, overactive bladder, and pelvic or perineal pain.

What we find particularly outstanding about this device is its user-friendly and intuitive interface, highly sensitive EMG signal, convenient size for easy transport, and most importantly, the overwhelmingly positive feedback from our patients! Therapy results are achieved much faster, and the playful format of the built-in therapeutic programs makes Stella a favorite, even among our youngest patients.

An especially interesting and useful feature is the option to use the telemedical portal, which allows for planning, programming, and monitoring therapy progress, enabling patients to continue their treatment independently at home.

Another important aspect is the excellent after-sales service, with the device being serviced in Poland. The great communication ensures that we, as clients, feel well-supported, and our needs and suggestions are always heard.

We highly recommend the Stella device to both beginner and experienced therapists.

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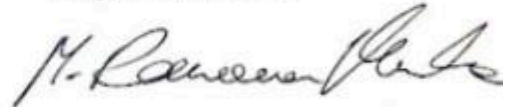
6.1. European Clinical Facilities

6.1.1. Poland

6.1.1.1. Independent Public Health Care Complex in Lesko

“Rehabilitation using EMG is a big step in the development of our department. Sidra LEG undoubtedly improves the quality of life of our patients and accelerates their return to health and fitness.”

MSc Daria Winiarczyk

Opinion

The Rehabilitation Department with the Sub-Department of Neurological Rehabilitation at SP ZOZ Lesko had the opportunity, thanks to the Ministry of Health program "Rehabilitation. Robots 2023", to obtain funding for the purchase of specialized devices, i.e. the LUNA EMG robot with the functionality of electromyography-assisted movement and Sidra LEG splints for lower limb exercises.

SP ZOZ in Lesko provides rehabilitation to patients at the Rehabilitation Ward with the Neurological Rehabilitation Sub-Department, Day Rehabilitation and patients using Outpatient Physiotherapy.

Thanks to the robots, the facility has expanded the range of treatments offered and has become more modern. These devices allow our facility to develop and help achieve better results. Working with Luna EMG and Sidra LEG allows for monitoring the progress of patients, motivates them to continue working, and the improvement process has been diversified.

The Sidra LEG mobile lower limb rehabilitation robot enables rehabilitation through passive, active-resistance, active-assisted and assisted-electromyographic exercises, both in the rehabilitation room and in the patient's bed. We work with patients after orthopedic procedures and with neurological patients. The most frequently used program is the assisted movement of the lower limb triggered by electromyography and electrostimulation. Thanks to the Sidra LEG splint, our patients can easily increase muscle strength, range of joint mobility and muscle activity. Working with this device gives the possibility of adapting programs to the patient's capabilities, through the intensity of rehabilitation or increasing the number of repetitions. Rehabilitation using EMG is a big step in the development of our department. Sidra LEG undoubtedly improves the quality of life of our patients and accelerates their return to health and fitness.

Programs from the Ministry of Health, which co-finance the purchase of modern rehabilitation equipment, are needed, provide valuable support to Polish rehabilitation departments and should be continued, both for the good of hospitals and patients.

We are grateful to EGZOTech for the equipment provided, training and good cooperation.

Headquarters:

District Hospital in Lesko
Kochanowskiego 2,
38-600 Lesko

The person issuing the reference:

MSc Daria Winiarczyk - Head of Rehabilitation Department

Translated by
Clinical Specialist @ Egzotech
Alicja Nogala

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6.1.1.2. Ludwik Rydygier Specialist Hospital

Opinion

“The supplied equipment significantly supports the work of physiotherapists, enabling adjustment of parameters and activity during exercises on the Sidra LEG device, which are impossible to achieve on a traditional CPM rail. It enables rehabilitation in a way that is difficult to achieve without the help of modern technologies.”

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The Neurological Rehabilitation Department with the Systemic Rehabilitation Unit of the Ludwik Rydygier Hospital in Krakow would like to inform you about the acquisition of the Sidra LEG medical device, thanks to the Ministry of Health program "Rehabilitation Robots 2023", which has had a positive impact on patient therapies in the department.

The Neurological Rehabilitation Department with the Systemic Rehabilitation Unit provides a wide range of services in the field of health disorders in neurological and orthopedic patients. With the patient's well-being and the quality of services provided in mind, the entire organizational team of the medical unit is involved in the care of patients.

The range of services within the scope of inpatient care includes comprehensive rehabilitation exercises, increasing the range of locomotion, physical treatments, and individual therapy with the patient.

The supplied equipment significantly supports the work of physiotherapists, enabling adjustment of parameters and activity during exercises on the Sidra LEG device, which are impossible to achieve on a traditional CPM rail. It enables rehabilitation in a way that is difficult to achieve without the help of modern technologies. It enables precise monitoring of patients' progress over time, and provides an interesting variety in therapy for patients. The device allows for precise and individual selection of therapy parameters, to the needs of each patient. Thanks to the appropriate training of the physiotherapy staff, treatments using Sidra LEG are carried out efficiently and effectively.

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6.1.1.3. Rehabilitation Center "Kwitnaça"

"The Sidra robot supports and controls the selected gait phases or movement in the selected joint and its range of motion. Thanks to the possibility of using electromyography and electrostimulation, we can adjust the level of support to the individual needs of the patient."

MSc Monika Wilk

Rehabilitation Center "Kwitnaça"

- user opinion on working with the SIDRA Leg robot

The Sidra device is a device supporting the rehabilitation of lower limbs.

It is used in the therapy of people after injuries, orthopedic or neurosurgical surgeries, as well as in the treatment of people with neurological diseases that affect the dysfunction of the lower limbs.

The Sidra robot supports and controls the selected gait phases or movement in the selected joint and its range of motion.

Thanks to the possibility of using electromyography and electrostimulation, we can adjust the level of support to the individual needs of the patient.

It allows personalization of the rehabilitation plan, monitoring the patient's progress and adjusting therapy parameters.

Thanks to the possibility of using the device in passive mode, supported by the EMG signal with muscle electrostimulation and active exercises, we can work with patients in all phases of convalescence, regardless of the degree of damage or limitation of the function of the lower limbs.

The variety of programs proposed in the device allows you to plan the therapy in a way that is attractive to the patient.

The device is easy to use, helpful in everyday work with the patient. The robot reduces the physical burden of physiotherapists and helps to systematize data on the range of motion and muscle strength, which makes it easier to monitor the effects of the therapy and adjust the plan for further rehabilitation to the individual needs of the patient.

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Headquarters:

Specialist Inpatient Rehabilitation and Long-Term Care Centre "Kwitnaça"
Kwitnaça 3,
32-091 Zagórzycze Dworskie

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6.1.1.4. Independent Public Health Care Facility in Rypin

"[...]we can work with patients in various stages of rehabilitation, regardless of the degree of damage or limitation of lower limb function. The variety of programs proposed in the device allows you to plan therapy in a way that is attractive to the patient."

MSc Patryk Żbikowski

Opinion

The Rehabilitation Department at SP ZOZ Rypin had the opportunity, thanks to the Ministry of Health program "Rehabilitacja Roboty 2023", to obtain funding for the purchase of a specialized device, i.e. Sidra LEG splint for lower limb exercises.

Thanks to the robots, the facility has expanded the range of treatments offered and has become more modern. This device allows our facility to develop and helps the patient achieve better results.

The Sidra LEG mobile lower limb rehabilitation robot enables rehabilitation through passive, active - resistance, active - assisted and assisted - electromyographic exercises both in the rehabilitation room and in the patient's bed, thanks to which we can work with patients in various stages of rehabilitation, regardless of the degree of damage or limitation of lower limb function. The variety of programs proposed in the device allows you to plan therapy in a way that is attractive to the patient.

The device allows for precise and individual selection of the therapy parameters to the needs of each patient. Thanks to the appropriate training of the physiotherapy staff, treatments using Sidra LEG are carried out efficiently and effectively.

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Headquarters:

Independent Public Health Care Facility in Rypin
ul. 3 Maja 2,
87-500 Rypin

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6.1.1.5. Prof. B. Frańczuk Malopolska Orthopaedic and Rehabilitation Hospital

“The Sidra Leg device is an innovative device that allows for quick muscle activation, which distinguishes it from the traditional CPM rail. This functionality significantly affects the faster recovery of patients after surgical interventions in the lower limbs.”

Dr Joanna Bukowska

The Department of General Rehabilitation of the Prof. B. Frańczuk Malopolska Orthopaedic and Rehabilitation Hospital in Krakow had the opportunity to use the Sidra Leg medical device in its therapeutic work. The use of the CPM splint is standard in the rehabilitation of patients after knee joint alloplasty. A significant number of patients in our department are people in the early stages of rehabilitation after orthopaedic procedures in the knee joint. In order to regain the range of motion and quick muscle activation, the following were used, among others: passive exercises performed mechanically, passive exercises initiated by EMG and isokinetic exercises using the Sidra Leg device.

The Sidra Leg device allows for individual selection of therapy parameters, i.e. angular values, speed of movement and resistance during exercise. Simultaneous work of several joints of the limb while working on the splint significantly improves the function of the entire lower limb. After appropriate training, the use of the device is simple and intuitive, but it takes more time than the traditional CPM rail due to the multitude of therapeutic programs. The ability to save therapy parameters for a specific patient shortens the time of programming exercises.

The Sidra Leg device is an innovative device that allows for quick muscle activation, which distinguishes it from the traditional CPM rail. This functionality significantly affects the faster recovery of patients after surgical interventions in the lower limbs.

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The person issuing the reference:

Dr Joanna Bukowska - PhD in Physiotherapy, Physiotherapy Specialist, Head of the Physiotherapy Department

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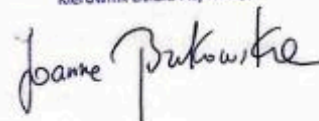
6.1.1.6. Masurian Health Center in Węgorzewo

Sidra LEG robot

“Sidra LEG is a valuable tool that greatly supports rehabilitation, particularly for patients with orthopedic and neurological conditions”

MSc Łukasz Popławski

The Sidra LEG robot is a technologically advanced device that supports the rehabilitation process in our facility on a daily basis. We use it in the therapy of orthopedic and neurological patients, including individuals with ligament and meniscus injuries. Thanks to its extensive capabilities, the robot allows for the implementation of various therapeutic programs. We frequently use the CAM Isokinetics mode, which is highly appreciated by patients, enhancing their motivation and engagement in exercises.

Sidra LEG significantly aids the rehabilitation process, delivering measurable results such as increased muscle strength, improved range of motion, and enhanced coordination. However, we recognize areas for improvement, such as enhancing the quality of attachments, reducing the number of malfunctions, and expanding the device's library of therapeutic games. Automating certain settings could also improve user convenience.

In terms of after-sales support, the EGZOTech team stands out for its professionalism, providing comprehensive training, high-quality educational materials, and fast, effective communication.

In summary, Sidra LEG is a valuable tool that greatly supports rehabilitation, particularly for patients with orthopedic and neurological conditions.

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Headquarters:

Masurian Health Center - District Hospital
3 maja street 17,
11-600 Węgorzewo

The person issuing the reference:

MSc Łukasz Popławski - head of the team of physiotherapists

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7.1.1.1. Kutno Municipal Hospital

Opinion

“The MEISSA OT robot affects the fitness of patients, but also their cognitive abilities. It is a modern device that enriches the work of physiotherapists and doctors treating patients in various rehabilitation facilities.”

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The Rehabilitation Department with the Neurological Rehabilitation Unit at the Kutno Local Government Hospital, thanks to the Ministry of Health program "Rehabilitation. Robots 2023" has been equipped with the MEISSA OT device.

The possibilities offered by the device's programs significantly improve the fitness of patients.

The device increases the strength of hand muscles when it is weakened, improves coordination, manual function. It contributes to the diversification of the occupational therapy program for patients with peripheral nerve damage, after orthopedic procedures, as well as for patients with rheumatic diseases.

The variety of extensions and the versatility of the programs in this robot, as well as the enrichment of the program with therapeutic games, are a significant attraction for patients accustomed to the traditional form of rehabilitation in other facilities.

The MEISSA OT robot affects the fitness of patients, but also their cognitive abilities. It is a modern device that enriches the work of physiotherapists and doctors treating patients in various rehabilitation facilities.

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Kutno Local Government Hospital
Tadeusza Kościuszki 52,
99-320 Kutno

The person issuing the reference:

Katarzyna Medyńska, M.D. - Head of the Rehabilitation Ward with the Neurological Rehabilitation Unit
Katarzyna Sośnierz-Piotrowska, M.A. - Physiotherapy Specialist, Head of Physiotherapists

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7.1.1.2. Rehabilitation Center "Kwitnacja"

“The offered interactive elements, such as games and simulations, make rehabilitation an attractive element of daily exercises. The device is intuitive and easy to use, relieves physiotherapists in their work and provides opportunities for creative therapy with the patient.”

MSc Monika Wilk

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- user's opinion on working with the MEISSA OT robot

The Meissa OT robot is used for rehabilitation and manual training, supports patients in the rehabilitation of upper limbs and provides a number of options for improving fine motor skills. It is particularly useful in the therapy of patients after strokes, injuries or with diseases affecting manual skills.

The integration of electromyography and electrostimulation in one device allows for precise tracking of the patient's progress, which allows for a better assessment of the effectiveness of the therapy and adjustment of the rehabilitation program to the individual needs of the patient, taking into account their level of functionality.

The robot allows the patient to perform precise, repeatable exercises in the selected range of motion, which is difficult to reproduce manually in daily sessions with a physiotherapist. Patients are happy to exercise with the robot because they receive immediate feedback on their progress, which affects their motivation and commitment to regular exercises.

The exercises given visualize the patients' daily activities that are possible for them to perform thanks to the support of the robot.

The offered interactive elements, such as games and simulations, make rehabilitation an attractive element of daily exercises.

The device is intuitive and easy to use, relieves physiotherapists in their work and provides opportunities for creative therapy with the patient.

Thanks to the ability to track and analyze EMG data, the robot becomes helpful in assessing the progress of therapy, planning and its effectiveness, which contributes to the effectiveness of patient rehabilitation.

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7.1.1.3. Masurian Health Center in Węgorzewo

“Meissa OT is an effective therapeutic tool that significantly increases patient involvement in the rehabilitation process, which makes it worth recommending to modern rehabilitation centers.”

MSc Łukasz Poptawski

Meissa OT robot

The Meissa OT robot, used in upper limb and hand therapy, provides valuable support in the rehabilitation process conducted in our facility. The device is particularly useful in the therapy of neurological patients, especially those after strokes. It allows to perform a variety of exercises that effectively increase muscle strength, improve range of motion and improve coordination. Thanks to modern functions, such as the CAM Isokinetics program, rehabilitation using the robot is more effective and attractive, and patients are eager to engage in the proposed exercises.

Operating the robot requires appropriate training, which is provided by the EGZOTech team. Additionally, the company offers solid after-sales support, which has a positive effect on working with the device. However, we notice some aspects that require improvement, such as the quality of the materials used. In addition, we have had to deal with several faults that required service intervention.

To sum up, Meissa OT is an effective therapeutic tool that significantly increases patient involvement in the rehabilitation process, which makes it worth recommending to modern rehabilitation centers.

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The person issuing the reference:

MSc Łukasz Poptawski - head of the team of physiotherapists

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